

675-24  
11/3  
P1131

FRONT PANEL

O-SYL  
DISINFECTANT . DETERGENT

GERMICIDAL \* Staphylocidal \* Pseudomonacidal \*Tuberculocidal  
FUNGICIDAL \* (Against pathogenic fungi)  
VIRUCIDAL \* (Against Influenza A<sub>2</sub>, Herpes simplex, Vaccinia, and  
Adenovirus Type 2)  
\*On Environmental Surfaces

ACTIVE INGREDIENTS:

o-Phenylphenol .....5.00%  
o-Benzyl-p-chlorophenol .....4.50%

INERT INGREDIENTS: .....90.50%  
TOTAL..100.00%

EPA Registration. No. 675-24


DANGER: KEEP OUT OF REACH OF CHILDREN  
Corrosive, Causes eye damage and skin irritation. Do not get in  
eyes, on skin or on clothing. Protect eyes and skin when handling  
concentrate. Harmful or fatal if swallowed. Avoid contamination  
of food. Rinse empty container thoroughly with water and discard  
it.

See First Aid statement and other precautions on side panel.

NET CONTENTS: ONE GALLON

National Laboratories  
Lehn & Fink Products Group  
Montvale, NJ 07645 Plant: Toledo, Ohio 43612

Not Reviewed. Registrant claims to be in accordance with  
Reregistration of Pesticide Product Guidance for Phase 2  
Response Page 2.12, Active to Inert Change in Status.

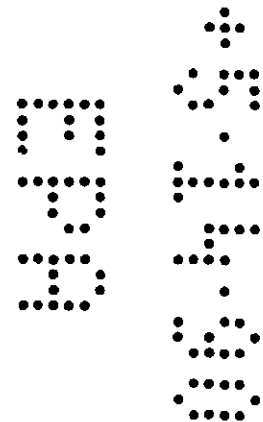

Right Side Panel

2 of 3

HOW TO USE HOSPITAL TYPE  
O-SYL  
DISINFECTANT . DETERGENT

O-SYL IS IN CONCENTRATED FORM, DILUTE ACCORDING TO DIRECTIONS.

ONE TABLESPOON EQUALS ABOUT 1/2 FLUID OUNCE OR 15 CC.

To prepare proper dilutions in required quantities, add full strength O-SYL to water as below:


PURPOSE	Correct Solution Strength	O-SYL	WATER
FOR GERMICIDAL CLEANING AND DEODORIZING. (Including premises occupied by tubercular patients) Floors, Walls, Furniture, Food and linen cars, Garbage cans.	0.78% (1:128)	8 cc.	1 quart
		30 cc.	1 gallon
		3 oz.	3 gallons
		5 oz.	5 gallons
FOR DISINFECTION OF: Thermometers, surgical and dental instruments.	3 % (3:100)	30 cc.	1 quart
		4 oz.	1 gallon
		12 oz.	3 gallons
		20 oz.	5 gallons

See brochure for detailed instructions.

FIRST AID:

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse.

If swallowed, drink promptly a large quantity of milk, egg whites, or gelatin solution. If these are available drink large quantities of water. Avoid alcohol. Call a physician immediately.

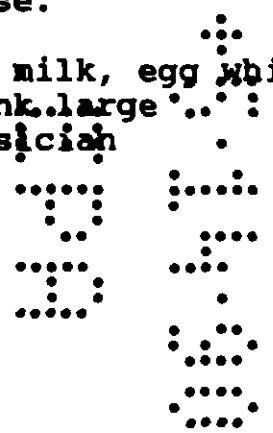

Left Side Panel

3/1/27

AMERICA'S FOREMOST MANUFACTURER OF  
DISINFECTANTS AND DISINFECTANT-DETERGENTS  
FOR ENVIRONMENTAL SANITATION

