

Front Panel

ONE GALLON

LIST NO 841


BARNES-HIND® GERMICIDAL SOLUTION


Active Ingredients: Alkyl (C₁₂ 65%, C₁₄ 25%, C₁₆ 10%)
dimethyl benzyl ammonium chloride 0.13%

Inert Ingredients: 99.87%

Contains sodium nitrite

Caution: Keep out of reach of children.
Avoid contamination of food.

 **BARNES-HIND LABORATORIES**
DIVISION OF BARNES-HIND PHARMACEUTICALS, INC.
Sunnyvale, California 94086 Made in U.S.A.

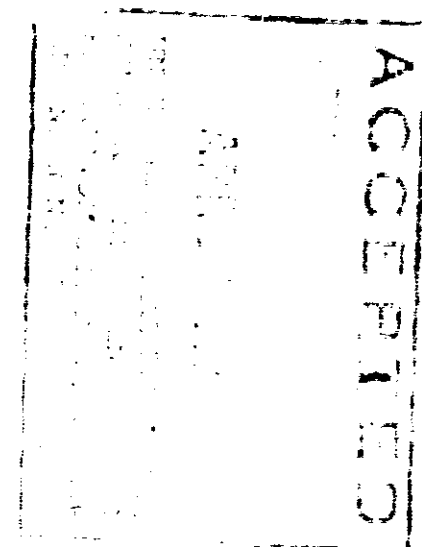

 BARNES-HIND PHARMACEUTICALS, INC.

Back Panel

GERMICIDAL SOLUTION

An efficient Buffered Germicide for the disinfection of Surgical and Dental Instruments

Ready to use • Non-rusting
Odorless • Stainless

This solution contains anti-rust agents

DIRECTIONS

Use full strength. Do not dilute. Clean and rinse items before submerging in solution. Soak at least ten minutes.

INDICATIONS

For disinfecting instruments of monel, stainless steel, surgical steel, nickel or chromium plated metals, and apparatus made of lucite, glass, polyethylene, bakelite, and rubber, including rubber catheters, gloves, etc. Storage of heat sterilized instruments, hypodermic equipment, needles, etc.

PRECAUTIONS

Aluminum-handled instruments should not be kept in solution for more than thirty minutes at a time. This solution is excellent for the storage of heat sterilized hypodermic needles. Prior heat sterilization is required as the inner lumen of the needles may not always come in contact with the solution.

Discard empty container after rinsing thoroughly with water.

EPA Reg No. 3559-2



BARNES-HIND LABORATORIES

DIVISION OF BARNES-HIND PHARMACEUTICALS, INC.
Sunnyvale, California 94086

Made in U.S.A.

03-0310-0273

10/18/73
3559-2



BARNES-HIND PHARMACEUTICALS, INC.