

1:750 Aqueous Benzalkonium Chloride solution for Disinfecting Clinical Thermometer

Active Ingredients: n-Alkyl (C₁₂-50%, C₁₁-40%, C₁₀-10%) Dimethylbenzylammonium Chloride 0.13%
Inert Ingredient 99.87%

EPA Reg. No. 10287-1

WARNING: KEEP OUT OF REACH OF CHILDREN

Net Contents 0.67 Fl. Oz.

DISTRIBUTED BY
AMERICAN HOSPITAL SUPPLY
DIVISION OF AMERICAN HOSPITAL SUPPLY
CORPORATION • MCGAW PARK • ILLINOIS 60059

Rinse vial with water before releasing to patient. See outer container for use directions.

EPA Est. 33695-IL-1

black
reproduction of printing
appearing on plastic (lucite)
screw cap vial,
cylindrical in shape,
3/4" x 3 3/4"
immediate container
of solution

Received in branch
actual kits - immediate
FEB 7 1975 containers presented.

FEB 2
10287-1



CAT. NO.
14590-030

TOMAC.
SENTRY™
THERMOMETER
KIT RECTAL

Contains 1:750 Aqueous Benzalkonium Chloride, a germicidal solution for the safe storage of your thermometer.

ACTIVE INGREDIENTS:
n-Alkyl* Dimethyl Benzyl Ammonium Chloride (Benzalkonium Chloride) 0.13%
*(C₁₂-40%; C₁₄-50%; C₁₆-10%)

INERT INGREDIENT
Water 99.87%
U.S.D.A. Registration No. 10287-3

Keep Out Of The Reach Of Children

LOT NO. _____

Net Contents 0.67 Fl. Oz.

DISTRIBUTED BY AMERICAN HOSPITAL SUPPLY,
DIV OF AMERICAN HOSPITAL SUPPLY CORP.,
EVANSTON, ILLINOIS 60201



CAT. NO.
14590-030

TOMAC.
SENTRY™
THERMOMETER
KIT RECTAL

CONTENTS:

- Thermometer
- Packet of Lubricant
- Prefilled Thermometer Jar*
- Holder and cover
- Adhesive square

*1:750 Aqueous Benzalkonium Chloride

SUGGESTED PROCEDURE:

- Remove contents from box
- Pull off plastic cover
- Remove thermometer
- Remove screw cap and discard liner
- Replace screw cap
- Remove release paper from one side of adhesive patch and apply to base for wall or surface mounting
- Mount unit in place
- Insert thermometer
- Replace plastic cover
- Replace with fresh solution when exhausted
- Clean thermometer after each use before replacing in solution

PATIENT NAME

ROOM NO.



CAT. NO.
14590-030

TOMAC.
SENTRY™
THERMOMETER
KIT RECTAL

CERTIFICATE OF EXAMINATION
We hereby certify that this registering Clinical Thermometer has been examined and tested and found to meet all requirements specified in the United States Department of Commerce Commercial Standard CS1-52 for Clinical Thermometers. This Thermometer has been aged a sufficient length of time to prevent further changes in its readings.

ACCEPTED
MAR 18 1968
UNDER THE FEDERAL INSECTICIDE AND RODENTICIDE ACT FOR ECONOMIC POISONED UNDER NO. 10287-3