



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY
WASHINGTON, DC 20460

OFFICE OF CHEMICAL SAFETY
AND POLLUTION PREVENTION

October 22, 2018

Ted McLaughlin
CEO
DiSorb Systems, Inc.
1800 W. Indiana Ave.
Philadelphia, PA 19132

Subject: Label and CSF Amendment –Modification of Use Directions and Changes to Basic CSF and Alternate CSF #1
Product Name: SUPER SOLID PLUS
EPA Registration Number: 86042-1
Application Date: June 20, 2018
Decision Number: 543342

Dear Mr. McLaughlin:

The amended label and CSF(s) referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide and Rodenticide Act, as amended, are acceptable. This approval does not affect any conditions that were previously imposed on this registration. You continue to be subject to existing conditions on your registration and any deadlines connected with them.

A stamped copy of your labeling is enclosed for your records. This labeling supersedes all previously accepted labeling. You must submit one copy of the final printed labeling before you release the product for shipment with the new labeling. In accordance with 40 CFR 152.130(c), you may distribute or sell this product under the previously approved labeling for 18 months from the date of this letter. After 18 months, you may only distribute or sell this product if it bears this new revised labeling or subsequently approved labeling. "To distribute or sell" is defined under FIFRA section 2(gg) and its implementing regulation at 40 CFR 152.3.

Please note that the record for this product currently contains the following CSF(s):

- Basic CSF dated 09/12/2018
- Alternate CSF 1 dated 09/12/2018

Should you wish to add/retain a reference to the company's website on your label, then please be aware that the website becomes labeling under the Federal Insecticide Fungicide and Rodenticide Act and is subject to review by the Agency. If the website is false or misleading, the product would be misbranded and unlawful to sell or distribute under FIFRA section 12(a)(1)(E). 40 CFR 156.10(a)(5) list examples of statements EPA may consider false or misleading. In addition, regardless of whether a website is referenced on your product's label, claims made on the website may not substantially differ from those claims approved through the registration process. Therefore, should the Agency find or if it is brought to our attention that a website contains false or misleading statements or claims substantially differing from the EPA approved registration, the website will be referred to the EPA's Office of Enforcement and Compliance.

Page 2 of 2
EPA Reg. No. 86042-1
Decision No. 543342

Your release for shipment of the product constitutes acceptance of these conditions. If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA section 6. If you have any questions, please contact Melanie Bolden by phone at (703) 347-0165, or via email at Bolden.Melanie@epa.gov.

Sincerely,

A handwritten signature in blue ink that reads "Wanda G. Fuller, for". The signature is written in a cursive style.

Demson Fuller, Product Manager 32
Regulatory Management Branch II
Antimicrobials Division (7510P)
Office of Pesticide Programs

Enclosure

NOTIFICATION

86042-1

PRECAUTIONARY STATEMENTS
HAZARDS TO HUMANS AND
DOMESTIC ANIMALS
DANGER

The Applicant has certified that no other than those reported to the Agency have been made to the labeling. The Agency acknowledges this notification by letter dated:

10/22/2018



THE EFFICACY OF THIS PRODUCT HAS BEEN DEMONSTRATED IN THE PRESENCE OF 20% BLOOD SERUM

ACTIVE INGREDIENT:
Sodium Dichloro-s-triazinetriene 10.2%
OTHER INGREDIENTS 89.8%
TOTAL 100.0%
(Available Chlorine 5.8%)

KEEP OUT OF REACH OF CHILDREN
DANGER

DANGER

Corrosive. Causes irreversible eye damage. Causes skin irritation. Harmful if swallowed, inhaled or absorbed through skin. Do not get in eyes, on skin or clothing. Wear goggles, face shield or safety glasses and rubber gloves when handling. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Remove and wash contaminated clothing before reuse.

ENVIRONMENTAL HAZARDS STATEMENT

This product is toxic to fish and aquatic organisms.

PHYSICAL HAZARDS

Avoid breathing dust from powder.

STORAGE AND DISPOSAL

Do not contaminate water, food or feed by storage or disposal.

STORAGE

Keep in a cool, dry place. Keep container tightly closed.

PESTICIDE DISPOSAL

Pesticide wastes are toxic. Improper disposal of excess pesticide is a violation of federal law. If these wastes cannot be disposed of by use according to label instructions, contact your state pesticide or environmental control agency or the hazardous waste representative of the nearest EPA Regional Office for guidance.

CONTAINER DISPOSAL

Nonrefillable container or Water Soluble Packet. Do not reuse or refill this container. Offer for recycling, if available or dispose of the empty outer plastic or foil pouch in the trash as long as the water soluble packet is unbroken.

REORDER NO.: _____
Liquid Medical Waste Encapsulation and Treatment
For use only in Healthcare and Veterinary Facilities

DIRECTIONS FOR USE

It is a violation of federal law to use this product in a manner inconsistent with its labeling.

ALWAYS WEAR HAND AND EYE PROTECTION WHEN HANDLING INFECTIOUS WASTE

This product treats (see table) CC's of fluid USE WITH SUCTION CANISTERS ONLY

-See attached Twist Top directions for use.

-See attached EZ Spout directions for use.

-See attached Yorker Spout directions for use.

-See attached Spout with Blue Tip directions for use.

-See attached AMD Spout directions for use.

-See attached Pouch directions for use.

Net Weight	Suction Canister Treated	Bottle Size
.7oz	500CC	4oz
1.4oz	1000CC	4oz
1.68oz	1200CC	4oz
2.1oz	1500CC	4oz
2.8oz	2000CC	4oz
4.2oz	3000CC	6oz

Manufactured By:
DiSorb Systems, Inc.
1800 W. Indiana Avenue
Philadelphia, PA 19132
www.disorb.com
1-215-207-9010
1-866-287-6672

EPA Reg. No. 86042-1
EPA Establishment No. 86042-PA-0001
SuperSolid Plus is a registered trademark of DiSorb Systems, Inc.

FIRST AID Have the product container or label with you when you call a poison control center, doctor or when going for treatment
IF IN EYES: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lens, if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice.
IF ON SKIN OR CLOTHING: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for further treatment advice.
IF SWALLOWED: Call a poison control center or doctor immediately for treatment advice. Have person sip a glass of water, if able to swallow. Do not induce vomiting unless told by a poison control center or doctor. Do not give anything by mouth to an unconscious person.
IF INHALED: Move person to fresh air. If person is not breathing, call 911 or ambulance, then give artificial respiration, preferably mouth to mouth. Call a poison control center or doctor for further treatment advice.
In case of accidental exposure, call CHEMTREC 1-800-424-9300
NOTE TO PHYSICIAN Probable mucosal damage may contraindicate the use of gastric lavage.

Cap and Packet/Pouch Configurations Directions for Use

Twist Top

- 1 Shake well to thoroughly mix the SuperSolid Plus Powder.
- 2 SuperSolid Plus Powder can be added before or after the procedure.
- 3 Open the accessory port on the top of the suction canister.
- 4 Empty bottle into suction canister.
- 5 Close the accessory port.
- 6 Allow sufficient time for solidification.
- 7 Dispose of the encapsulated material according to institutional procedures, State, Federal and Local regulations.

Spout with Blue Tip

- 1 Shake well to thoroughly mix the SuperSolid Plus Powder.
- 2 SuperSolid Plus Powder can be added before or after the procedure.
- 3 Open pour spout on suction canister.
- 4 Remove blue tip from bottle cap.
- 5 Pour SuperSolid Plus into canister and allow to gel.
- 6 Close the accessory port.
- 7 Dispose of the encapsulated material according to institutional procedures, State, Federal and Local regulations.

Yorker Spout

- 1 Shake well to thoroughly mix the SuperSolid Plus Powder.
- 2 SuperSolid Plus Powder can be added before or after the procedure.
- 3 Trim Plastic tip with scissors at a 45° angle (cut 1/2" off the top of the pour spout).
- 4 Insert the pour spout into the largest suction port.
- 5 Empty bottle into suction canister.
- 6 Allow sufficient time for solidification.
- 7 Dispose of the encapsulated material according to institutional procedures, State, Federal and Local regulations.

Yorker Spout Modified

- 1 Shake well to thoroughly mix the SuperSolid Plus Powder.
- 2 Trim Plastic tip with scissors at a 45° angle (cut 1/2" off the top of the pour spout).
- 3 Insert the pour spout into the largest suction port.
- 4 Empty bottle into suction canister.
- 5 Allow sufficient time for solidification.
- 6 Dispose of the encapsulated material according to institutional procedures, State, Federal and Local regulations.

EZSpout

- 1 Shake well to thoroughly mix the SuperSolid Plus Powder.
- 2 Turn off suction and remove patient tubing.
- 3 Position EZ Spout over patient or tandem port.
- 4 Turn on suction.
- 5 Push bottle down to break the liner.
- 6 After powder is drawn into the canister, turn off suction. Remove bottle and cap the open port.
- 7 Allow sufficient time for solidification.
- 8 Dispose of the encapsulated material according to institutional procedures, State, Federal and Local regulations.

EZ Spout Modified

- 1 Shake well to thoroughly mix the SuperSolid Plus Powder.
- 2 Turn off suction and remove patient tubing.
- 3 Position EZ Spout over patient port.
- 4 Push bottle down to break the liner.
- 5 After powder is drawn into the canister, remove bottle and cap the open port.
- 6 Allow sufficient time for solidification.
- 7 Dispose of the encapsulated material according to institutional procedures, State, Federal and Local regulations.

AMD Spout

- 1 Shake well to thoroughly mix the SuperSolid Plus Powder.
- 2 Turn off suction and remove patient tubing.
- 3 Peel off "Remove Before Using" Sticker.
- 4 Position Spout Cap over patient to ortho port on top of suction canister.
- 5 Turn on suction.
- 6 With suction on, place palm of hand on bottom of bottle and gently push down to break membrane.
- 7 Suction will draw the SuperSolid Plus into the suction canister. Be sure all SuperSolid Plus Powder is drawn into the suction canister.
- 8 Turn off suction.
- 9 Remove bottle, and cap open port.
- 10 Allow sufficient time for solidification.
- 11 Dispose of the encapsulated material according to institutional procedures, State, Federal and Local regulations.

Water Soluble Packet/Pouch

- 1 Prior to beginning the procedure drop the SuperSolid Plus Packet into the canister.
- 2 As fluid is introduced into the canister the SuperSolid Plus Powder is released.
- 3 Allow sufficient time for solidification.
- 4 Dispose of the encapsulated material according to institutional procedures, State, Federal and Local regulations.