

DIRECTIONS FOR USE

TO DESTROY PS. AERUGINOSA, USE 4 OUNCES OF WESTAFECT G PER QUART OF WATER


1. FOR ROUTINE USE: Swab, wipe or rinse surfaces (such as floors, walls, urinals, washstands, garbage pails) thoroughly, using one per cent solution (see "DILUTION TABLE"). Disinfect smaller objects by immersion in one per cent solution for at least 10 minutes. Permit all surfaces to dry before use.

2. FOR SPECIAL USE:

A. If known heavy contamination exists, use two per cent solution on surfaces. Swab, wipe or rinse surfaces with disinfectant solution at least twice, allowing solution to act for at least 10 minutes between applications. Disinfect smaller objects by immersion for at least 10 minutes. Permit all surfaces to dry before use.

B. Cold disinfection of surgical instruments sterilization by autoclave should be performed

3. PRECAUTION: To disinfect RUBBER and PLASTIC objects, swab surfaces if possible. Immerse only if required. LIMIT EXPOSURE TO 10 MINUTES. PRO-LONGED IMMERSION IN DISINFECTANT MAY AFFECT PLASTICS AND RUBBER SERIOUSLY.

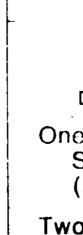

isinteriant, germicidal Ingicial concentrate

0201

																										Percent By Weight
				•	•	•	•		•		•	•	•	•		•	•						•			12.70%
	•	•		•		•									•	•				•	•	•				5.00%
														•					-				•			4.84%
			•			•		-		•				•	•	•				•		•				77.46%
																			٦	r (O	T	-/	41	_	100.00%

Specification: O.D.-406 • Stock Number: 6840-116-9364 • Type: L Manufacturer's Designation: WESTAFECT G

*Kills pathogenic fungi such as Trichophyton interdigitale

	DILUTION	TABLE	
Dilution	Ounce concentrate per gallon water	MI concentrate per liter water	Teaspoons concentrate per gallon water
One Per Cent Solution (Routine Us	1¼ se)	10	10
Two Per Cent	-	-	

Specification: O.D.-406 • Stock Number: 6840-116-9364 • Type: L Manufacturer's Designation: WESTAFECT G

*Kills pathogenic fungi such as Trichophyton interdigitale

ALWAYS DILUTE ACCORDING TO "DILUTION TABLE" BEFORE USING Do Not Add to Soap or Detergent Solution

DIRECTIONS FOR USE

TO DESTROY PS. AERUGINOSA, USE 4 OUNCES OF WESTAFECT G PER QUART OF WATER

1. FOR ROUTINE USE: Swab, wipe or rinse surfaces (such as floors, walls, urinals, washstands, garbage pails) thoroughly, using one per cent solution (see "DILUTION TABLE"). Disinfect smaller objects by immersion in one per cent solution for at least 10 minutes. Permit all surfaces to dry before use.

2. FOR SPECIAL USE:

コ

 \sim

S

F17

160

A. If known heavy contamination exists, use two per cent solution on surfaces. Swab, wipe or rinse surfaces with disinfectant solution at least twice, allowing solution to act for at least 10 minutes between applications. Disinfect smaller objects by immersion for at least 10 minutes. Permit all surfaces to dry before use.

B. Cold disinfection of surgical instruments – sterilization by autoclave should be performed wherever possible. Cold disinfection of surgical instruments with this material may be used as an emergency alternate. In the absence of heat, immerse in 2% solution for at least 10 minutes after removing excess soil by cleaning.

EPA Reg. No. 52-160

CAUTION: Keep Out Of Reach Of Children.

Do not take internally. May be harmful if swallowed. Avoid contact with eyes. In case of eye contact, flush thoroughly with water. Rinse empty container thoroughly with water and discard it.



Manufacturer: WEST CHEMICAL PRODUCTS, INC. 42-16 WEST ST., LONG ISLAND CITY, NEW YORK NET CONTENTS: 5 GALLONS (18.92 litres

3. PRECAUTION: To disinfect RUBBER and PLASTIC objects, swab surfaces if possible. Immerse only if required. LIMIT EXPOSURE TO 10 MINUTES. PRO-LONGED IMMERSION IN DISINFECTANT MAY AFFECT PLASTICS AND RUBBER SERIOUSLY.

	DILUTION	TABLE	
Dilution	Ounce concentrate per gallon water	MI concentrate per liter water	Teaspoons concentrate per gallon water
One Per Cent Solution (Routine Use	1 ¼ e)	10	10
Two Per Cent Solution (Special Use	2 ¹ / ₂	20	20
To Destroy Ps. aeruginosa	16	125	128

(11101	DATE OF PACK
	CONTRACT NUMBER
es)	LOT NUMBER