PENNGAS STERILIZING GAS

REG. U.S. PAT. OF

ACCEPTED

APR 1 1970

NON-FLAMMABLE ETHYLENE OXIDE FREON MIXTURE

WARNING

The liquefied mixture in this container has toxic properties. Use care in handling. Vapor harmful. Prevent spraying on hands, face, eyes, or other skin areas. May injure eyes or irritate skin. If contents come in contact with any of these parts of the body flush them promptly with water. For eyes, get medical attention. If clothing becomes sprayed, wash clothing before reuse. Avoid breathing vapors. If inhaled sufficiently to cause headache or nausea, breathe fresh air, lie down, and use no stimulants.

INGREDIENTS-BY WEIGHT

ACTIVE INGREDIENTS

ETHYLENE OXIDE 12.0%

INERT INGREDIENTS

DICHLORODIFLUOROMETHANE 88.0%

REG. U. S. PAT. OFF. No. 2,956,708

CONTENTS: 140 POUNDS NET WEIGHT

U.S.D.A. REG. NO. 87-9

Manufactured in U.S.A. by
PENNSYLVANIA ENGINEERING COMPANY
Philadelphia, Pa. 19123

CAUTION USE ONLY WITH VALVE IN UPWARD POSITION

During and after use areas should be weil ventilated.

Do not store in temperatures above 140°F.

FOR USE

Remove seal. Unscrew plug, turning clockwise (this is a left hand thread). Insert sterilizer outlet in opening and tighten thread, turning counterclockwise.

When cylinder is not connected to sterilizer and when cylinder is empty, close valve tightly and insert plug.