

PM 31

REG# 87-9

**PENNGAS STERILIZING GAS**  
REG. U. S. PAT. OFF.  
**NON-FLAMMABLE**  
**ETHYLENE OXIDE FREON MIXTURE**

ACCEPTED

JAN 08 1987

Under the Federal Insecticide,  
 Fungicide and Rodenticide Act,  
 registered under the pesticide  
 label under EPA Reg. No. 87-9

The liquified mixture in this container has toxic properties. Use care in handling. Vapor is harmful. Do not breathe gas. Do not get in eyes, on skin, or clothing. Store and use with adequate ventilation.

**FIRST AID: IF INHALED,** remove to fresh air. If not breathing, give artificial respiration. If breathing is difficult, give oxygen. Call a physician.

**IN CASE OF CONTACT,** immediately flush eyes or skin with plenty of water for at least 15 minutes while removing contaminated clothing and shoes. Call a physician. Discard contaminated clothing and shoes.

**LEAK:** Evacuate area and keep personnel upwind. Use self-contained breathing apparatus and protective clothing and shut off lead if without risk.

**FIRE:** Use water spray or fog nozzle to keep cylinder cool. Move cylinder away from fire if without risk.

**IN EMERGENCY:**

CALL CHEMTREC 800-424-9300

**INGREDIENTS—BY WEIGHT**

<b>ACTIVE INGREDIENTS</b>	
ETHYLENE OXIDE	12.0%
<b>INERT INGREDIENTS</b>	
DICHLORODIFLUOROMETHANE	88.0%

REG. U. S. PAT. OFF. No. 2, 56,708

Keep Out of Reach of Children

**DANGER**

**PRESSURIZED NONFLAMMABLE LIQUID AND GAS. HARMFUL IF INHALED. CAUSES EYE AND SKIN BURNS. SUSPECT CANCER HAZARD. ODOR: ETHER-LIKE.**

**CONTENTS:**

**140 POUNDS NET WEIGHT**

E.P.A. REG. NO. 87-9

E.P.A. EST. NO. 87-PA-1

Manufactured in U.S.A. by

**PENNSYLVANIA ENGINEERING COMPANY**

1107 21 N. Howard St.

Philadelphia, Pa. 19123

Health Care Facilities and Industrial Use Only

**USE ONLY WITH VALVE  
 IN UPWARD POSITION**

During and after use areas should be well ventilated.

Do not store in temperatures above 140°F.

**DIRECTIONS FOR USE**

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Remove seal. Unscrew plug, turning clockwise (this is a left hand thread). This cylinder can only be connected with a male 510 left-hand threaded outlet. Insert sterilizer outlet in opening and tighten thread, turning counterclockwise.

When cylinder is not connected to sterilizer and when cylinder is empty, close valve tightly and insert plug, and return to supplier when empty.

**STORAGE AND DISPOSAL**

**STORAGE:** Do not store over 140°F. (Longer shelf life if stored under 100°F.)

**DISPOSAL:** Do not contaminate water, food or feed by storage or disposal.

Return empty cylinders for reuse—Waste should not be disposed of, but returned in cylinders with valves closed and plugs inserted.

Pesticide wastes are toxic—Improper disposal of excess pesticide, spray mixture, or rinsate is a violation of Federal Law—If these wastes cannot be disposed of by use according to label instructions above, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste representative at the nearest EPA Regional Office for guidance.