



DISINFECTING AND STERILIZING SOLUTION

SPORICIDAL ■ PSEUDOMONACIDAL ■ FUNGICIDAL
BACTERICIDAL ■ VIRUCIDAL ■ TUBERCULOCIDAL

Especially recommended for the disinfection or sterilization of . . .

- *Dental Instruments* ■ *Mirrors* ■ *Plastic Objects*
- *Thermometers* ■ *Contaminated Instruments* ■ *Rubber Objects*
- *Sharp Instruments* ■ *Aluminum Objects* ■ *Burs*

Active Ingredient—Glutaraldehyde 2.0% Inert Ingredients 98.0%

NOTE — Attached container contents MUST be added to solution before this product is effective. SEE DIRECTIONS.

Caution
Keep out of reach of children

Johnson & Johnson

Dental Division
New Brunswick, N. J. 08903

Code 2751
PAT. 3016328

*Trademark

CONTENT ONE QUART
(32 FLUID OZ.)

U.S.D.A. Reg.
No. 90-3

MADE IN U.S.A.

DIRECTIONS FOR ACTIVATING

Add only contents of attached container to this container of CIDEX fluid. Shake well. Solution is ready to use. When activated, solution changes color to green. Do not use this solution beyond 14 days after mixing. Note dates on container where indicated.

DIRECTIONS FOR USE

For complete disinfection or sterilization of dental instruments and appliances, place thoroughly cleansed and rough-dried instruments or objects in FULL STRENGTH CIDEX SOLUTION. Flush and fill cleansed lumen of any hollow instruments. Use covered containers to minimize odor and to prevent evaporation.

DISINFECTION: Immerse completely for a minimum of 10 MINUTES to destroy viruses and vegetative pathogens including *M. tuberculosis* and *Pseudomonas aeruginosa*.

STERILIZATION: Immerse completely for a minimum of 10 HOURS to destroy resistant pathogenic spores including *Cl. sporogenes* and *Cl. tetani*.

Remove instruments from CIDEX solution using sterile technique and RINSE THOROUGHLY with sterile water if available. If not, use 70% isopropyl alcohol or hot tap water for rinse.

PRECAUTIONS

EYES — AVOID EYE CONTACT. Contact with solution causes eye irritation. In case of contact, flush with water immediately and get medical attention.

SKIN — AVOID SKIN CONTACT, as a possibility of sensitization exists. In case of contact, wash thoroughly with soap and water.

FOOD — Avoid contamination of food.

INSTRUMENTS — Do not immerse carbon steel objects over 24 hours. Solution contains rust inhibitor, but will not prevent or remove rust caused by poor plating or improper cleaning of instruments. Not recommended for use on injection needles.

STORE IN COOL PLACE

MIXING DATE

EXPIRATION DATE



2751 BL

ACCEPTED WITH COMMENTS