

The image is a dark, high-contrast scan, likely of a document page. It is predominantly black with significant noise and artifacts. A prominent white outline of a shape, possibly a letter or a small object, is visible in the lower-left quadrant. There are also several small white specks and a larger white circular artifact on the left edge. The overall appearance is grainy and noisy, typical of a low-quality scan.

ACCEPTED

OCT 23 1971

UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND RODENTICIDE ACT
FOR ECONOMIC POISON REGISTERED
UNDER NO. 201-24 SUBJECT
TO ATTACHED COMMENTS.

UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND RODENTICIDE ACT
FOR ECONOMIC POISON REGISTERED
UNDER NO. 211-28 SUBJECT
TO ATTACHED COMMENTS.

11. The following will be added to the report statement: "Avoid the contamination of feed and feedstuffs. Do not apply toxic materials on feed or feedstuffs. Do not use feedstuffs containing toxic substances."

FOR THE FBI PALM BEACH
AND BENTONVILLE ACT
FOR THE PERSON REGISTER.
THEY ARE NO SUBJECT
TO ALL OTHER COMMENTS.

NET CONTENTS

GALLONS

Pheno-Cen®

HOSPITAL GERMICIDAL DETERGENT

TUBERCULOCIDAL — PSEUDOMONACIDAL — GERMICIDAL — FUNGICIDAL

**A highly effective multi-purpose
activated soapless, phenolic, hospital
germicide, detergent and deodorant.**

Use-Dilution Confirmation (A.O.A.C. 20°C)

Salmonella choleraesuis	1:256
Staphylococcus aureus	1:256
Pseudomonas aeruginosa	1:256
Mycobacterium tuberculosis	1:256

Active Ingredients

Isopropyl alcohol	10.00 %
Potassium ortho-benzyl para-chlorophenate....	7.20 %
Potassium ortho-phenylphenate	6.28 %
Potassium para-tertiary-amylphenate	4.30 %
Tetrasodium ethylenediamine tetraacetate....	3.04 %
Potassium 4 chloro 2 cyclo-pentylphenate	.47 %
Potassium 4 chloro 2 phenylphenate.....	.31 %

Inert Ingredients 68.40 %

Contains Sodium Nitrite

FIRST AID: In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse. If swallowed, do not induce vomiting, drink large quantities of fluid and call a physician immediately.



CENTRAL CHEMICAL COMPANY, INC.
3130 Brinkerhoff Road, Kansas City, Kansas 66115

USDA Reg. No. 211-25

DANGER: KEEP OUT OF REACH OF CHILDREN**DIRECTIONS FOR USE**

Always dilute PHENO-CEN with water as indicated below.

For all general disinfecting, deodorizing and cleaning of floors, furniture, equipment, fixtures and other inanimate objects, thoroughly wet surface with 1 part **PHENO-CEN** to 256 parts water (½ oz. to the gallon of water). If higher detergency is desired use 1 to 2 oz. per gallon of water.

To Disinfect Surgical Instruments: Remove adhering blood and serous exudate. Soak for 15 minutes in a solution of **PHENO-CEN** diluted ½ oz. per gallon of water. If instruments are soiled, increase dilution to 1 to 2 oz. per gallon of water.

To Disinfect and Deodorize Plastic and Rubber Goods: Soak for exactly 15 minutes in a solution of **PHENO-CEN** diluted ½ oz. per gallon of water.

PHENO-CEN has been tested and approved for use on conductive floors.

DILUTION TABLE(Quantity of **PHENO-CEN** for 1:256 dilutions)

Quart	1 Gallon	3 Gallons	5 Gallons
4 cc	15 cc	45 cc	75 cc
(1/8 oz.)	(1/2 oz.)	(1 1/2 oz.)	(2 1/2 oz.)

DANGER: May cause eye damage. May cause skin irritation. Do not get in eyes, on skin, on clothing. Wear goggles or face shield and rubber gloves when handling. Harmful or fatal if swallowed. Avoid contamination of food. Rinse empty container thoroughly with water and discard after use.

KEEP FROM FREEZING

ACCEPTED

OCT 23 1971

FOR THE NATIONAL BUREAU OF
LABORATORY STANDARDS
U.S. DEPARTMENT OF COMMERCE
SUBJECT
TO THE FOLLOWING COMMENTS: