. 275 - 139

11/14/2002

Page 181 ENVIRONMENTAL HAZARDS

AEROSET WATER BATH ADDITIVE

TO PREVENT AND CONTROL MICROBIAL CONTAMINATION IN THE AEROSET SYSTEM WATER BATH

 EPA Reg. No. 275-139
 EPA Est. No. 75196-JPN-001

 Active Ingredients:
 0.211%

 5-chloro-2-methyl-4-isothiazolin-3-one
 0.075%

 Inert Ingredients:
 99,714%

KEEP OUT OF REACH OF CHILDREN CAUTION

FIRST AID	
If in eyes:	 Hold eye open and rinse slowly and gently with water for 15 to 20 minutes. Remove contact lenses, if present, after the first 5 minutes then continue rinsing the eye. Call a poison control center or doctor for treatment advice
If on skin	Take off contaminated clothing.
or on	Rinse skin immediately with plenty of water
clothing:	for 15-20 minutes.
	Call a poison control center or doctor for
	treatment advice
If swallowed:	person control of decision
	immediately for treatment advice.
	Have person sip a glass of water if able to swallow.
	Do not induce vomiting unless told to do so
	by the poison control center or doctor.
	Do not give anything by mouth to an
	unconscious person.
Have the product container or label with you when calling a	
poison control center or doctor, or going for treatment.	
Note to	Probable mucosal damage may contraindicate
Physician:	the use of gastric lavage.

PRECAUTIONARY STATEMENTS

HAZARDS TO HUMANS AND DOMESTIC ANIMALS

CAUTION. Causes substantial but temporary eye injury and skin irritation. Harmful if swallowed or inhaled. Do not get in eyes, on skin or on clothing. Wear protective eyewear and chemically-resistant gloves when adding this product to the water bath additive container used with Aeroset System. Wash thoroughly with soap and water after handling. Prolonged or frequently repeated skin contact may cause allergic reactions in some individuals.

USER SAFETY RECOMMENDATIONS

Users should:

- Wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet.
- Remove clothing immediately if pesticide gets inside. Then wash thoroughly and put on clean clothing.
- Remove PPE immediately after handling this product. Wash the outside of gloves before removing. As soon as possible, wash thoroughly.

This chemical is toxic to terrestrial and aquatic plants, fish and aquatic invertebrates.

STORAGE AND DISPOSAL

Do not contaminate water, food or feed by storage and disposal.

Store in a cool, dry area at 15-30°C. Securely wrap empty container in several layers of newspaper and discard in trash.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

Aeroset Water Bath Additive is only for use with the Aeroset System Blood Analyzer. The product contains and antimicrobial agent which prevents and/or controls microbial contamination in the water bath of the Aeroset System Blood Analyzer. By preventing or controlling microbial contamination, the performance of the Aeroset System Blood Analyzer is optimized. Do not use this product to disinfect the water bath or any surfaces on the Aeroset System Blood Analyzer.

To Use: Add Aeroset Water Bath Additive to the wedged shaped bottle marked Water Bath Additive. Then place the filled water bath additive bottle into its proper carousel location in the Aeroset System Blood Analyzer. The antimicrobial solution is automatically dispensed into the water bath during the Change Water Bath option performed in the Start UP or SHUTDOWN Procedures.

Net Contents 1 liter (1.1 qt.)

Manufactured for: ABBOTT Laboratories 200 S. Abbott Park Road Abbott Park, IL 60064-6157

ACCEPTED

NOV 1 4 2002

Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide, registered under EPA Reg. No. 275 - 139