

SOLU-STYRIL

SOLU-STYRIL will not deteriorate with age or changes of temperatures and will not rust or corrode surgical steel or metals. (Not recommended for Aluminum.) Its low surface tension permits intimate contact with the crevices and surfaces and will not dull the keenest cutting edge or sharp points. Not recommended for complete disinfection of instruments, contaminated with spore bearing pathogens or rusty instruments, etc., etc.

Avoid getting into the eyes. In case of contact wash thoroughly with plenty of water. If irritation persists obtain medical attention at once. Harmful if swallowed. Avoid contamination of food. Not to be taken internally. For technical use at hospitals, medical and dental professions.

REG. NO. 283-3



SOLU-STYRIL

GERMICIDAL SOLUTION
Brand of Benzalkonium Chloride
ALCOHOLIC READY FOR USE
NON STAINING NON CORROSIVE
RUST PREVENTING

An efficient chemical germicide for the cold disinfection of surgical and dental instruments. High germicidal and Bacteriostatic potency.

Active Ingredients:

ALKYL* (C ₁₄ - 23%, C ₁₂ - 61%, C ₁₀ - 11%, C ₈ - 5%)	0.13%
dimethylbenzyl ammonium chlorides	60.00%
Isopropyl Alcohol	39.87%
Inert Ingredients	100.00%

Keep out of the reach of children.
New Strength — 1 Gallon
Manufactured by
STERI KEM PRODUCTS
WHITTIER, CALIFORNIA

DIRECTIONS

SOLU-STYRIL Alcoholic germicidal solution for the cold germicidal treatment and care of Metallic, Plastic, Rubber, or glass instruments and apparatus that will not withstand sterilization by heat. Instruments should be thoroughly cleansed of contaminating organic and spore bearing materials, with soap or a mild alkaline detergent and well rinsed, before immersing in solution. Hollow instruments should be slightly tilted to displace trapped air pockets and to avoid chemical reactions with injectorial solutions, and before use should be thoroughly flushed with sterile water.

Instruments should remain immersed in the solution for at least 30 minutes or longer. Do not pour solution that has once been used back into stock bottle. Does not contain Phenol, Mercuric, Iodine or Formaldehyde.

USE FULL STRENGTH