

Front Panel

JULY 10 1968

GERMICIDAL SOLUTION


Brand of benzalkonium chloride
Aqueous Solution 1:750

FOR DISINFECTION OF DENTAL AND SURGICAL INSTRUMENTS
READY FOR USE NON-STAINING
NON-RUSTING

ACTIVE INGREDIENTS:

ALKYL* C₁₁-23% C₁₂-61%, C₁₆-11%, C₁₈- 5%
Dimethyl-benzyl ammonium chlorides 0.133 %
Inert Ingredients (Distilled H₂O) 92.867 %
100.00 %

An efficient chemical germicide for the disinfection of
Dental and Surgical instruments.

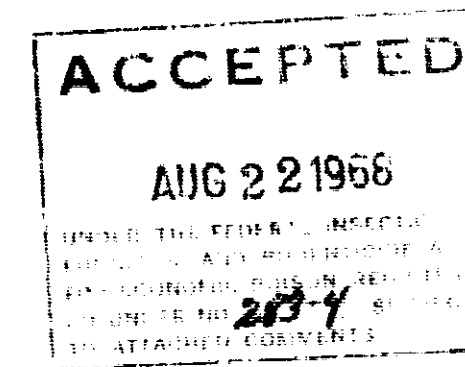

CAUTION: KEEP OUT OF REACH OF CHILDREN

New Strength 1:750 -Replaces 1:1000 1 Gallon

Manufactured for
COAST SURGICAL SUPPLY COMPANY
Inglewood, California

RIGHT SIDE PANEL

~~This solution is non-toxic to the skin.~~ DELETED
It will not rust or corrode surgical steel steel or other metals.
(Aluminum excepted)

May be used for disinfection of metal, plastic, rubber or glass
instruments and apparatus including catheters, gloves, hypodermic
equipment and tubing.

LEFT SIDE PANEL

DIRECTIONS

The solution should be used full strength.

Clean instruments well and rinse. Submerge article in solution for
at least 30 minutes.

Instruments may be immediately used without further rinsing or drying.

For destruction of all spore-bearing bacteria, auto claving is recommended,
after which instruments or needles may be stored in this solution until used.

USE FULL STRENGTH

Gentlemen; This will be our label for the Coast Surgical Supply Co, but of course will be
silk screened thus until your approval we cannot supply printed labels, seperately.

FRONT LABEL

JULY 10 1968

Neo SOLU-STYRIL No. 5
GERMICIDAL SOLUTION
Frano of Benzalkonium Chloride
AQUEOUS READY FOR USE

NON STAINING RUST PREVENTING

An efficient chemical germicide for the cold disinfection of surgical and dental instruments. High germicidal and Bacteriostatic potency.

Active Ingredients:


ALKYL * (C₁₄ -23%, C₁₂ -61%, C₁₆-11%, C₈-5%
dimethylbenzyl ammonium chlorides 0.133%
Inert Ingredients (Distilled H₂O) 99.867%

CAUTION: KEEP OUT OF REACH OF CHILDREN.

New Strength--- 1:750

1 Gallon

STERIL- KEM PRODUCTS
WHITTIER, CALIFORNIA

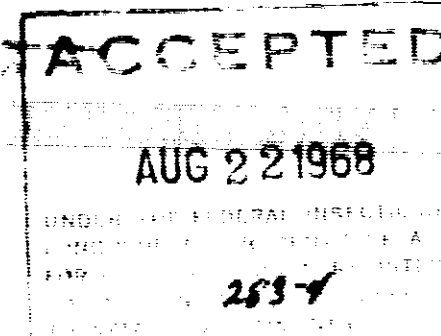

LEFT SIDE PANEL

NEO SOLU-STYRIL No.5

Neo Solu-Styiril No.5 will not deteriorate with age or changes of temperatures and will not rust or corrode surgical steel or metals. (Not recommended for Aluminum) Its low surface tension permits intimate contact with the crevices and surfaces and will not dull the keenest cutting edge or sharp points. Not recommended for complete disinfection of instruments, contaminated with spore bearing pathogens or rusty instruments, Etc. Etc.

CAUTION : Avoid getting into the eyes; In case of contact wash thoroughly with plenty of water. If irritation persists obtain medical attention at once. Harmful if swallowed. Avoid contamination of food. Not to be taken internally. For technical use of hospitals, medical and dental professions.

RIGHT SIDE PANEL

DIRECTIONS

Neo Solu- Styiril No.5 aqueous germicidal solution for the cold germicidal treatment and care of Metallic, Plastic, Rubber, or glass instruments and apparatus that will not withstand sterilization by heat. Instruments should be thoroughly cleaned of contaminating organic and spore bearing materials, with soap or a mild alkaline detergent and well rinsed, before immersing in solution. Hollow instruments should be slightly tilted to displace trapped air pockets and to avoid chemical reactions with injectional solutions, and before use should be thoroughly flushed with sterile water.

Instruments should remain immersed in the solution for at least 30 minutes or longer. Do not pour solution that has once been used back into stock container.

USE FULL STRENGTH

Gentlemen, This will be our label, these will be silk screened, thus until your approval, we cannot supply the labels separately.