This powerful hospital disinfectant and deodorant acts quickly on germ laden surfaces such as: Refuse Cans. Toilet Seats, Telephones, Wash Basins, Urinals, Laundry Chutes, Door Knobs. Metal Beds and Springs. Wheel Chairs, Linen Carts, Hampers. Dressing Carts, Clean-Up Carts, Air Ducts, Light Switches, Office Machinery, and other equipment, after the surface has been properly cleaned. Will reduce the hazard of cross infection by bacteria.

Non-selective odor suppressant for space odor conditioning and deodorizing surgical dressings. Molecular aerosol deodorant. Attacks malodor at the source. Does not cause anosmia. Suppresses and eliminates odors by chemically neutralizing odor-producing materials.

Effective on odors from any source in hospitals, nursing homes, ambulances, from post-colostomy, postileostomy dressing and bags, malignant diseases, homes, kitchens, bath rooms, industrial plant processes, food processing and packaging plants and food stores.

Acts instantly and remains effective for long periods on unpleasant odors from fecal matter, urine, burns, smoke, mildew and mold, decay, garlic. onions, food putrefaction, perspiration, stale air, tobacco, sick rooms, musty closets, fish.

WARNING- CAUSES EYE IRRI-TATION. Do not get in eyes. In case of contact with eyes, flush at once with wa ter for 15 minutes. Call a physician. Avoid skin contact. Avoid breathing of spray mist. Use adequate ventilation. Avoid contact with food.

**CONTENTS UNDER PRESSURE:** Do not puncture or incinerate container. Do not expose to heat or store at temperatures above 120 . F. DO NOT USE NEAR HEAT OR FLAME.

### TUBERCULOCIDAL-STAPHYLOCIDAL BACTERICIDAL FUNGICIDAL-PSEUDOMONICIDAL CONTROLS MILDEW (MOLD OR FUNGI)

Disinfects Influenza A<sub>2</sub>/Hong Kong Virus on inanimate environmental surfaces.

# **DISINFECTANT #8**





## This Product Will Control the Following Organisms:

ASPERGILLUS NIGER **ESCHERICHIA COLI MYCOBACTERIUM SMEGMATIS** PENICILLIUM GLAUCUM PSEUDOMONAS AERUGINOSA SALMONELLA CHOLERAESUIS SALMONELLA SCHOTTMUELLERI (PARATYPHOID B) SALMONELLA TYPHOSA SHIGELLA PARADYSENTERIAE STAPHYLOCOCCUS AUREUS STREPTOCOCCUS PYOGENES TRICHOPHYTON INTERDIGITALE **TUBERCLE BACILLI** 

Controls mildew (mold or fungi). Completely inhibits growth on cloth.

# **DIRECTIONS:**

#### SHAKE WELL BEFORE USING.

#### FOR DISINFECTING:

Hold dispenser upright about 6 to 8 inches from surface to be disinfected and spray 3 seconds until surface is wet.

#### FOR DEODORIZING:

Hold dispenser upright—point the opening away from face. Spray into air towards the center of room for about 4 seconds.

#### FOR MILDEW CONTROL:

Clean surface. Spray surface until wet. Repeat application every week to maintain control.

#### **IMPORTANT:**

Do not use on polished wood, furniture or rayon fabrics.

## EPA Reg. No. 334-313 AB

Ortho-phenylphenol 0.176°, Para-tertiary-amylphenol 0.044% 100.000%