

**DIRECTIONS**  
 Dilute 2 Ounces Macphene  
 with one gallon water.

Use: MACPHENE may be used on floors, furniture, porcelain, rubber, thermometers, lensed instruments, metals, walls, equipment, paint, plastic, linens, tubing, basins, ~~and for general sanitation.~~

Application: MACPHENE may be applied with mop, sponge or cloth. Also, spray or soak. Rinsing is not necessary on floors, except if surface is to be waxed or polished.

For Heavy Cleaning: Increase amount of MACPHENE according to amount of soil.



*Classified as to Fire Hazard only.*

USDA REG. NO. 421-~~11~~

ACCEPTED  
 Jan 13, 1969  
 FOR THE FEDERAL INSECTICIDE  
 AND RODENTICIDE ACT  
 PUBLIC HEALTH POISON REGISTERED  
 UNDER NO. .... SUBJECT  
 TO ATTACHED COMMENTS.

# Macphene

## GERMICIDAL DETERGENT

421-392

Staphylocidal, tuberculocidal, pseudomonacidal, fungicidal and virucidal against most viruses.

**PHENOL COEFFICIENT, A.O.A.C. METHOD**

Staphylococcus aureus 9.5      Salmonella typhosa 7.5

**A.O.A.C. USE DILUTION CONFIRMATION TESTS:**

Staphylococcus aureus 1-190      Pseudomonas aeruginosa 1-100  
 Salmonella choleraesuis 1-150      Trichophyton interdigitale 1-128

**A.O.A.C. CONFIRMATORY TUBERCULOCIDAL TEST:**

M. tuberculosis 1-100

**ACTIVE INGREDIENTS: 39%**

(1) Isopropyl Alcohol, (2) Vegetable Oil Soap, (3) o-Benzyl p-chloro-phenol, (4) Methyl Salicylate, (5) Essential Oils.

**INERT INGREDIENT, Water 61%**

**CAUTION: KEEP OUT OF REACH OF CHILDREN**

(See side panel for further caution)

Sold Exclusively By



3508 CLASSEN BLVD. • OKLAHOMA CITY, OKLA.  
 ST. LOUIS, MO.

MAC  
 deter  
 recor  
 purp  
 omm  
 home  
 Requ

If sw  
 dotes  
 follow  
 must  
 physi  
 Avoid  
 as we  
 In cas  
 tities

NET

ACCEPTED  
Jan 13, 1969  
UNDER THE FEDERAL INSECTICIDE  
AND RODENTICIDE ACT  
FOR ECONOMIC POISON REGISTERED  
UNDER NO. .... SUBJECT  
TO ATTACHED COMMENTS.

# Macphene

## GERMICIDAL DETERGENT

421-392

Staphylocidal, tuberculocidal, pseudomonacidal, fungicidal and virucidal against most viruses.

### PHENOL COEFFICIENT, A.O.A.C. METHOD

Staphylococcus aureus 9.5      Salmonella typhosa 7.5

### A.O.A.C. USE DILUTION CONFIRMATION TESTS:

Staphylococcus aureus 1-190      Pseudomonas aeruginosa 1-100  
Salmonella choleraesuis 1-150      Trichophyton interdigitale 1-128

### A.O.A.C. CONFIRMATORY TUBERCULOCIDAL TEST:

M. tuberculosis 1-100

### ACTIVE INGREDIENTS: 39%

(1) Isopropyl Alcohol, (2) Vegetable Oil Soap, (3) o-Benzyl p-chlorophenol, (4) Methyl Salicylate, (5) Essential Oils.

### INERT INGREDIENT, Water 61%

### CAUTION: KEEP OUT OF REACH OF CHILDREN

(See side panel for further caution)

Sold Exclusively By



3508 CLASSEN BLVD. • OKLAHOMA CITY, OKLA.  
ST. LOUIS, MO.

MACPHENE is a one step disinfectant, detergent and deodorizing agent. We recommend this product for all cleaning purposes except as a skin cleaner. Recommended for use in hospitals, nursing homes, schools and all public institutions. Requires no rinsing.

### CAUTION

#### Poisonous if taken internally

If swallowed, give the following antidotes: Milk, raw egg, mucilage, and follow with an emetic (tablespoonful of mustard in glass of warm water). Call a physician.

Avoid inhalation of fumes of material as well as contact with the skin and eyes. In case of contact, wash with large quantities of water.

KEEP AWAY FROM CHILDREN  
AND FOODSTUFFS

NET CONTENTS: