

DIRECTIONS

Cold Disinfection of Surgical Instruments

Clean thoroughly, rinse, and immerse in a solution of one ounce of UNI-CHLOR per gallon of water for ten minutes. One tablespoonful of sodium nitrite should be added to each gallon of solution as a corrosion inhibitor.

Disinfection of Rubber Goods

Clean thoroughly, then soak in solution diluted one ounce of UNI-CHLOR per two gallons of water for ten minutes.

Disinfection and Sanitization of Fabrics

After washing, agitate in solution containing one ounce of UNI-CHLOR per gallon of water to disinfect clothing and linens. To render diapers and other linens bacteriostatic and to retard development of ammonia, immerse in solution containing one ounce of UNI-CHLOR for each ten pounds of cloth.

UNI-CHLOR

HOSPITAL DISINFECTANT AND SANITIZER

Phenol Coefficients: *Salmonella typhosa* - 21
Staphylococcus aureus - 36

Non-inflammable, non-volatile, odorless.

CAUTION: KEEP OUT OF REACH OF CHILDREN

Do NOT use with SOAP or SYNTHETIC DETERGENTS.

Always thoroughly wash and rinse any article to be disinfected. AVOID CONTAMINATION OF FOOD.

Harmful if swallowed. Avoid contact with skin and eyes. In case of contact flush with plenty of water. If irritation persists, get medical attention.

Active Ingredient

Paradiisobutylphenoxyethoxyethyl dimethylbenzyl ammonium chloride

13%

Inert Ingredient Water

87%

Contents 1 gallon

Manufactured by

UNITED CHEMICAL COMPANY, INC.

FOUNDED

1913

Philadelphia, Delaware



Kansas City, Mo. 64105

USDA Reg. No. 623-30

DIRECT

Sanitization of Glassware, Dishes, Cooking Utensils

Clean with a suitable detergent, then dip in a solution of one ounce of UNI-CHLOR per two gallons of water, and dry on a clean towel. (Dilution should be as per health department regulations.)

General Disinfection of Hospitals, Public Rooms

After cleaning, apply one ounce of UNI-CHLOR per two gallons of water to usual sanitizing applications. For disinfection of contagious areas and for general cleaning, use 2/3 ounce per gallon.

ACCEPTED

AUG. 16, 1968

UNDER THE FEDERAL INSECTICIDE, FUNGICIDE AND RODENTICIDE ACT AND ECONOMIC POISON REGISTRATION UNDER NO. 623-30 SUBJECT TO ATTACHED COMMENTS.

UNI-CHLOR

HOSPITAL DISINFECTANT AND SANITIZER

Phenol Coefficients: Salmonella typhosa - 21
Staphylococcus aureus - 36

Non-inflammable, non-volatile, odorless.

CAUTION: KEEP OUT OF REACH OF CHILDREN
Do NOT use with SOAP or SYNTHETIC DETERGENTS.
Always thoroughly wash and rinse any article to be disinfected.
AVOID CONTAMINATION OF FOOD.

Harmful if swallowed. Avoid contact with skin and eyes. In case of contact flush with plenty of water. If irritation persists, get medical attention.

Active Ingredient

Paradiisobutylphenoxyethoxyethyl dimethylbenzyl ammonium chloride

13%

Inert Ingredient Water

87%

Contents 1 gallon

Manufactured by

UNITED CHEMICAL COMPANY, INC.

FOUNDED

1913



11th & Delaware

Kansas City, Mo. 64105

USDA Reg. No. 623-30

DIRECTIONS

Sanitization of Glassware, Silverware, Dishes, Cooking Utensils, etc.

Clean with a suitable detergent, rinse thoroughly, then dip in a sanitizing rinse made with one ounce of UNI-CHLOR in four (or five) gallons of water, and air dry. Do not use a towel. (Dilution should conform with local health department regulations.)

General Disinfection and Sanitation in Hospitals, Public Rooms, and At Home

After cleaning, apply solution containing 1/4 ounce of UNI-CHLOR per gallon of water for usual sanitizing applications. For hospital use, contagious areas and surfaces difficult to clean, use 2/3 ounce per gallon of water.