

HOW TO USE

INSTRUMENT GERMICIDE

DIRECTIONS FOR DISINFECTION OF

SURGICAL AND DENTAL INSTRUMENTS After mechanical cleaning with particular attention to removal of any contamination matter from crevices, joints and narrow spaces, immerse in the I&F Instrument Germicide for 15 minutes or until ready to use. To be safe from the instrument, wash with sterile distilled water.

HYPODERMIC NEEDLES AND SYRINGES Flush with distilled water. Immerse in I&F Instrument Germicide for 15 minutes. Then flush with sterile distilled water.

RUBBER GLOVES, TUBES AND OTHER RUBBER ARTICLES Immerse for not over 15 minutes. Remove then immerse in alcohol for one minute.

EYE AND DELICATE LENS INSTRUMENTS Instruments with optical systems, nose, off-ocular and endoscopes and place in I&F Instrument Germicide for 15 minutes but not longer. Assure thorough contact by turning instruments and rinse with sterile water. When not used, immediately place in sterile wrap for storage.

If rapid drying is required, immerse in 70% alcohol after removal from the I&F Instrument Germicide.

Rinse empty container thoroughly with water and dry.



LEHN & FINK

INSTRUMENT GERMICIDE

FOR RAPID DISINFECTION OF SURGICAL AND DENTAL INSTRUMENTS

GERMICIDAL - Kills bacteria, fungi, and viruses.

FUNGICIDAL - Kills fungi.

VIRUCIDAL - Kills viruses. Approved by U.S. Food and Drug Administration for use on surgical instruments.

ACTIVE INGREDIENTS: 1. 1,2-Dichloroethane, 2. Hydrogen Peroxide, 3. Sodium Hypochlorite, 4. Sodium Hydroxide, 5. Sodium Chloride, 6. Sodium Bicarbonate, 7. Sodium Citrate, 8. Sodium Phosphate, 9. Sodium Sulfate, 10. Water.

INERT INGREDIENTS:

CAUTION: KEEP OUT OF REACH OF CHILDREN.

For information on the use of this product, contact your local health department or the manufacturer. For more information, contact the manufacturer.

NET CONTENTS ONE GALLON

NATIONAL LABORATORIES
LEHN & FINK INDUSTRIAL PRODUCTS DIVISION
of Sterling Drug Inc.
Montclair, N.J. 07045 Plant, Toledo, Ohio 43612

**FOR COLD DISINFECTION
OF SURGICAL
AND DENTAL
INSTRUMENTS**

Contains Rust Inhibitor

ACCEPTED

30 JUN 1961

INDENTED GENERAL INSTRUCTIONS
FOR THE USE OF THIS PRODUCT
SEE THE BOTTOM OF THE CONTAINER



LEHN & FINK

INSTRUMENT GERMICIDE

FOR COLD DISINFECTION OF SURGICAL AND DENTAL INSTRUMENTS

GERMICIDAL - Kills bacteria, fungi, and viruses.
FUNGICIDAL - Kills fungi.
VIRUCIDAL - Kills viruses.
ACTIVE INGREDIENTS - ...
INERT INGREDIENTS - ...

CAUTION: KEEP OUT OF REACH OF CHILDREN

NET CONTENTS ONE GALLON

NATIONAL LABORATORIES
LEHN & FINK INDUSTRIAL PRODUCTS DIVISION
Cincinnati, Ohio 45214