

675-17 FRONT AND BACK LABEL

HOSPITAL TYPE
TERGISYL
DETERGENT * DISINFECTANT
CLEANS AND DISINFECTS IN ONE OPERATION

GERMICIDAL* Staphylocidal Pseudomonacidal Tuberculocidal
FUNGICIDAL* (Against pathogenic fungi)
VIRUCIDAL (Against most Lipophilic viruses)

*On Environmental Surfaces

ACTIVE INGREDIENTS: o-Phenylphenol 5.7%
p-tert-Amphylphenol 1.8%

INERT INGREDIENTS: 92.5% (Includes detergent,
other cleaning agents
and no phosphorous
compounds)

Total 100.0%

EPA REG. NO. 675-17

WARNING: KEEP OUT OF REACH OF CHILDREN

Harmful if swallowed. Causes skin and severe eye irritation. Avoid contact with concentrate or dilution with skin or eyes. Wear rubber gloves when handling. In case of contact flush immediately with water and contact a physician. Avoid contamination of food. See side panel for First Aid.

STORAGE & DISPOSAL--Storage: Store in original container in areas inaccessible to small children. Keep away from food or feed. Disposal: Do not re-use empty container. Rinse thoroughly with water and detergent. Discard in trash.

NET CONTENTS ONE GALLON

NATIONAL LABORATORIES
LEHN & FINK PRODUCTS
MONTVALE, NJ 07645

PLANT: TOLEDO, OH 43612

2

675-17 LEFT SIDE PANEL

TERGISYL
DETERGENT · DISINFECTANT

TERGISYL IS IN CONCENTRATED FORM. DILUTE ACCORDING TO DIRECTIONS.
ONE TABLESPOON EQUALS ABOUT 1/2 FLUID OUNCES OR 15 CC.

To prepare proper dilutions in required quantities, add full strength TERGISYL to water as below:

PURPOSE	CORRECT SOLUTION STRENGTH	TERGISYL	WATER
FOR GENERAL CLEANING AND DISINFECTING	1% (1:100)	10 cc.	1 quart
		40 cc.	1 gallon
		4 oz.	3 gallons
		7 oz.	5 gallons

DIRECTIONS

No need to add soap or other detergents. TERGISYL does a complete job, cleans, disinfects, deodorizes. Avoid waste, dilute as directed. For complete directions, refer to Technical Brochure.

FIRST AID

Remove and wash contaminated clothing before reuse. If swallowed, promptly drink a large quantity of milk, egg whites, or gelatin solution. If these are not available, drink large quantities of water. Avoid alcohol. Call a physician.

675-15 LEFT PANEL

FOR USE BY PROFESSIONAL MAINTENANCE PERSONNEL ONLY IN INSTITUTIONAL AND INDUSTRIAL FACILITIES

SAVES TIME · LABOR · WEAR AND TEAR ON FLOORS AND EQUIPMENT