

FRONT PANEL

AMPHYL DISINFECTANT DEODORANT * DETERGENT

PHENOL COEFFICIENT 10

GERMICIDAL* staphylocidal, pseudomonacidal,
tuberculocidal.

FUNGICIDAL* (Against pathogenic fungi)

VIRUCIDAL * (Against most Lipophilic viruses
such as Influenza A₂, Herpes simplex and
Vaccinia)

*** On Environmental Surfaces**

ACTIVE INGREDIENTS:	O-phenylphenol	15.0%
	p-tert-Amylphenol	6.3%
INERT INGREDIENTS:	<u>78.7%</u>
	TOTAL	100.00%

EPA REGISTRATION NO. 675-21

DANGER:

KEEP OUT OF REACH OF CHILDREN


See first aid and precautions on side panel.

NATIONAL LABORATORIES

LEHN & FINK PRODUCTS

Montvale, New Jersey 07645 Plant: Toledo, Ohio 43612

Not Reviewed. Registrant claims to be in accordance with
Reregistration of Pesticide Product Guidance for Phase 2
Response, Page 2.12, Active to Inert Change in Status.

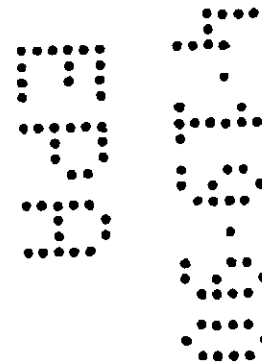

SIDE PANEL

HOW TO USE HOSPITAL TYPE AMPHYL

ALWAYS DILUTE AMPHYL ACCORDING TO DIRECTIONS


To prepare proper dilutions in required quantities, add full strength Amphy1 to water as below:

	CORRECT SOLUTION STRENGTH	AMPHYL	WATER
FOR GENERAL DISINFECTION- (Including areas occupied by tubercular patients) -- Floors, walls, and furniture should be mopped, wiped or scrubbed with the recommended solution to assure thorough wetting.		5 cc.	1 quart
	1/2 %	20 cc.	1 gallon
	(1:200)	40 cc.	2 gallons

FOR PREVENTING RUST in heat
sterilization -Amphyl prevents
the rusting action of boiling
water.

FOR DISINFECTION OF SURGICAL AND DENTAL INSTRUMENTS - Rinse off blood and exudates. Immerse instruments in Amphyl solution for 15 minutes. If Amphyl solution is to be stored for any period of time add 2 oz. of isopropyl alcohol to each quart of Amphyl solution. FOR DISINFECTION OF RUBBER GOODS. (e.g. tubing, gloves) - Rinse off blood and exudates. Immerse for 15 minutes. Remove, then immerse in 70% alcohol for one minute.		20 cc.	1 quart
	2 %	2 1/2 oz.	1 gallon
	(2:100)	5 oz.	2 gallons

ONE TABLESPOONFUL EQUALS 1/2 FLUID OUNCE OR 15 cc.

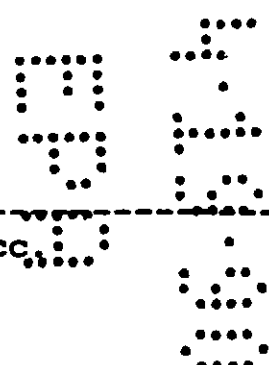

PAGE 2 OF SIDE PANEL

May be fatal or harmful if swallowed. Concentrate is corrosive to skin. Do not get concentrate or dilutions on skin or in eyes. Use rubber gloves for manual cleaning operations. Properly shield the eyes when handling the concentrate, in case of splashing. Avoid contamination of food. Do not get on clothing. Rinse empty container thoroughly with water and discard it.

FIRST AID: In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse. If swallowed, promptly drink a large quantity of milk, egg whites or gelatin solution: if these are not available drink large quantities of water. Avoid alcohol. Call a physician immediately.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

05
25
35
45
55
65
75
85
95