

NEW

Amphyl[®]

675-25 SPRAY JUN 1973

UNDE... AL INSECTICIDE
FUNGIC... FINE CIDE ACT
FOR ECONOMIC PESTICIDE REGISTER.
FD UNL... NO. 675-24

NEW Amphyl[®]

SPRAY U.S. Pat. No. 3,287,214

POWERFUL HOSPITAL SPRAY DISINFECTANT for environmental surfaces

such as: air ducts door handles dressing carts
bed springs linen carts, hampers stretcher wheels
toilet seats, urinals clean-up carts bassinets
garbage cans blood pressure sets wheel chairs

GERMICIDAL: Staphylocidal • Tuberculocidal • Pseudomonacidal
FUNGICIDAL

VIRUCIDAL (against most Lipophilic viruses—Including Herpes simplex
Influenza A₂ and Vaccinia).

KILLS MOLD AND MILDEW. Dries instantly, leaves no oily film.
Repeat application when new mildew growth appears.

QUICK ACTION DEODORIZER. Removes offensive odors from
dressing rooms, patient rooms, utility areas.

**WARNING—FLAMMABLE MIXTURE. DO NOT USE
NEAR FIRE OR FLAME N.Y.F.D.C. of A. No. 2815**

Avoid spraying in eyes. Avoid contact with food. Contents under
pressure. Do not puncture, use or store near heat or open flame.
Exposure to temperature above 120°F or in sun or discarding can
in fire or incinerator may cause bursting.

DIRECTIONS

DO NOT USE ON POLISHED WOOD FURNITURE OR
RAYON FABRICS.

TO DISINFECT:

1. Hold dispenser upright, 6" to 8" from surface.
2. Spray surface 2 to 3 seconds until thoroughly wet.

TO DEODORIZE:

1. Hold dispenser upright, pointing valve in
desired direction.
2. Spray into air 3 to 5 seconds, towards center
of room.

TO PREVENT MILDEW:

Spray surface until moist.

ACTIVE INGREDIENTS:

o-Phenylphenol	0.136%
N-Alkyl (C ₁₈ 92%; C ₁₆ 8%) -N- Ethylmorpholinium Ethylsulfates	0.045
3, 4, 5 Tribromosalicylanilide	0.019
3, 5 Dibromosalicylanilide	0.005
Alcohol	67.108

INERT INGREDIENTS: 32.687%

EPA Registration #6. 675-25-AA

DISINFECTANT DEODORANT

WARNING: KEEP OUT OF
REACH OF CHILDREN. See back
panel for additional precautions.

NET WEIGHT 14 oz. avdp.

NATIONAL LABORATORIES

LEHN & FINK INDUSTRIAL PRODUCTS DIV. OF STERLING DRUG, INC. EK-159
MONTVALE, N.J. 07645 PLANT TOLEDO, OHIO 43612

NEW

Amphyl

SPRAY

675-25

NEW Amphyl

POWERFUL HOSPITAL SPRAY DISINFECTANT for environmental surfaces such as

GERMICIDAL - Staphylocidal - Tuberculocidal - Pseudomonacidal
FUNGICIDAL

VIRUCIDAL against most Lipophilic viruses. Including Herpes simplex, Influenza A, and Vaccinia

KILLS MOLD AND MILDEW. Do not use on surfaces until dry for Repeat application when new mildew growth appears.

QUICK ACTION DEODORIZER. Eliminates offensive odors from dressing rooms, patient rooms, utility areas.

WARNING— FLAMMABLE MIXTURE. DO NOT USE NEAR FIRE OR FLAME N.Y.F.D.C. of A. No. 2815

Avoid spraying in eyes. Avoid contact with food. Contents under pressure. Do not puncture, use or store near heat or open flame. Exposure to temperature above 120 F or in sun or discarding can in fire or incinerator may cause bursting.

DIRECTIONS

DO NOT USE ON POLISHED WOOD FURNITURE OR RAYON FABRICS.

TO DISINFECT:

1. Hold dispenser upright, 6" to 8" from surface.
2. Spray surface 2 to 3 seconds until thoroughly wet.

TO DEODORIZE:

1. Hold dispenser upright, pointing valve in desired direction.
2. Spray into air 3 to 5 seconds, towards center of room.

TO PREVENT MILDEW:

Spray surface until moist.

ACTIVE INGREDIENTS:	o-Phenylphenol.....	0.136%
	N-Alkyl (C ₁₅ 92%; C ₁₆ 8%) - N-Ethylmorpholinium Ethylsulfates..	0.045
	3, 4, 5 Tribromosalicylanilide.....	0.019
	3, 5 Dibromosalicylanilide.....	0.005
	Alcohol.....	67.108
INERT INGREDIENTS:		32.687%

EPA Registration No. 675-25-AA

DISINFECTANT DEODORANT

WARNING: KEEP OUT OF REACH OF CHILDREN. See back panel for additional precautions.

NET WEIGHT 11 oz (311g)

NATIONAL LABORATORIES

LEVIN & HINE, INC., 1000 PINE BLVD., NEW BRUNSWICK, N.J. 08901

REGISTERED TRADEMARK OF NATIONAL LABORATORIES