

**DIRECTIONS**

To clean, disinfect and deodorize bed frames, window ledges, metal furniture and similar surfaces, use 2 ounces of NURSES' AID to each gallon of water and wash with a sponge.

To clean, disinfect and deodorize bed pans, urinals and similar bedside utensils, flush and wash with a swab using a minimum of 2 ounces of NURSES' AID to a gallon of water.

To clean, disinfect and deodorize rubber mattress covers, spray with a solution using a minimum of 1/2 ounce of NURSES' AID to a pint of water and wipe dry with a towel.

To clean and deodorize floors, mop with 1 to 1 1/4 ounces of NURSES' AID in each gallon of water. To clean and deodorize bathroom fixtures, wash with a sponge or swab using 1 to 1 1/2 ounces of NURSES' AID in each gallon of water.

To clean, disinfect and deodorize floors, use 2 ounces of NURSES' AID in each gallon of water. Thoroughly wet the floor with the disinfectant solution and pick up the solution with a separate mop and bucket or a vacuum pick up.

Use 2 ounces of NURSES' AID to each gallon of water for scrubbing. Thoroughly wet the floor with a sprayer or mop, machine scrub and pick up the solution with a separate mop and bucket.

# Nurses' Aid

**FOR HOSPITAL USE**

**ODOR CONTROL • DISINFECTING • CLEANING**

**A.O.A.C. Phenol Coefficient 7.5**

**ACTIVE INGREDIENTS:**

Isopropanol	7.40%
o-phenylphenol	5.90%
Tetrasodium salt of ethylene-	
diamine tetraacetic acid	5.90%
ortho-benzyl-p-chlorophenol	4.40%
p-tert amylphenol	0.60%
<b>INERT INGREDIENTS:</b>	<b>75.80%</b>

## DANGER

**Keep out of reach of children.**

**KEEP OUT OF EYES.**

**HARMFUL IF SWALLOWED.**


**AVOID CONTAMINATION OF FOOD.**

**See First Aid and Additional Precautions on Right Panel.**

**EPA Reg. No. 784-81**



**Consolidated Chemical, Inc.**  
St. Louis, Missouri 63110

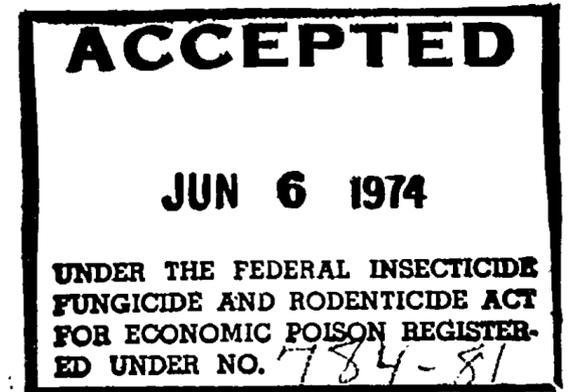

NUR  
soap  
bacte  
Stapl  
M. tu  
ment


**DA**

Keep  
swa  
of li  
a ph

Parti  
conce  
case  
If ex  
and  
Use  
On e  
water  
Avoic

When  
water

**NET**

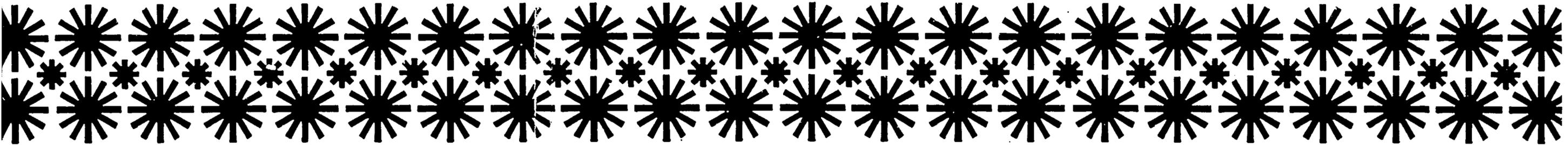


# Nurses' Aid

**FOR HOSPITAL USE**  
**ODOR CONTROL • DISINFECTING • CLEANING**

A.O.A.C. Phenol Coefficient 7.5

ACTIVE INGREDIENTS:	
Isopropanol	7.40%
o-phenylphenol	5.90%
Tetrasodium salt of ethylene-	
diamine tetraacetic acid	5.90%
ortho-benzyl-p-chlorophenol	4.40%
p-tert amylphenol	0.60%
INERT INGREDIENTS:	75.80%

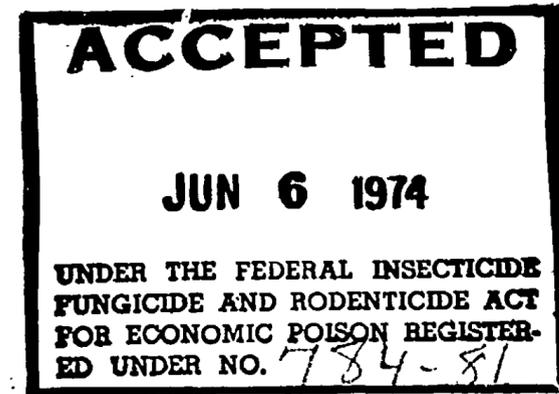

**DANGER**  
**Keep out of reach of children.**  
**KEEP OUT OF EYES.**  
**HARMFUL IF SWALLOWED.**  
**AVOID CONTAMINATION OF FOOD.**  
**See First Aid and Additional Precautions on Right Panel.**

EPA Reg. No. 784-81

 **Consolidated Chemical, Inc.**  
St. Louis, Missouri 63110

NURSES' AID is a concentrated, hospital type soapless detergent that kills many pathogenic bacteria including Pseudomonas aeruginosa, Staphylococcus aureus, Salmonella choleraesuis, M. tuberculosis and Salmonella typhosa on environmental surfaces when used as directed.

## DANGER

**Keep out of reach of children. Harmful if swallowed. First Aid: Drink a large quantity of liquids, do NOT induce vomiting, consult a physician.**

Particular care should be taken in handling the concentrate, including proper shielding of eyes in case of splashing. May cause eye damage. If exposed, flush with water for fifteen minutes and consult physician. May cause skin irritation. Use rubber gloves for manual cleaning operations. On exposure, wash immediately with soap and water. Remove and wash contaminated clothing. Avoid contamination of food.

When empty, rinse this container thoroughly with water and discard.

**NET CONTENTS: ONE GALLON**

