

2212-17

7-2-2002

1/4

Bottle LABEL FRONT:

LEGPHENE 2

Germicidal Hospital Grade Ultrasonic Cleaner Concentrate
One step: Detergent, Disinfectant, Deodorant,
Germicidal, Fungicidal, Virucidal, Tuberculocidal.

Active Ingredients:

Ortho-benzyl-para-chlorophenol	3.80%
Para-tertiary-amylphenol	3.74%
Ortho-phenylphenol	2.35%
Inert Ingredients	<u>90.11%</u>
Total	100.00%

DANGER: KEEP OUT OF REACH OF CHILDREN

Refer to back label and hang tag for additional information.

Manufactured by:


Walter G. Legge Company, Inc.
 444 Central Ave
 Peekskill NY 10566
 914-737-5040 Fax 914-737-2636
 E.P.A. Registration No. 2212-17
 E.P.A. Establishment No. 2212-NY-1
 Contents 16 fl. oz.
 (473 ml)
 Product Code UD036

1 ounce concentrate makes 1 gallon solution.

ACCEPTED

JUL -2 2002

Under the Federal Insecticide, Fungicide, and
 Rodenticide Act as amended, for the
 pesticide, registered under
 EPA Reg. No. 2212-17

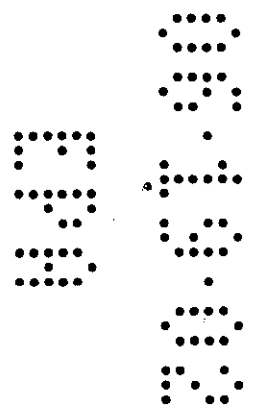

2/4

Bottle LABEL BACK

**PRECAUTIONARY STATEMENT
HAZARD TO HUMAN AND DOMESTIC ANIMALS**


DANGER:

CORROSIVE. Causes eye damage and skin irritation. Do not get in eyes, on skin, or on clothing. Wear goggles or face shield and rubber gloves when handling. Harmful if swallowed. Avoid contamination of food.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. **FOR USE IN ULTRASONIC CLEANING MACHINE.** LEGPHENE 20 Germicidal Hospital Grade Ultrasonic Cleaner Concentrate is designed to be used in an Ultrasonic Cleaning System as a cleaner or decontaminant **prior to terminal sterilization or high level disinfection** of non-porous items such as instruments or appliances.

See hang tag for additional precautions and First Aid.

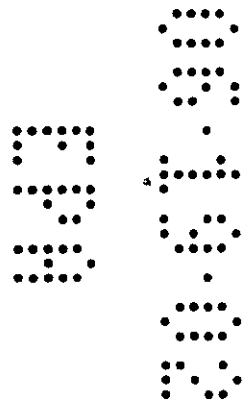

HANG TAG FRONT:

3/4

LEGPHENE 2

Germicidal Hospital Grade Ultrasonic Cleaner Concentrate
1oz/gallon

One step: Detergent, Disinfectant, Deodorant,
Germicidal, Fungicidal, Virucidal, Tuberculocidal.

Active Ingredients:

Ortho-benzyl-para-chlorophenol	3.80%
Para-tertiary-amylphenol	3.74%
Ortho-phenylphenol	2.35%
Inert Ingredients	<u>90.11%</u>
Total	100.00%

Contents: 16 fl. oz. (473 ml) makes 16 gallons

KEEP OUT OF THE REACH OF CHILDREN

DANGER:

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

DIRECTIONS FOR USE: For use in Ultrasonic Cleaning Machines. LEGPHENE 2[®] Germicidal Hospital Grade Ultrasonic Cleaner Concentrate is designed to be used in an Ultrasonic Cleaning System as a cleaner or decontaminant **prior to terminal sterilization or high level disinfection** of non-porous items such as instruments or appliances.

DILUTION: To supply the proper dose of solution, remove cap from unit dose chamber and squeeze bottle to fill reservoir. Use at a dilution level of ½ ounce of LEGPHENE 2[®] Germicidal Cleaner per half gallon of water or 1 ounce per gallon of water (1:128). Pour concentrate into tray, pan or ultrasonic tank and add water to recommended level.

PRE-SOAK: Place contaminated instruments in a holding tray or auxiliary pan filled with properly diluted LEGPHENE 2[®] Germicidal. Transfer to a L&R Ultrasonic Cleaning System also filled with LEGPHENE 2[®] Germicidal when ready for cleaning cycle. Dispose of solution daily.

DECONTAMINATION: Place contaminated instruments and/or appliances after they have been thoroughly cleaned (preferably in an L&R Ultrasonic Cleaning System) into properly mixed LEGPHENE 2[®] Germicidal. Make sure to properly rinse and dry instruments prior to placing in tray or pan. Completely immerse for 10 minutes at room temperature (20°-23° C/68°-72° F) for proper decontamination. Instruments and appliances must be removed and thoroughly rinsed prior to terminal sterilization or high level disinfection. Change solution daily or sooner if required.

ULTRASONIC CLEANING EQUIPMENT: Add LEGPHENE 2 Germicidal Hospital Grade Ultrasonic Cleaner Concentrate and water, at the proper dilution ratio, as stated above (1:128) to your ultrasonic machine. Fill the basket with pre-cleaned instruments, place cover on machine and run a cycle of 10 minutes. DO NOT use the heater on the machine, as this will cause vaporization of the germicide. After completing the cycle, turn off machine and lift the basket, allowing drainage and then rinse in running water. The solution should be replaced daily, more often if heavy contamination occurs. DO NOT MIX WITH ANY OTHER MATERIALS.

NOTE: This product is not to be used as a terminal sterilant / high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

HANG TAG BACK:**VIROCIDAL PERFORMANCE**

At 1 oz. per gallon use level, LEGPHENE 2 was evaluated in the presence of 5% serum and 400 ppm hard water for a contact time of 10 minutes at room temperature (20° - 23° C) against the HIV-1 (AIDS VIRUS) found on inanimate, hard, non-porous environmental surfaces. *Kills HIV on pre-cleaned environmental surfaces/objects previously soiled with blood/body fluids in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS).

"SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON

SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS" Contact Time: Leave surface wet for 10 minutes with 1 oz. per gallon use dilution. Cleaning Procedure: Blood and other body fluids containing HIV must be thoroughly cleaned from surfaces and objects before application of LEGPHENE 2 Germicidal.

Personal Protection: Clean up should always be done wearing protective latex gloves, gowns, masks, and eye protection.

Material: Blood, body fluids, cleaning materials, and clothing should be autoclaved and disposed of according to local regulations for infectious waste disposal.

GERMICIDAL ACTIVITY: LEGPHENE 2 Germicidal is effective against the following organisms as demonstrated by the AOAC test method, modified in 400 ppm hard water (as CaCO₃): Mycobacterium Bovis (TB), HIV-1, Staphylococcus aureus, Salmonella choleraesuis and Pseudomonas aeruginosa. Additional testing was performed using the standard AOAC test method. Representative organisms include: Aeratia marcescens, Influenza A2 (Hong Kong), Herpes simplex, Vaccinia (pox virus), Trichophyton interdigitale, and Escherichia coli.

PRECAUTIONARY STATEMENTS**HAZARDS TO HUMANS AND DOMESTIC ANIMALS**

DANGER; Corrosive. May cause irreversible eye damage and skin burns. Do not get in eyes, on skin, or on clothing. Wear protective eye wear (goggles, face shield or safety glasses), protective clothing and rubber gloves. Harmful if swallowed and/or if absorbed through the skin. Wash thoroughly with soap and water after handling. Remove contaminated clothing before reuse.

FIRST AID

Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling poison control center or doctor or going for treatment.

IF IN EYES: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after first 5 minutes then continue rinsing eye.

IF ON SKIN: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes.

IF SWALLOWED: Call a poison control center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by a poison control center or doctor. Do not give anything by mouth for an unconscious person.

IF INHALED: Move person to fresh air. If person is not breathing, call 911 or an ambulance then give artificial respiration, preferably by mouth-to-mouth, if possible. Call a poison control center or doctor for further treatment advice.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of gastric lavage.

ENVIRONMENTAL HAZARDS

Do not discharge effluent containing this product into lakes, streams, ponds, estuaries, oceans or other waters unless in accordance with the requirements of a National Pollutant Discharge Elimination Systems (NPDES) permit and the permitting authority has been notified in writing prior to discharge. Do not discharge effluent containing this product to sewer systems without previously notifying the local sewage treatment plant authority. For guidance, contact your State Water Board or Regional Office of the EPA.

STORAGE AND DISPOSAL

Do not store on side. Avoid creasing or impacting of side walls. Do not reuse empty container. Discard in trash or recycle.

PESTICIDE DISPOSAL

Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray or mixture of rinseate is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste representative at the nearest EPA Regional Office for guidance.

CONTAINER DISPOSAL

Triple rinse (or equivalent). Then offer for recycling or reconditioning or puncture and dispose of in a sanitary landfill, or by incineration or if allowed by state and local authorities by burning. If burned, stay out of smoke.

CAS NUMBER

7732-18-5

67-63-0

120-32-1

80-46-6

90-43-7

CHEMICAL NAME

Water

Isopropanol

o-benzyl-p-chlorophenol

para-tertiary-amyphenol

o-phenylphenol

CAS NUMBER

119345-4-9

64-02-8

7757-83-7

1310-58-3

Mixture

CHEMICAL NAME

DOWLEX ZAI Solution

Tetrasodium EDTA Tetrahydrate

Sodium sulfite

Potassium hydroxide

Perfume