

TO PREPARE SOLUTION:

Place 2 tablets in 1 pint of warm water, crush and stir until completely dissolved.

CAUTION

"Caution, harmful if swallowed; avoid contact with skin and eyes; in case of contact, flush with plenty of water; avoid contamination of food."

ACTIVE INGREDIENTS:

"Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium

Chloride monohydrate
25%

INERT INGREDIENTS:

75%

Net weight - 35¼ ozs.

500 2-Gram Tablets

ACMI
CYSTAN
Green Tablets

See Left panel for precautionary information

American Cystoscope Makers, Inc.

8 Pelham Parkway, Pelham Manor (Pelham), N.Y. 10803

CATALOG NO. 2595A

DIRECTIONS:

To prepare cystoscope or similar instruments for use, clean instruments by washing in tepid water using green soap and soft brush to reach all contours and crevices. Rinse instruments and immerse them for half an hour in CYSTAN solution. Rinse again in sterile water before using.

"USDA Reg. No. 3865 - 4"

