



# OCIDE HOSPITAL SURFACE DISINFECTANT DEODORANT

**Kills mold and mildew  
Inhibits further growth up to 7 days**  
Also kills

*Mycobacterium tuberculosis,*  
*Pseudomonas aeruginosa,*  
*Staphylococcus aureus,*  
and *Salmonella choleraesuis*

### WARNING

KEEP OUT OF REACH OF CHILDREN. Contents under pressure. See back panel for additional precautions.

**NET WT. 18 OZ. (1 LB. 2 OZ.)**

### USES

This Hospital Disinfectant destroys mold and mildew on contact. It deodorizes and kills most bacteria on hard surfaces in hospital operating rooms, emergency rooms, patients rooms, clinics. It kills most bacteria and mildew on surgical carts, laundry, toilet, bathroom fixtures and patient care items such as bedside tables and bed pans. This Hospital Disinfectant destroys on surfaces most bacteria that cause odors — does not just mask odors.

### DIRECTIONS

**FOR DISINFECTING:** Hold can upright about 6 to 10 inches from surface to be disinfected and spray for three seconds.  
**FOR DEODORIZING:** Hold dispenser upright and spray toward ceiling for 6 to 8 seconds in an average size room (10x10).  
**FOR MILDEW:** Clean the surface prior to spraying. Spray the surface for three seconds. Repeat once a week to inhibit further mildew growth.

### ACTIVE INGREDIENTS:

o-phenyl-phenol .....	0.172%
2-Chloro-4-phenylphenol.....	0.035%
Ethanol .....	67.646%
<b>INERT INGREDIENTS .....</b>	<b>32.147%</b>

### WARNING

Contents under pressure. Do not puncture. Do not use or store near heat or open flame. Exposure to temperatures above 130°F may cause bursting. Never throw container into fire or incinerator. Avoid spraying in eyes or on food or food stuffs. In case of contact, immediately flush eyes with plenty of water. If irritation persists, get medical attention. Keep out of reach of children.

EPA Reg No 4313-44

EPA Est 10807-GA-1

4313-44

Mfd. for

**CARROLL COMPANY**

2900 W. Kingsley Rd. • Garland, Texas 75041

Cameraman's Note

Poor Copy