

PM 31 4330-1

10/2

FOR PROFESSIONAL USE • CONCENTRATE - MUST BE DILUTED

Waverly
ANTI-RUST

SURGISEPT

DISINFECTANT, FUNGICIDE AND VIRUCIDE •

ACTIVE INGREDIENTS:

n-alkyl C₁₂, 50%, C₁₂, 40%, C₁₂, 10%
dimethyl benzyl ammonium chloride 3%

INERT INGREDIENTS 97%

Contains Sodium Nitrite

DANGER

KEEP OUT OF REACH OF CHILDREN

STATEMENT OF PRACTICAL TREATMENT

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse. If swallowed, promptly drink large quantities of water. Avoid alcohol. Call a physician.

- NOTE TO PHYSICIAN -

Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

SEE BACK PANEL FOR ADDITIONAL PRECAUTIONARY STATEMENTS AND DIRECTIONS

CONTENTS 128 FL. OZ. (1 GALLON)

EPA, EST. NO 4330-NY01 • EPA REGISTRATION NO. 4330-1

WAVERLY BEAUTY PRODUCTS, INC.

Holtville, NY 11742

BEST AVAILABLE COPY

2 of 2

**KILLS HIV-1 (AIDS VIRUS) ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD AND BODY FLUIDS.
SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV OF SURFACES/OBJECTS SOILED WITH BLOOD AND BODY FLUIDS**

On pre-cleaned environmental surfaces/objects in health-care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids in which the surfaces/objects likely to be soiled can be associated with the potential for transmission of HIV.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

PERSONAL PROTECTION

When handling items soiled with blood or body fluids, use appropriate barrier protections such as disposable latex gloves, gowns, masks or eye coverings.

CLEANING PROCEDURE

Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfectant.

CONTACT TIME

Leave surface wet for 10 minutes.

DISPOSAL OF INFECTIOUS MATERIALS

Blood and other body fluids should be autoclaved and disposed of according to Federal, State and local regulations for infectious waste disposal.

VIROCIDAL PERFORMANCE: At 4 oz. to a gallon use-level was evaluated in the presence of 5% serum and found to be effective against HIV-1 (AIDS virus) on inanimate hard non-porous environmental surfaces.

FUNGICIDAL PERFORMANCE: At 4 oz. to a gallon use-level is an effective fungicide against Trichophyton mentagrophytes (the athlete's foot fungus) when used on surfaces in areas such as locker rooms, dressing rooms, shower and bath areas and exercise facilities.

DISINFECTING PERFORMANCE: At 4 oz. to a gallon use-level apply to previously cleaned hard surfaces. Is effective against Pseudomonas aeruginosa, Staphylococcus aureus and Salmonella choleraesuis.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Apply Surgisept with a cloth, mop or mechanical spray device. When applied with a mechanical spray device, surface must be sprayed until thoroughly wetted. Treated surfaces must remain wet for 10 minutes. Fresh solution should be prepared daily or when the use-solution becomes visibly dirty.

DISINFECTION IN HOSPITALS, NURSING HOMES AND OTHER HEALTH CARE INSTITUTIONS: For disinfecting floors, walls, countertops, bathing areas, lavatories, bedrooms, tables, chairs, garbage pails and other hard non-porous surfaces. Add 4 oz. Surgisept to 1 gallon water. Apply to previously cleaned surfaces.

FOR BARBER AND BEAUTY SHOP EQUIPMENT DILUTION: 1 oz. (2 tablespoons) to 1 quart water or 4 oz. to 1 gallon. **USE:** Completely immerse cleaned implements (free from soap) in SURGISEPT solution for 60 seconds (or as required by local authorities - usual requirements, 10 minutes).

PRECAUTIONARY STATEMENT

Hazards to Humans and Domestic Animals. **DANGER:** Keep out of reach of Children. Corrosive. Causes eye damage and skin irritation. Do not get in eyes, on skin or on clothing. Wear goggles, face shield and rubber gloves. Harmful if swallowed. Avoid contamination of food.

STORAGE AND DISPOSAL

Do not contaminate water or food by storage or disposal. Avoid creasing or impacting of side walls. Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray or mixture of insecticide is a violation of Federal Law. Wastes resulting from the use of this product may be disposed of on site or at an approved waste disposal facility.

CONTAINER DISPOSAL

Do not reuse empty container. Wrap container and put in trash.

BEST AVAILABLE COPY

ACCEPTED
FEB 01 1995
Under the Federal Insecticide, Fungicide, and Herbicide Act, as amended, for the pesticide registered under EPA Reg. No. 7330-1