

Twelve 10cc Vials



Net Weight 0.37 oz. each

**SURGICIDE**  
BRAND OF  
**CETYL DIMETHYL ETHYL AMMONIUM BROMIDE**

Germicidal Concentrate for Chemical Disinfection of  
Surgical and Dental Instruments

**ACTIVE INGREDIENTS:**

Cetyl Dimethyl Ethyl Ammonium Bromide	-----	26.0%
Isopropyl Alcohol	-----	13.0%
Triethanolamine	-----	10.0%
<b>INERT INGREDIENTS:</b>	-----	<u>51.0%</u>
<b>Total</b>	-----	<b>100.0%</b>

Contains sodium nitrite

**DANGER: KEEP OUT OF REACH OF CHILDREN.**

NOT for injection. Harmful or fatal if swallowed. Do not get in eyes, on skin or on clothing. Avoid contamination of food. Read carefully other precautions elsewhere on label.

EPA Reg. No. 4476-37



**MORTON PHARMACEUTICALS, INC**  
ETHICAL DISTRIBUTORS MEMPHIS TENN 38108

**DIRECTIONS -**

**FOR INSTRUMENTS, GLOVES, CONTAINERS, ETC.:**

Add contents of Surgicide Vial to one liter of tap water. If water supply is known to be hard, use distilled water. Clean and rinse instruments well prior to immersion in the solution. Do not introduce soap or synthetic detergents into solution. Allow instruments to remain in solution for a minimum of fifteen (15) minutes.

To ensure disinfection, articles must be cleansed thoroughly and immersed completely in the solution for at least 15 minutes.

The disinfectant should be replaced periodically. Surgicide will not kill spores and *Mycobacterium tuberculosis* or destroy the etiologic agent for viral hepatitis. It should not be relied on to kill *Pseudomonas aeruginosa* or to disinfect hy-

Signal w to be in 12 pt. type

Remainder of this portion to be in 8 pt. type

TOP OF BOX

SIDE AND BOTTOM

Remainder of this

tion. Do not introduce soap or synthetic detergents into solution. Allow instruments to remain in solution for a minimum of fifteen (15) minutes.

To ensure disinfection, articles must be cleansed thoroughly and immersed completely in the solution for at least 15 minutes.

The disinfectant should be replaced periodically. Surgicide will not kill spores and *Mycobacterium tuberculosis* or destroy the etiologic agent for viral hepatitis. It should not be relied on to kill *Pseudomonas aeruginosa* or to disinfect hypodermic needles, syringes, and articles with hinges, deep narrow crevices, or defective plating.

**DANGER:**

Keep Out of Reach of Children. Undiluted solution causes eye damage and severe skin irritation. Do not get in eyes, on skin or on clothing. Wear goggles or face shield and rubber gloves when handling. Harmful or fatal if swallowed. Avoid contamination of food.

**FIRST AID:**

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse.

If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution or if these are not available, drink large quantities of water. Avoid alcohol. Call a physician immediately.

**NOTE TO PHYSICIAN:**

Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

IDE AND BOTTOM OF BOX

GENCO 10111111

4971 01