

Directions for use and activation must be followed.

Record Activation and Expiration dates on label.

Formulated for the Disinfection or Sterilization of:

Sharp Instruments - Thermometers Oral and Rectal - Metal and Rubber Objects -
Lensed Instruments - Plastic Objects.

The directions recommended for sterilization must be used for the treatment of
respiratory therapy equipment, anesthesia equipment, or for any other item where
a tubercuoid product is desired.

STORAGE AND DISPOSAL

STORE IN A COOL PLACE

PROHIBITIONS:

Do not contaminate water, food or feed by storage or disposal. Open dumping is
prohibited.

Do not reuse empty container.

PESTICIDE DISPOSAL:

Pesticide spray mixture or rinse water that cannot be used according to label
instructions must be disposed of according to Federal or approved state
procedures under Subtitle C of the Resource Conservation and Recovery Act.

CONTAINER DISPOSAL:

Triple rinse (or equivalent). Then offer for recycling or reconditioning, or dispose
of in a sanitary landfill, or by incineration if allowed by state and local authorities.

GENERAL:

Consult Federal, State or Local Disposal authorities for approved alternative
procedures.

PRECAUTIONARY STATEMENTS:

HAZARDS TO HUMANS AND DOMESTIC ANIMALS

DANGER: Corrosive. Causes burns. Causes eye damage and skin irritation. Do
not get in eyes, on skin or on clothing. May be absorbed through skin. Wear goggles
or face shield and rubber gloves when handling.

HARMFUL IF SWALLOWED. Avoid contact with food. Use in ventilated area.

STATEMENT OF PRACTICAL TREATMENT

EYES: - In case of eye contact, immediately flush with water and seek prompt
medical attention.

SKIN: - In case of skin contact, immediately flush thoroughly with water.

INGESTION: - If swallowed, drink milk, egg whites, gelatin solution or, if these are
not available, large quantities of water. Avoid alcohol. Call a physician.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of
gastric lavage. Measures against circulatory shock, respiratory depression and
convulsion may be needed.

Harm can be caused to carbon steel instruments immersed for over 24 hours.

Steril-ize™ Solution contains a rust inhibitor but will not inhibit rust caused by
poor or damaged plating or improper care of instruments.

Net Contents: One Gallon (3.785 Liters)

ORDER NO. LL1-250

Aqueous I

S

Bactericidal
Pseudomonal
Virucidal
Fungicidal
Sporicidal
Tubercu-


activat

- Rapid-Acti
- Rust-Inhib
- Non-Stain

DANGER



Manufactured by
E.P.A. Reg. No. 5367-7
E.P.A. Est. No. 5367-Pa

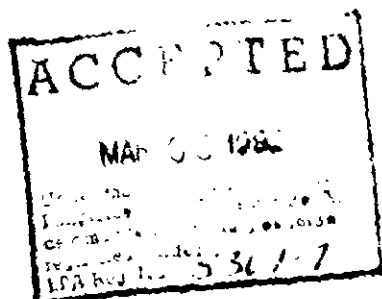

Aqueous Disinfecting and Sterilizing Solution

Steril-Ize

Bactericidal
Pseudomonacidal
Virucidal*
Fungicidal
Sporicidal
Tuberculocidal**

T.M.

activated dialdehyde

- Rapid-Acting
- Rust-Inhibiting
- Non-Staining

ACTIVE INGREDIENT:
Glutaraldehyde 2.0%
INERT INGREDIENTS: 98.0%
100.0%

DANGER

KEEP OUT OF REACH OF CHILDREN
SEE LEFT PANEL FOR ADDITIONAL
PRECAUTIONARY STATEMENTS

Manufactured by
E.P.A. Reg. No. 5367-7
E.P.A. Est. No. 5367-Pa-1

Larson Laboratories, Inc.
Erie, Pennsylvania 16505

DIRECTIONS FOR USE

For use as directed on labels. Use with caution in the manner indicated on the label.

DIRECTIONS FOR ACTIVATING

To prepare the activated stock solution for disinfection and sterilization, add contents of the attached vial to one gallon of the solution. Do not dilute. The shelf life of the activated unused stock solution is 14 days.

For use in Hospitals, Nursing Homes and Health Care Institutions.

For disinfection or sterilization of medical instruments, equipment and apparatus, the items must be thoroughly cleaned, rinsed and thoroughly dried before immersion in the full strength activated solution. Flush and thoroughly rinse any hollow instrument. Avoid entrapment of air bubbles.

DISINFECTION: Immerse item completely for 10 minutes at room temperature.

STERILIZATION: Immerse item completely for 10 hours at room temperature.

Using aseptic technique, remove item from the solution and rinse thoroughly with sterile water. Discard the used solution after each day's usage.

*When used as directed for Disinfection, Steril-Ize™ will kill vegetative pathogens such as *Pseudomonas aeruginosa* and viruses (Poliovirus type 1, 2, Coxsackie B-1, B-3, Echo type 6, Adenovirus type 2, Herpes simplex, Influenza A2, Hong Kong, Vaccinia) on surfaces of inanimate objects.

**When used as directed for Sterilization, Steril-Ize™ will kill resistant pathogenic spores of *Clostridium sporogenes*, *Bacillus subtilis*, *Clostridium tetani* and *Mycobacterium tuberculosis*.

Activation
Date

Expiration
Date

