

11 2: 5749-8

27 1

U.S. ENVIRONMENTAL PROTECTION AGENCY
OFFICE OF PESTICIDES PROGRAMS
REGISTRATION DIVISION (75-767)
WASHINGTON, DC 20460

EPA REGISTRATION NO.
5749-8

EXPIRES
FEB 01 1995

TERM OF ISSUANCE

NOTICE OF PESTICIDE: REGISTRATION
RERE REGISTRATION
(Under the Federal Insecticide, Fungicide,
and Rodenticide Act, as amended)

NAME OF PESTICIDE PRODUCT

Vaprox Hydrogen Peroxide
Sterilant

NAME AND ADDRESS OF REGISTRANT (Include ZIP code)

AMSCO
American Sterilizer Company
P.O. Box 62
424 W. 23rd Street
Erie, PA 16506

NOTE: Changes in labeling formula differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Registration Division prior to use of the label in commerce. In any correspondence on this product always refer to the above U.S. EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby Registered/Reregistered under the Federal Insecticide, Fungicide, and Rodenticide Act.

A copy of the labeling accepted in connection with this Registration/Reregistration is returned herewith.

Registration is in no way to be construed as an endorsement or approval of this product by this Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

This product is conditionally registered in accordance with FIFRA sec. 3(c)(7)(A) provided that you:

1. Submit and/or cite all data required for registration/ reregistration of your product under FIFRA sec. 3(c)(5) and sec. 4 when the Agency requires all registrants of similar product to submit such data.
2. Make the labeling changes listed below before you release the product for shipment:
 - a. Add the phrase, "EPA Registration No. 5749-8" to your label before you release the product for shipment.
 - b. On the product labels (not cartridge labels):
 1. Place the ingredient section above the child hazard warning statement and signal word.
 2. Delete "General Classification" and "First Aid".
 - c. Place the signal word "DANGER" above the child hazard warning statement. (Product and Cartridge labels)

ATTACHMENT IS APPLICABLE

SIGNATURE OF APPROVING OFFICIAL

DATE

- d. Revise your use directions to read:

Directions For Use

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling


This product is intended for use as a non-medical/ non-health care sterilant (i.e., for industrial use only). This product is for use in AMSCO VHP Industrial Equipment only, and only by trained personnel. Use this product for sterilization accordance with procedures defined in the Equipment Manual.

3. Submit five (5) copies of final printed labeling before you release the product for shipment. Refer to the A-79 Enclosure for a further description of final printed labeling.

If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA sec. 6(e). Your release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records.

Sincerely yours,

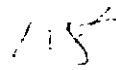

Walter C. Francis
Acting Product Manager (31)
Antimicrobial Program Branch
Registration Division (7505C)

Enclosure

BEST AVAILABLE COPY

AMSCO
VAPROX™
 HYDROGEN PEROXIDE
 STERILANT 200 ML
 REORDER NO. PB009

ACTIVE INGREDIENT: 35% Hydrogen Peroxide
 INERT INGREDIENT: 65% Water
 INGRÉDIENT ACTIF: 35% Peroxyde hydrogène
 INGRÉDIENT INERTE: 65% Eau

KEEP OUT OF REACH OF CHILDREN.
DANGER
 OXIDIZER;
 CORROSIVE
 SEE BACK PANEL FOR STATEMENT OF PRACTICAL TREATMENT.
 AMSCO CANADA, INC., WOODBRIDGE ON
 DIN 02047616

CONSERVER HORS DE LA PORTÉE DES ENFANTS.
DANGER
 OXYDANT; CORROSIF
 VOIR PRÉCAUTIONS D'EMPLOI AU DOS.
 AMSCO CANADA, INC., WOODBRIDGE ON
 AMV 01
 DIN 02047616

VAPROX™ HYDROGEN PEROXIDE UN2014 STERILANT

STATEMENT OF PRACTICAL TREATMENT

Do not get in eyes! IN CASE OF EYE CONTACT: Immediately flush with plenty of water for at least 15 minutes, lifting upper and lower lids intermittently. Call a physician.

Do not take internally! IF SWALLOWED: Drink plenty of water immediately to dilute. Do not induce vomiting. Never give anything by mouth to an unconscious person. Call a physician.

Avoid breathing vapors! USE ONLY WITH ADEQUATE VENTILATION! IF INHALED: Remove to fresh air. If not breathing, give artificial respiration. If breathing is difficult, give oxygen. Call a physician.

Avoid contact with skin and clothing! IN CASE OF SKIN CONTACT: Immediately flush skin with plenty of water for 15 minutes while removing contaminated clothing and shoes. Get medical attention if irritation persists. Wash contaminated clothing promptly. Discard contaminated shoes in a fireproof container.

SEE EQUIPMENT MANUAL AND MATERIAL SAFETY DATA SHEET FOR ADDITIONAL INFORMATION.

POUR PLUS D'INFORMATIONS, CONSULTER LE MANUEL DE L'ÉQUIPEMENT ET LA FICHE TECHNIQUE SUR LA



PRECAUTIONARY STATEMENT
Hazard to humans and environment
DANGER OXIDIZER, CORROSIVE
 Liquid hydrogen peroxide is harmful if swallowed and is harmful if swallowed. Direct eye contact may cause irritation and blindness. Wear goggles and neoprene gloves when handling. Irritation to the eyes, nose and throat.
CHEMICAL AND PHYSICAL HAZARD
 Liquid hydrogen peroxide is a strong oxidizer. It may cause fire, explosion or reaction with combustible materials. Clothing or other materials that come into contact with it may become ignited. Immediately and thoroughly dry in the materials, a fireproof container.
IN CASE OF FIRE use water or other extinguishing agents. When handling use an MSIA approved respirator, goggles or full face shield, protective clothing and boots.
CONTAIN SPILLS and clean up immediately. Notify the spill, such as 1-800-424-9303. Its SO2 equivalent per 100 g destroys the peroxide.

ACCEPTED with COMMENTS in EPA Letter Dated:

FEB 01 1995

Under the Federal Insecticide, Fungicide, and Herbicide Act as amended, for the pesticide registered under EPA Reg. No. 5749-8

AMSCO Corporate Design Artwork

CDA Date

Part No. 400077-597 Art No.

PROTO 1

Size 11.625 X 3.4375

"Draft Cartridge Label"

DIRECTIONS FOR USE

It is a violation of federal law to use this product in a manner inconsistent with its labeling.

~~It is a violation of federal law to use this product in a manner inconsistent with its labeling.~~
~~It is a violation of federal law to use this product in a manner inconsistent with its labeling.~~
~~It is a violation of federal law to use this product in a manner inconsistent with its labeling.~~

STORAGE AND DISPOSAL

Store cartridges at or below 77°F or 25°C. Do not expose to cyanide, hexavalent chromium compounds, other oxidizers, reducers, combustible materials, or flammable vapors.

PESTICIDE DISPOSAL

Dilute contents of cartridge with 20 parts water and then empty into sink with running water. Hydrogen Peroxide is classified as a DOT oxidizer and is hazardous waste under U.S. EPA hazardous waste regulation, and it is a violation of federal law to improperly dispose of peroxides.

CONTAINER DISPOSAL

Rinse cartridge with running water and dispose of with normal nonhazardous waste.

LOT NUMBER: Not for sale of use after

[Empty rectangular box for Lot Number]

EPA REG. NO.

EPA EST. NO. 5749 NC 1

AMSCO, APEX NC 27502 400077-597 11/30/94

BEST AVAILABLE COPY

This product is intended for use as a non-medical/non-health care sterilant (i.e., for industrial use only). This product is for use in AMSCO VHP Industrial Equipment only, and only by trained personnel. Use this product for sterilization in accordance with procedures defined in the Equipment Manual.