pm31

5749-8

1/5

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DEC 18 1996

Dawn Cole  
Steris Corporation  
5960 Heisley Road  
Mentor, OH 44060

SUBJECT: Vaprox Hydrogen Peroxide  
EPA Registration No. 5749-8 *Transferred to 58779-4*  
Notification Per PR-Notice 95-2

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**PRECAUTIONARY STATEMENTS**  
Hazard to humans and domestic animals.  
**DANGER! OXIDIZER, CORROSIVE**  
Liquid hydrogen peroxide causes eye damage, skin irritation, and is harmful if swallowed, inhaled or absorbed through the skin. Direct eye contact may cause irreversible tissue damage and blindness. Wear goggles or face shield and vinyl or neoprene gloves when handling open containers. Vapors cause irritation to the eyes, nose, throat, and lungs.

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**DANGER** KEEP OUT OF REACH OF CHILDREN. OXIDIZER. CORROSIVE. SEE BACK PANEL FOR STATEMENT OF PRACTICAL TREATMENT. FOR USE IN AMSCO VHP® INDUSTRIAL EQUIPMENT ONLY.  
 DIN 02047616  
 AMSCO CANADA, INC. Etobicoke, Ontario

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VAPROX™ UN2014  
 HYDROGEN PEROXIDE STERILANT

**STATEMENT OF PRACTICAL TREATMENT**  
 Do not get in eyes! **IN CASE OF EYE CONTACT:** Immediately flush with plenty of water for at least 15 minutes, lifting upper and lower lids intermittently. Call a physician. Do not take internally! **IF SWALLOWED:** Drink plenty of water immediately to dilute. Do not induce vomiting. Never give anything by mouth to an unconscious person. Call a physician. **Avoid breathing vapors! USE ONLY WITH ADEQUATE VENTILATION! IF INHALED:** Remove to fresh air. If not breathing, give artificial respiration. If breathing is difficult, give oxygen. Call a physician. **Avoid contact with skin and clothing!** **IN CASE OF SKIN CONTACT:** Immediately flush skin with plenty of water for 15 minutes while removing contaminated clothing and shoes. Get medical attention if irritation persists. Wash contaminated clothing and shoes promptly. Discard contaminated shoes in a fireproof container.

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**PRECAUTIONARY STATEMENTS**  
 Hazard to humans and domestic animals.  
**DANGER! OXIDIZER, CORROSIVE**  
 Liquid hydrogen peroxide causes eye damage, skin irritation, and is harmful if swallowed, inhaled or absorbed through the skin. Direct eye contact may cause irreversible tissue damage and blindness. Wear goggles or face shield and vinyl or neoprene gloves when handling open container. Vapors cause irritation to the eyes, nose, throat, and lungs.  
**CHEMICAL AND PHYSICAL HAZARDS**  
 Liquid hydrogen peroxide is a strong oxidant and poses a FIRE, EXPLOSION OR CONTAINER RUPTURE HAZARD. Avoid excessive heat, contamination, or contact with combustible materials. Clothing, shoes, or other combustible materials that have come into contact with hydrogen peroxide must be immediately and thoroughly washed with water. If allowed to dry in the materials, a fire may result. Discard shoes in a fireproof container.  
**IN CASE OF FIRE** use water only  
**WHEN HANDLING** an open container, wear a NIOSH/MSHA approved respirator as necessary, chemical splash goggles or full face shield, vinyl or neoprene gloves and other protective clothing and boots.  
**CONTAIN SPILLS** and dilute with 20 parts of water. After diluting the spill, sodium metabisulfite or sodium sulfite (1.9 lbs. SO<sub>2</sub> equivalent per 500 ml of peroxide) may be used to destroy the peroxide.

**DIRECTIONS FOR USE**  
 It is a violation of federal law to use this product in a manner inconsistent with its labeling.  
 This product is intended for use as a non-medical/non-health care sterilant (i.e., for industrial use only). This product is for use in AMSCO VHP® industrial equipment only, and only by trained personnel. Use this product for sterilization in accordance with procedures defined in the Equipment Manual.

**STORAGE AND DISPOSAL**  
 Store cartridges at or below 77°F or 25°C in a dry location. Do not expose to cyanide, hexavalent chromium compounds, other oxidizers, reducers, combustible material, or flammable vapors.

**PESTICIDE DISPOSAL**  
 Dilute contents of cartridge with 20 parts water and then empty into sink with running water. Hydrogen Peroxide is classified as DOT oxidizer and a hazardous waste under U.S. EPA hazardous waste regulations and it is a violation of federal law to improperly dispose of pesticides.

**CONTAINER DISPOSAL**  
 Rinse cartridge with running water and dispose of with normal nonhazardous waste.

LOT NUMBER: Not for sale or use after

EPA REG NO 5749-B  
 EPA EST. NO 58779-OH-001  
 AMSCO 5960 Heasley Road • Mentor, OH 44060 • 800-548-4873

AMSCO HYDROGEN PEROXIDE STERILANT		Date	11/96
Part No.	400077-578	Art No.	XAC 054
		Rev.	●
Art Farm Job No.	3.1.95.CMF 400077-578		
Size	11.625" X 3.4375"		

5/5

pm31

5749-8

1/5

UNITED STATES ENVIRONMENTAL PROTECTION AGENCY

DEC 18 1996

Dawn Cole  
Steris Corporation  
5960 Heisley Road  
Mentor, OH 44060

SUBJECT: Vaprox Hydrogen Peroxide  
EPA Registration No. 5749-8 *Transferred to 58779-4*  
Notification Per PR-Notice 95-2.

Dear Ms. Cole:

This will acknowledge receipt of your notification, for a change of manufacture site, EPA Establishment Number and address for The AMSCO, Canada Inc. submitted with regards to the provisions of FIFRA section 3 (c) 9. Based on a review of the submitted material, the following comment applies.

The application is acceptable and the notification has been made a part of the records for this file.

Sincerely,

Wanda Y. Mitchell  
Notification Coordinator  
Antimicrobial Program Branch  
Registration Division (7505)

SIG 1-17-97

CONCURRENCES							
SYMBOL							
SURNAME							
DATE							

November 15, 1996

Marion Johnson  
Product Manager (31)  
Antimicrobial Program Branch  
Registration Division  
U.S. EPA  
Washington, DC 20406

Dear Mrs. Johnson:

This letter of notification is in compliance with the standards outlined in 40 CFR §152.46(a) and 152.46(b). This letter is to notify EPA of the following changes to the labels for Vaprox Hydrogen Peroxide, EPA Reg. No. 5749-8: *Transferred to 58779-4*

1. The previous manufacture site of Apex, North Carolina is being changed to the current manufacture site at 5960 Heisley Road, Mentor, Ohio 44060, 800-548-4873;
2. The previous EPA Establishment Number 5749-NC-1 is being changed to the current EPA Establishment Number 58779-OH-001; and
3. The AMSCO, Canada Inc. address, previously located in Woodbridge, Ontario, is being changed to reflect the current location in Etobicoke, Ontario.

Attached, please find the following modified labels:

1. Five copies of the Vaprox product label; and
2. Five copies of the Vaprox case label (shipping container label).

Please note that the above changes relate specifically to the label which EPA approved on March 16, 1995.

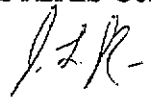
STERIS Corporation is aware that AMSCO submitted a request for disposal instruction amendments of the Vaprox label on June 17, 1995. Given the long history of correspondence regarding amendment of the Vaprox label, STERIS Corporation would like to arrange a conference call in which STERIS Corporation can fully discuss and resolve all outstanding issues. STERIS Corporation will call you within 10 business days from the date of this letter to confirm a convenient date and time for the conference call.

You may wish to note that effective May 13, 1996 STERIS Corporation acquired AMSCO International. AMSCO continues to exist as a wholly owned subsidiary of STERIS Corporation.


If you have any questions or require further information, please do not hesitate to contact me at 216/354-2600, extension 21231 or Lori Haller at 814/870-8559. Please direct future correspondence regarding this product to my attention at the address below.

Sincerely,

STERIS Corporation

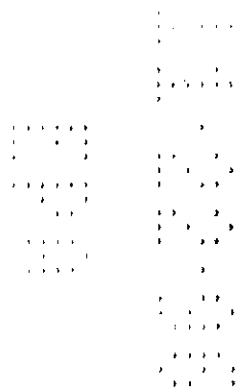Dawn Cole  
Environmental Compliance Specialist

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VAPROX™  
HYDROGEN PEROXIDE  
STERILANT

UN2014

**STATEMENT OF PRACTICAL TREATMENT**  
Do not get in eyes! **IN CASE OF EYE CONTACT:** Immediately flush with plenty of water for at least 15 minutes, lifting upper and lower lids intermittently. Call a physician.  
Do not take internally! **IF SWALLOWED:** Drink plenty of water immediately to dilute. Do not induce vomiting. Never give anything by mouth to an unconscious person. Call a physician.  
Avoid breathing vapors! **USE ONLY WITH ADEQUATE VENTILATION IF INHALED:** Remove to fresh air. If not breathing, give artificial respiration. If breathing is difficult, give oxygen. Call a physician.

Avoid contact with skin and clothing!  
**IN CASE OF SKIN CONTACT:** Immediately flush skin with plenty of water for 15 minutes while removing contaminated clothing and shoes. Get medical attention if irritation persists. Wash contaminated clothing and shoes promptly. Discard contaminated shoes in a fireproof container.

**PRECAUTIONARY STATEMENTS**  
Hazard to humans and domestic animals.  
**DANGER! OXIDIZER, CORROSIVE**  
Liquid hydrogen peroxide causes eye damage, skin irritation, and is harmful if swallowed, inhaled or absorbed through the skin. Direct eye contact may cause irreversible tissue damage and blindness. Wear goggles or face shield and vinyl or neoprene gloves when handling open container. Vapors cause irritation to the eyes, nose, throat, and lungs.

**CHEMICAL AND PHYSICAL HAZARDS**  
Liquid hydrogen peroxide is a strong oxidant and poses a FIRE, EXPLOSION OR CONTAINER RUPTURE HAZARD. Avoid excessive heat, contamination, or contact with combustible materials. Clothing, shoes, or other combustible materials that have come into contact with hydrogen peroxide must be immediately and thoroughly washed with water. If allowed to dry in the materials, a fire may result. Discard shoes in a fireproof container.

SEE EQUIPMENT MANUAL AND MATERIAL SAFETY DATA SHEET FOR ADDITIONAL INFORMATION.

AMSCO CANADA, INC. Etobicoke, Ontario

ACTIVE INGREDIENT: 35% Hydrogen Peroxide  
INERT INGREDIENT: 65% Water

**DANGER**  
KEEP OUT OF REACH OF  
CHILDREN.  
OXIDIZER; CORROSIVE  
STORAGE

Store cartridges at or below 77°F or 25°C in a dry location.

EPA REG. NO. 5749-8 Transferred to 58779-4  
EPA EST. NO. 58779-OH-001

AMSCO, 5960 Heisley Road • Mentor, OH 44060  
800-548-4873 400077-579 11/96  
(FOR AMSCO USE ONLY: 400021-145)

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**DANGER** KEEP OUT OF REACH OF CHILDREN. OXIDIZER. CORROSIVE. SEE BACK PANEL FOR STATEMENT OF PRACTICAL TREATMENT. FOR USE IN AMSCO VHP® INDUSTRIAL EQUIPMENT ONLY.  
**DIN 02047616**  
 AMSCO CANADA, INC. Etobicoke, Ontario

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**VAPROX™ UN2014**  
**HYDROGEN PEROXIDE STERILANT**

**STATEMENT OF PRACTICAL TREATMENT**  
**Do not get in eyes! IN CASE OF EYE CONTACT:** Immediately flush with plenty of water for at least 15 minutes, lifting upper and lower lids intermittently. Call a physician. **Do not take internally! IF SWALLOWED:** Drink plenty of water immediately to dilute. Do not induce vomiting. Never give anything by mouth to an unconscious person. Call a physician. **Avoid breathing vapors! USE ONLY WITH ADEQUATE VENTILATION! IF INHALED:** Remove to fresh air. If not breathing, give artificial respiration. If breathing is difficult, give oxygen. Call a physician. **Avoid contact with skin and clothing!** **IN CASE OF SKIN CONTACT:** Immediately flush skin with plenty of water for 15 minutes while removing contaminated clothing and shoes. Get medical attention if irritation persists. Wash contaminated clothing and shoes promptly. Discard contaminated shoes in a fireproof container.

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