

Immediate container for Disinfectant
Actual Pouch (plastic)
with red print over milk
white background with
transparent borders

received 8-27-79 BCL

TEAR ALONG DOTTED LINE
AND POUR CONTENTS INTO
THERMOMETER HOLDER

CP Clinical Products®

GERMICIDE TINCTURE
(Brand of refined Benzalkonium Chloride)
TINCTURE 1:750

CAUTION: Keep out of reach of children.
May be harmful if swallowed.

ACTIVE INGREDIENTS:

Alkyl (67% C₁₂, 25% C₁₄, 7% C₁₆,
1% C₈, C₁₀, C₁₈) dimethyl benzyl
ammonium chloride 0.13%
Isopropanol 40%

INERT INGREDIENTS:

Water 59.87%

E.P.A. REGISTRATION NO. 5795-1

E.P.A. Est. NO. 09480

2/3 FL. OZ. Made in U.S.A.

141466
THERMOMETER HOLDER
AND POUR CONTENTS INTO
THERMOMETER HOLDER

For use in thermometer holders. Immerse pre-cleaned Thermometer
in undiluted solution in holder for 10 minutes to disinfect, then
store thermometer in holder at bedside until needed. Change so-
lution in holder every 7 days or with each new patient.

Do not use, pour, spill or store near heat or open flame.
Discard used package in a safe place.

DISTRIBUTED BY
Cheesebrough Ponds Inc. HOSPITAL PRODUCTS DIVISION
GREENWICH, CONNECTICUT 06030

REORDER
S-6049

Immediate container for Disinfectant
Actual Pouch (plastic)
with red print over milk
white background with
transparent borders

received 8-27-79 BCL

TEAR ALONG DOTTED LINE
AND POUR CONTENTS INTO
THERMOMETER HOLDER

CP Clinical Products

GERMICIDE TINCTURE
(Brand of refined Benzalkonium Chloride)
TINCTURE 1:750

CAUTION: Keep out of reach of children.
May be harmful if swallowed.

ACTIVE INGREDIENTS:
Alkyl (67% C₁₂, 25% C₁₄, 7% C₁₆,
1% C₈, C₁₀, C₁₈) dimethyl benzyl
ammonium chloride 0.13%
Isopropanol 40%

INERT INGREDIENTS:
Water 59.87%
E.P.A. REGISTRATION NO. 5795-1
E.P.A. Est. NO. 09480
2/3 FL. OZ. Made in U.S.A.

141466
141466
141466

For use in thermometer holders. Immerse precleaned Thermometer
in undiluted solution in holder for 10 minutes to disinfect, then
store thermometer in holder at bedside until needed. Change so-
lution in holder every 7 days or with each new patient.

Do not use, pour, spill or store near heat or open flame.
Discard used package in a safe place.


DISTRIBUTED BY
Chesebrough Ponds Inc. HOSPITAL PRODUCTS DIVISION
GREENWICH, CONNECTICUT 06030

REORDER
5-6049

Labeling for one of the kits

Visible under through
cellophane wrapping

CP Clinical Products	
UNIVERSAL THERMOMETER KIT Contains — Oral tri-top thermometer by Faichney , color coded thermometer holder with Velcro tape — take home thermometer holder — Identification tag — one 2.5 fl oz packet of tincture of Benzalkonium Chloride Caution: Keep out of reach of children (see back panel for additional precautions and directions for use). Distributed by: Chesebrough-Pond's Inc. Hospital Products Div. • Greenwich, Conn. 06830	GERMICIDE TINCTURE Brand of Refined Benzalkonium Chloride Tincture 1:750 Active Ingredients Alkyl (67% C ₁₂ , 25% C ₁₄ , 7% C ₁₆) 1% C ₁₂ , C ₁₄ , C ₁₆ dimethyl benzyl ammonium chloride 0.13% Isopropanol 40% Inert Ingredients Water 59.87% EPA Reg. No. 5795-1, EPA Est. No. 09480

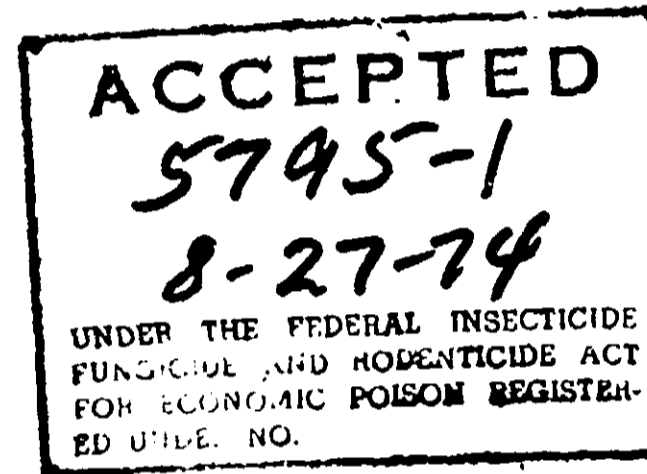

located on underside of
foam package

DIRECTIONS FOR USE:

1. OPEN PACKET
2. PLACE HOLDER IN BASE; OR FOR HOLDERS WITH PRE-ATTACHED VELCRO MOUNTING, ATTACH ADHESIVELY BACKED VELCRO TAPE TO WOOD, METAL OR SMOOTH WALL SURFACES BY REMOVING PAPER BACKING AND PRESSING UNIT FIRMLY AGAINST SURFACE.
3. EMPTY CONTENTS OF PACKET INTO HOLDER.
4. PLACE PRECLEANED THERMOMETER, BULB DOWN, INTO UNDILUTED SOLUTION IN HOLDER FOR 10 MINUTES TO DISINFECT. THEN STORE THERMOMETER IN HOLDER AT BEDSIDE UNTIL NEEDED.
5. WIPE THERMOMETER CLEAN BEFORE REPLACING IN HOLDER.
6. CHANGE SOLUTION WHEN IT BECOMES VISIBLY CONTAMINATED OR EVERY SEVEN DAYS OR WITH EACH NEW PATIENT.

DO NOT USE, POUR, SPILL OR STORE SOLUTION NEAR HEAT OR OPEN FLAME.
DISCARD USED PACKAGE IN A SAFE PLACE.
CAUTION: KEEP OUT OF THE REACH OF CHILDREN.
SOLUTION MAY BE HARMFUL IF SWALLOWED.

