

SANI-ORTHO-PHENE

(1:256 Dilution)

TUBERCULOCIDAL FUNGICIDAL ANTI-BACTERIAL DETERGENT FOR ONE STEP CLEANING

A concentrated bactericidal formulation combined with anionic detergents for compatible one-step cleaning of floors, walls and all hospital surfaces and equipment.

EPA Reg. NO. 6142-11

EPA Est., 6142-IL-1

DANGER - "KEEP OUT OF REACH OF CHILDREN"

SEE RIGHT PANEL FOR FIRST AID STATEMENT & OTHER
REQUIRED STATEMENTS

ACTIVE INGREDIENTS: Isopropanol 10%, Orthophenylphenol 8%, 4-Chloro-2-Cyclopentylphenol 4%, Para-tert-amyphenol 4%, Tetrasodium ethylene diamine tetraacetate 3%
TOTAL 29%

INERT INGREDIENTS TOTAL 71%



161 Avenue of the Americas, New York, N.Y. 10013
11 East Illinois Street, Chicago, Illinois 60611

PRODUCT DESCRIPTION

SANI ORTHO PHENE is a highly concentrated germicide which when used in the recommended use dilution kills many pathogenic bacteria and pathogenic fungi (Trichophyton interdigitale) including Staphylococcus aureus, Salmonella choleraesuis, Pseudomonas aeruginosa and Mycobacterium tuberculosis. It also cleans and deodorizes all in one step and is recommended for use in hospitals, nursing homes, and other institutions.

DIRECTIONS FOR USE

A. O. A. C. Use Dilution Confirmation Tests

Kills Mycobacterium Tuberculosis 1:256 Kills Pseudomonas aeruginosa 1:256
Kills Staphylococcus aureus 1:256 Kills Salmonella choleraesuis 1:256
Add 1/2 oz SANI ORTHO PHENE to each gallon of tap water at the desired temperature for use as wash solution. This produces a uniform mixture ready for immediate use as a combined detergent, disinfectant, and bactericide. All surfaces should be thoroughly moistened. As the applied SANI ORTHO PHENE solution dries, it leaves a microscopic bacteriostatic film of active ingredients that continues to inhibit the growth of bacteria until removed by rubbing or scrubbing. Since bacteria are not killed instantly, the longer the time of contact of the bacteria with SANI ORTHO PHENE before it dries, the better the results. For heavy duty cleaning, increase SANI ORTHO-PHENE usage to 1 oz per gallon. Rinse empty containers with water before discarding.

SANI ORTHO-PHENE can be used on all surfaces, such as floors, walls, cabinets, fixtures in Hospital Operating Rooms, Contagious-Disease areas and Washrooms.

**DANGER HARMFUL IF SWALLOWED AVOID CONTAMINATION OF
FOOD PROTECT EYES AND SKIN WHEN HANDLING MUST NOT GET
IN EYES OR ON SKIN AS IT MAY CAUSE SEVERE SKIN AND EYE
DAMAGE**

First Aid: In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a Physician. If swallowed, do not induce vomiting. Drink large quantities of fluid and call a Physician immediately.

NET CONTENTS 1/2 Gallon

ACCEPTED

SEP 2 1976

UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND ROBOTICIDE ACT
FOR LEAD AND POISON REGISTER
FD-300 (Rev. 10-6-64) 6142-11