Reg. No. 7078-1 (PM 33)

Cameraman's Note

Accepted label is printed
in reverse. Another label is
being shot here as an assistance
to the user of these fiches.
It follows the accepted label.

Directions for Use

Directions for activating

DISI

C

NOTE DATES HERE
MIXING DATE

EXPIRATION DATE

See sig

Arbrock inc.
Arbrock Inc. 1000 2nd St.

Art

Disinfecting and sterilizing solution

Cidex

activated dialdehyde

See side of Label for Precautions

Arbrook*

UN
MAD
PAT

A

solution

Dr. ...

Keep in a cool place!

Arbrook.

DIRECTIONS FOR USE

For complete disinfection or sterilization of medical instruments and appliances, thoroughly clean, rinse and rough-dry objects before immersing in FULL STRENGTH CIDEX solution. Flush and fill cleansed lumen of any hollow instrument.

DISINFECTION: Immerse completely for a minimum of **10 MINUTES** to destroy viruses and vegetative pathogens including **M. tuberculosis** and **Pseudomonas aeruginosa**.

STERILIZATION: Immerse completely for a minimum of **10 HOURS** to destroy resistant pathogenic spores including **Cl. sporogenes** and **Cl. tetani**.

Remove instruments from CIDEX solution using sterile technique and **RINSE THOROUGHLY** with sterile water.

DIRECTIONS FOR ACTIVATING

Add contents of attached container to this container of CIDEX fluid. Shake well. Solution is ready to use.

When activated, solution changes color to green.

DO NOT USE THIS SOLUTION BEYOND 14 DAYS AFTER MIXING. NOTE DATES HERE

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Arbrook Inc.