

15 OCT 1984

Surqikos, Inc.
P.O. Box 130
Arlington, TX 76010

Attn: Mr. W.J. McOuade

Subject: Cidex Aqueous Activated Dialdehyde
Solution
EPA Reg. No. 7078-1
Submission dated September 6, 1984

The labeling referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide and Rodenticide Act, as amended, is acceptable, and a stamped copy is enclosed for your records.


Based on the recommendation by the Co-General Referees, Disinfectants, for adoption as a procedural modification for the AOAC Sporidical Test, the revised procedure used to prepare the suture loops in the previously submitted AOAC Sporidical Test data will be considered acceptable.

Therefore, the submitted sporidical data are considered acceptable to support effectiveness of the product as a sterilizer in 10 hours at 25C after 14 days of simulated reuse in a manual bucket system.

The additional submitted data/information, as requested in the previous review, are acceptable.

Since the data to support the sterilizing and sporicidal claims for the product after 14 days of simulated reuse challenge were developed at 25C, rather than 20C as specified in the AOAC Sporicidal Test, the directions for use should be qualified accordingly, i.e., "Immerse equipment completely for a minimum of 10 hours at 25C ..." or "Immerse equipment completely for a minimum of 10 hours at room temperature (25C) ...".

Sincerely yours,

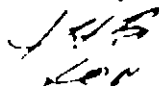

John H. Lee
Product Manager 31
Disinfectants Branch
Registration Division (TS-767C)

Enclosure

Front Panel

REUSABLE

Reorder Number

Sterilizing and Disinfecting Solution

Rapid-Acting/Non-Staining/Non-Rusting

Sporicidal Tuberculocidal

Virucidal† Bactericidal

Fungicidal Pseudomonacidal

Active Ingredients

Glutaraldehyde ... 2.0%
Inert Ingredients... 98.0%
Total.....100.0%

CIDEX® Aqueous Activated Dialdehyde Solution

Keep out of reach of children.

Note - Attached vial contents must be added to solution before this product is effective. See Directions for Activating.

DANGER: See right side panel for precautions

† See Directions for Use

EPA Reg. No. 7078-1
EPA Est. No. 36126-PR-1

SURGIKOS

*Trademark

ACCEPTED
with COMMENTS
in EPA Letter Dated:

OCT 15 1984

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.

7078-1

