

REUSABLE

Sterilizing and Disinfecting Solution

Rapid-acting/Non-staining/Non-rusting

Bactericidal
Fungicidal
Virucidal

Tuberculocidal
Bactericidal
Pseudomonacidal

Cidex 7^{formula}

Long-life activated dialdehyde solution

Active ingredient
Sodium Nitrite 2.0%
Sodium Hypochlorite 2.0%
Total 4.0%

Note — Contents of attached vial must be added to fluid before this product is effective. See directions for activating.

Contains Sodium Nitrite as a corrosion inhibitor.

Keep out of reach of children

DANGER: See right side panel for precautions.

See Directions for Use

SURGIKOS

FORMARA

BEST AVAILABLE COPY

PRECAUTIONARY STATEMENTS

DANGER:

HAZARD TO HUMANS...
FOLLOW ALL LABEL DIRECTIONS

Eyes — Contact with eyes causes damage. In case of contact, flush with water immediately and get medical attention.
Skin — Avoid skin contact, as possibility of sensitization exists. May cause skin irritation. For skin contact flush thoroughly with water.

Ingestion — Avoid contamination of food. Harmful if swallowed.

Inhalation — Use in ventilated area.

STORAGE AND DISPOSAL:

Keep in a cool place. Triple rinse empty container with water and dispose in an incinerator or landfill approved for pesticide containers or bury in a safe place.

Especially recommended for sterilization or disinfection of...

Anesthesia equipment	Rubber objects
Respiratory therapy equipment	Plastic objects
Metallic equipment or instruments	Thermometers

CONTENTS ONE GAL (128 FLUID OZ.)
MADE IN U.S.A.
CONTROL NUMBER ON BOTTOM
2750011

REGISTERED

JAN 23 1995


U.S. PATENT & TRADEMARK OFFICE

Act.

Under

Reg. No. 7078-4

SURGIKOS

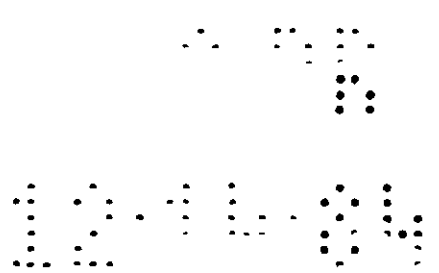

For complete sterilization or disinfection of medical instruments and equipment, *thoroughly clean, rinse and rough dry* objects before immersing in CIDEX FORMULA 7* Long-Life Activated Dialdehyde Solution. Cleanse and rinse the lumens of hollow instruments before filling with CIDEX FORMULA 7 Solution.

Reorder No.
2750

DIRECTIONS FOR ACTIVATING:

Add contents of attached vial to this container of CIDEX FORMULA 7 Solution. Shake, solution is ready to use. *When activated, solution changes color to green.* This solution may be used and reused for up to 28 days after activation. Do not use activated solution beyond 28 days. Efficacy of this product during its use-life can be verified by the CIDEX FORMULA 7 Solution Effectiveness Test Kit.


DIRECTIONS FOR USE:

General Classification — It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

For repeated use in manual (bucket and tray) systems.

Disinfection: Immerse completely for a minimum of 10 minutes to destroy vegetative organisms including *Pseudomonas aeruginosa*, pathogenic fungi, and viruses (Poliovirus Type 1; Adenovirus Type 2; Herpes simplex Type 1, 2; Influenza Type A [WSN/33]; Vaccinia; Coronavirus; Cytomegalovirus; Rhinovirus Type 14; Coxsackievirus B-1) on inanimate surfaces. For *Mycobacterium tuberculosis*, CIDEX FORMULA 7 Solution destroys 98.4% in 10 minutes and 100% in 90 minutes at 25°C on inanimate surfaces. Remove equipment from CIDEX FORMULA 7 Solution and rinse thoroughly.

Sterilization: Immerse equipment completely for a minimum of 10 hours to destroy resistant pathogenic spores as represented by *Clostridium sporogenes* and *Bacillus subtilis*. Remove equipment from CIDEX FORMULA 7 Solution using sterile technique and rinse thoroughly with sterile water.

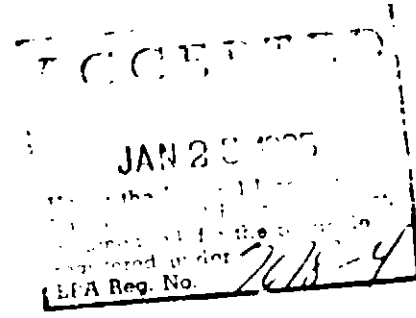

E.P.A. Reg. No. 7078-4
E.P.A. Est. No. 36126-PR-1

NOTE DATES HERE

MIXING DATE _____ EXPIRATION DATE _____



+H22227501-

DISTRIBUTED BY

SURGIKOS

a Johnson & Johnson company

SURGIKOS, INC., ARLINGTON, TEXAS 76010
*TRADE MARK ©SURGIKOS 1975

BEST AVAILABLE COPY

12104

Cidex⁷
long-life activated dialdehyde solution

Activator Vial

For Use With One Gallon
CIDEX FORMULA 7 Solution Only

*TRADEMARK

DANGER: *Contact with eyes causes damage.* In case of contact with eyes, flush with water immediately and get medical attention. For skin contact, flush thoroughly with water.

DISPOSAL: Rinse empty container thoroughly with water prior to discarding.

NET CONTENTS 153 GRAMS
MADE IN U.S.A.
CONTROL NUMBER ON BOTTOM

DISTRIBUTED BY:

SURGIKOS

a Johnson & Johnson company

SURGIKOS, INC., ARLINGTON, TEXAS 76010
2750 V: 1 © SURGIKOS 1976

JAN 27 1975
EPA Reg. No. 7628-4

CIDEX 7 2750 ONE GAL