

# Sterilizing and Disinfecting Solution

Rapid-acting / Non-staining / Non-rusting

Sporicidal

Tuberculocidal

Virucidal

Bactericidal

Fungicidal

Pseudomonacidal

# Cidex 3X

CONCENTRATE \*

long-life activated dialdehyde

**CC-850**

Cidex

Arbrooks

# Sterilizing and Disinfecting Solution

Rapid-acting Non-staining Non-rusting

Sporicidal

Tuberculocidal

Virucidal

Bactericidal

Fungicidal

Pseudomonacidal

# Cidex 3X

CONCENTRATE \*

## long-life activated dialdehyde

ACCEPTED

SEP 26 1975

UNITED STATES FEDERAL INSPECTOR  
GENERAL'S OFFICE ACCIDENTICIDE  
FOR CHEMICAL POISON RECORDS  
EDISON RD. NO. 7078-6

Comment

**Directions:** AFTER dilution of CIDEX 3X Concentrate, slowly add contents of this vial to 3 gallons of CIDEX Fluid. Stir well. Activated solution will be green.

**DANGER: Contact with eyes causes damage.** In case of contact with eyes, flush with water immediately and get medical attention. For skin contact, flush thoroughly with water.

\*TRADEMARK © ARBROOK 1975 PATENT #3016328

# Cidex3X

long-life activated dialdehyde

## Activator Vial

For Use With CIDEX Solution

### Arbrook inc.

Arbrook Inc. 1000 E. 10th St. St. Paul, MN 55101

Product

## CC-850

Active Ingredient

Glutaraldehyde .. 6.0%

Inert Ingredients .. 94.0%

Total .. 100.0%

Contains Sodium Nitrite as a corrosion inhibitor.

**Note** — Attached vial contents *must* be added to solution before this product is effective. See directions.

Rinse empty container thoroughly with water prior to discarding.

**DANGER:** See Back Label for Precautions

\*TRADEMARK

†See Directions for Use

# Arbrook\*

## Directions for Dilution and Activation:

Add contents of this container of CIDEX 3X Concentrate to 2 gallons of distilled or deionized water. Then slowly add contents of attached vial. Stir well. Solution is ready to use when salts are completely dissolved.

## Directions for Use:

For complete sterilization or disinfection of medical instruments and appliances, *thoroughly clean, rinse and rough-dry* objects before immersing in CIDEX Solution. Cleanse and rinse the lumens of hollow instruments before filling with CIDEX Solution.

**Disinfection and Terminal Decontamination:** Immerse completely for a minimum of **10 minutes** to destroy vegetative pathogens including **Mycobacterium tuberculosis, Pseudomonas aeruginosa** and **viruses (Poliovirus Type 1, 2, 3; Influenza Type A<sub>1</sub>, A<sub>2</sub> (Hong Kong), B; Vaccinia; Coxsackievirus B-1; Reovirus Type 3; Herpes simplex)** on inanimate surfaces

**Sterilization:** Immerse completely for a minimum of **10 hours** to destroy resistant pathogenic spores including **Clostridium sporogenes** and **Clostridium tetani**. Remove instruments from CIDEX Solution using sterile technique and *rinse thoroughly* with sterile water

*When activated, solution changes color to green.*

Do not use activated stock solution beyond 28 days after mixing.

NOTE DATES HERE  
MIXING DATE

E. P. A. Reg. No. 7078-6

E. P. A. Est. No. 7078-TX-1

**ACCEPTED**

EXPIRATION DATE

SEP 26 1975

UNDER THE FEDERAL INSECTICIDE  
FUNGICIDE AND RODENTICIDE ACT  
EPA REGISTRATION NUMBER  
7078-6

© Arbrook, Inc. 1975

# DANGER:

FOLLOW DIRECTIONS FOR USE

**HARMFUL IF SWALLOWED**

**Skin** — **Avoid skin contact**, as possibility of sensitization exists. May cause skin irritation. For skin contact flush thoroughly with water.

**Eyes** — **Contact with eyes causes damage. In case of contact** flush with water immediately and get medical attention.

**Food** — Avoid contamination of food.

**Inhalation** — Use in ventilated area.

**Instruments** — CIDEX Solution will not prevent or remove oxidation caused by poor or improper plating, cleaning, or care of instruments.

## KEEP IN A COOL PLACE

Especially recommended for terminal decontamination procedures as well as sterilization or disinfection of . . .

Lensed instruments

Anesthesia equipment

Respiratory therapy equipment

Rubber objects

Plastic objects

Sharp instruments

Thermometers

CONTENT ONE GAL (128 FLUID OZ.)

MADE IN U.S.A.

PAT. 3016328

CONTROL NUMBER ON BOTTOM

**Arbrook inc.**

Arlington, Texas 76010

ACCEPTED WITH COMMENTS