

CONCENTRATE PRODUCT

IN GAL PUT-UP

CENTER PANEL

Sterilizing and Disinfecting Solution

Rapid-acting/nonstaining/nonrusting
Virucidal Tuberculocidal
Fungicidal Bactericidal
Sporicidal Pseudomonicidal

ESPECIALLY RECOMMENDED FOR REPEATED USE IN BUCKET AND TRAY SYSTEMS.

CIDEX* II concentrate

Activated Dialdehyde

Active Ingredient	
Glutaraldehyde---	12.0%
Inert Ingredients-----	88.0%
TOTAL	100.0%

Contains sodium nitrite
as a corrosion inhibitor.

NOTE: Activator must be added to diluted
fluid before this product is effective.
See directions.

KEEP OUT OF REACH OF CHILDREN

DANGER: See right Panel for precautionary statements.

*See Directions for use.

ACCEPTED
with COMMENTS
in EPA Letter Dated

MAR 22 1983

Under the Federal Insecticide,
Fungicide and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.

*Trademark of SURGIKOS, Inc.

7078-12

RECEIVED

FEB 27 1983

Registration Division (TS-707)

CONCENTRATE PRODUCT
IN GAL PUT-UP

LEFT PANEL

GENERAL CLASSIFICATION- It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Reorder No.

DIRECTIONS FOR ACTIVATION:

1. Prior to activation, dilute contents of one-gallon container of CIDEX* II Fluid Concentrate with four (4) gallons of water.
2. After diluting, add contents of one 22-ounce container of Activator to solution.
3. Mix; solution is ready to use. When activated and diluted CIDEX II Solution changes color to yellow.

DIRECTIONS FOR USE:

For complete sterilization or disinfection of medical instruments and equipment, thoroughly clean, rinse and rough-dry objects before immersing in CIDEX II Solution. Cleanse and rinse lumens of hollow instruments before filling with CIDEX II Solution.

DISINFECTION AND TERMINAL DECONTAMINATION: Immerse completely for a minimum of 10 minutes to destroy vegetative pathogens including Mycobacterium tuberculosis, Pseudomonas aeruginosa and Viruses [Poliovirus Type 1; Influenza Type A₂ (Hong Kong); Herpes Simplex Type 1] on inanimate surfaces.

STERILIZATION: Immerse completely for a minimum of 10 hours to destroy resistant pathogenic spores including Clostridium sporogenes and Bacillus subtilis. Remove instruments from CIDEX II Solution using sterile technique and rinse thoroughly with sterile water.

Solution may be repeatedly used for not more than 14 days.

ACCEPTED
with COMMENTS
in EPA Letter dated:

MAR 22 1963

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.

7078-12

EPA Reg. No. _____
EPA Est. No. _____

*Trademark SURGIKOS 1982

RECEIVED

FEB 23 1963

Registration Division (15-707)

RIGHT PANEL

Especially recommended for disinfection of...
Anesthesia equipment Rubber objects
Respiratory therapy equipment Plastic objects
Metallic equipment or instruments Glass objects

STORAGE AND DISPOSAL

Keep in a cool place. Triple rinse empty container with water and dispose in an incinerator or landfill approved for pesticide containers or bury in a safe place.

Content 128 fl. oz. (3.785 liters)
Made in USA
Control number on bottom

PRECAUTIONARY STATEMENTS

DANGER:

HAZARD TO HUMANS
FOLLOW ALL LABEL DIRECTIONS

Prevent contact with skin or eyes by using eye protection and rubber gloves when handling or pouring.

STATEMENT OF PRACTICAL TREATMENT

EYES-- Corrosive, causes eye damage. Do not get in eyes. In case of contact, flush with water immediately and get medical attention.

SKIN-- Avoid skin contact, as possibility of skin sensitization exists. Can cause skin irritation. For skin contact, flush thoroughly with water. Get medical attention if irritation occurs.

INGESTION-- Harmful if swallowed. Avoid contamination of food.

INHALATION-- Avoid spillage, use in a well-ventilated area.

ACCEPTED
with COMMENTS
in EPA Letter 10/82

MAR 22 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, and the pesticide
registration number EPA Reg. No.

7078-12

RECEIVED

FEB 23 1983

Registration Division (TS-767)

ACTIVATOR LABEL

CONCENTRATE
Cidex II
activated dialdehyde solution
ACTIVATOR
For use with
CIDEX® II Fluid only

WARNING: Contact with eyes causes damage. In case of contact with eyes, flush with water immediately and get medical attention. For skin contact, flush thoroughly with water.
Rinse empty container thoroughly with water prior to recharging.

TRADEMARK
SURGIKOS INC.
SPRINGFIELD

DISTRIBUTED BY
SURGIKOS
SURGIKOS INC. ARLINGTON TEXAS 76010

APPROVED
with conditions
in 1978
MAR 23 1983
EPA Reg. No. 7078-12

RECEIVED

FEB 23 1983

Registration Division (TS-767)

U.S. ENVIRONMENTAL PROTECTION AGENCY OFFICE OF PESTICIDES PROGRAMS REGISTRATION DIVISION (HH-567) WASHINGTON, D.C. 20460	EPA REGISTRATION NO.	DATE OF ISSUANCE
	TERM OF ISSUANCE	22 MAR 1983
NOTICE OF PESTICIDE: REGISTRATION RENEWAL <i>(Under the Federal Insecticide, Fungicide and Rodenticide Act as amended)</i>	NAME OF PESTICIDE PRODUCT	
	II A	
NAME AND ADDRESS OF REGISTRANT (Include ZIP code)		
<p>NOTE: Changes in labeling formula differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Registration Division prior to use of the label in commerce. In any correspondence on this product always refer to the above U.S. EPA registration number.</p> <p>On the basis of information furnished by the registrant, the above named pesticide is hereby Registered/Reregistered under the Federal Insecticide, Fungicide, and Rodenticide Act.</p> <p>A copy of the labeling accepted in connection with this Registration/Reregistration is returned herewith.</p> <p>Registration is in no way to be construed as an indorsement or approval of this product by this Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.</p>		
<input type="checkbox"/> ATTACHMENT IS APPLICABLE		
SIGNATURE OF APPROVING OFFICIAL		DATE