




INSTRUCTION INSERT TO BE INCLUDED IN EACH CASE.

# Cidex\* II

activated dialdehyde solution concentrate

## DIRECTIONS FOR ACTIVATION:

1. Prior to activation, dilute contents of one-quart container of CIDEX\* II Fluid Concentrate with three (3) full quarts of water.
2. After diluting CIDEX II Fluid Concentrate, add contents of one 4-ounce container of Activator to solution.
3. Mix; solution is ready to use. When activated and diluted CIDEX II Solution changed color to yellow.

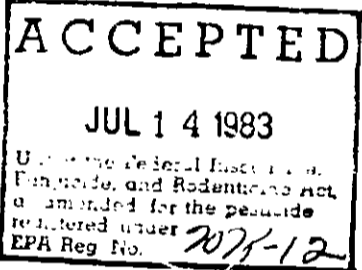

## SURGIKOS

\*TRADEMARK SURGIKOS, INC. 1983 227011-0

SURGIKOS, INC., ARLINGTON, TEXAS 76010

100% ALCOHOL W/OUT WATER

ADHESIVE LABEL TO BE PLACED  
ON POLYETHYLENE HOLDER FOR  
ACTIVATOR VIAL.

**CAUTION:**  
Dilute  
Concentrate  
Before  
Activation

Handwritten text, possibly a date or reference number, oriented diagonally.

**ACCEPTED**  
JUL 14 1983  
Under the Federal Insecticide,  
Fungicide, and Rodenticide Act,  
as amended for the pesticide  
registered under  
EPA Reg. No. 7078-12