

B
A

WEAR PROTECTIVE GLASSES AND RUBBER GLOVES WHEN MIXING AND FOR ALL SUBSEQUENT HANDLING OF THE PRODUCT.

RECOMMENDATIONS FOR USE


When used as directed for **DISINFECTION**, 3M Brand Disinfecting and Sterilizing Solution will kill pathogenic bacteria (including *M. tuberculosis*, *Ps. aeruginosa*, *S. aureus* and *S. choleraesuis*) and pathogenic fungi (*T. interdigitale*).

*This product has also been shown to irreversibly inactivate lipophilic viruses (*Adenovirus Type 2*, *influenza A-2* [England] and *Herpes simplex Type 1*) and hydrophilic virus (*poliovirus Type 1*) on environmental surfaces.

When used as directed for **STERILIZATION**, 3M Brand Disinfecting and Sterilizing Solution will kill the desiccated resistant spores of *Cl. sporogenes* & *B. subtilis*.

PRECLEANING SOAK TREATMENT

This solution does NOT coagulate blood. Items from septic cases should be placed directly into fresh solution until final cleaning and terminal disinfection or sterilization of contaminated objects.

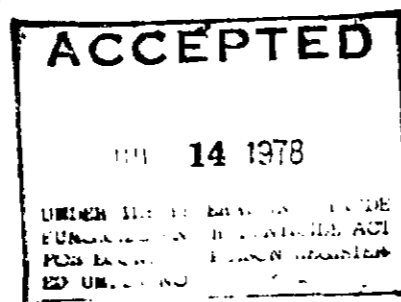

Medical Products Division **3M** COMPANY
3M CENTER • SAINT PAUL, MINNESOTA 55101

DANGER — Causes severe EYE irritation and may cause permanent eye damage. AVOID contact with SKIN and clothing. AVOID contamination of FOOD. MAY BE HARMFUL IF SWALLOWED.

If EYE contact occurs, flush IMMEDIATELY and THOROUGHLY with water. Get medical attention if SKIN contact occurs. Wash thoroughly with water. Wash clothing.

STORAGE AND DISPOSAL

Store this container and any of its unused contents in a cool, safe area reserved for such disinfecting or sterilizing products. This should be away from drugs, medicines or food supplies.


This product must be kept in locked storage sufficient to make it inaccessible to children or persons unfamiliar with its proper use.

Do not re-use empty container.

Dispose container and any unused product by crushing and wrapping container for hospital trash collection.

METAL ITEMS — This solution will not remove rust deposits due to improper plating, nor will it prevent rust if the plating is damaged. Carbon steel items should not be left in the solution longer than the ten-hour sterilization time.

3M Company makes no warranty, expressed or implied, concerning the use of this product other than as indicated on the label. Buyer assumes all risk of use or handling of this material, when such use or handling is contrary to label instructions.

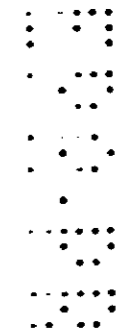

A
B