

CONTAINER LABEL FOR ACTIVE SOLUTION

RECC  
When  
ing at  
M tul  
pathc  
T  
lipopt  
Herpe  
envir  
WR  
fectin  
spore

# Glutarex™



## Disinfecting And Sterilizing Solution

Corrosion Inhibited

Catalyzed GLUTARALDEHYDE

PREI  
This  
shou  
and

### Broad Spectrum Antimicrobial

Sporicidal	Pseudomonacidal
Virucidal*	Bactericidal
Tuberculocidal	Fungicidal

**Ni**

### Danger - Corrosive To Eyes -

Active Ingredient: Glutaraldehyde 2%  
Inert Ingredients: 98%

Net Contents ONE GALLON (128 fl oz) When Activated

\*See Recommendations for Use

Med

EPA Reg. No. 7182-4 EPA Est. No. 42705-MN 01

7182-4

#### DIRECTIONS FOR ACTIVATING

Add contents of the attached solution activator container to one gallon of the solution provided. **MIX WELL**. Do not dilute. Glutarex™ Disinfecting and Sterilizing Solution is now green in color and ready for use. The shelf life of the activated unused stock solution is 28 days.

#### DIRECTIONS FOR USE

##### GENERAL CLASSIFICATION

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

For Use in Hospitals, Dental Offices, Nursing Homes and Health Care Institutions.


For disinfection or sterilization of medical instruments and equipment, the items should be thoroughly cleaned, rinsed and rough dried before immersing in full strength solution. Flush and fill cleansed lumen of any hollow instrument. Avoid entrapment of air bubbles. Surfactant action will facilitate penetration of hinged and hard-to-reach areas.

**DISINFECTION** - Immerse item completely for a minimum of 10 minutes at room temperature.

**STERILIZATION** - Immerse item completely for a minimum of 10 hours at room temperature.

Using aseptic technique, remove item from the solution and rinse thoroughly with sterile water.

**RECOMMENDATION FOR USE - SEE BACK PANEL**

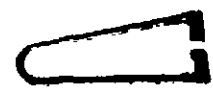

CONTAINER LABEL FOR ACTIVE SOLUTION  
GLUTAREX Disinfecting and Sterilizing Solution

#### RECOMMENDATIONS FOR USE

When used as directed for DISINFECTION, Glutarex™ Disinfecting and Sterilizing Solution will kill pathogenic bacteria (including *M. tuberculosis*, *P. aeruginosa*, *S. aureus* and *S. choleraesuis*) and pathogenic fungi (*T. mentagrophytes*).

\* This product has also been shown to irreversibly inactivate lipophilic viruses (*Adenovirus Type 5*, *Influenza A (England)* and *Herpes simplex Type 1*) and hydrophilic virus (*Poliovirus Type 1*) on environmental surfaces.


When used as directed for STERILIZATION, Glutarex™ Disinfecting and Sterilizing Solution will kill the desiccated resistant spores of *C. sporogenes* & *B. subtilis*.

#### PRECLEANING SOAK TREATMENT

This solution does NOT coagulate blood. Items from septic cases should be placed directly into fresh solution until final cleaning and terminal disinfection or sterilization of contaminated objects.

## No. 0500

Medical Products Division 3M  
3M Center, St. Paul, MN 55101

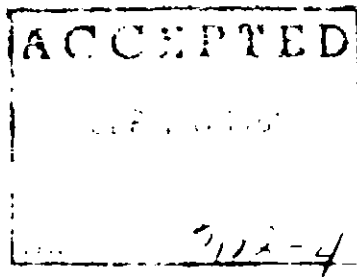

#### Wear Protective Glasses and Rubber Gloves When Mixing and for all Subsequent Handling of the Product.

**DANGER** — Causes severe EYE irritation and may cause permanent eye damage. AVOID contact with SKIN and clothing. AVOID contamination of FOOD. MAY BE HARMFUL IF SWALLOWED.

If EYE contact occurs, flush IMMEDIATELY and THOROUGHLY with water. Get medical attention. If SKIN contact occurs, wash thoroughly with water. Wash clothing.

#### STORAGE AND DISPOSAL

Store this container and any of its unused contents in a cool, safe area reserved for such disinfecting or sterilizing products. This should be away from drugs, medicines or food supplies.

This product must be kept in secured storage sufficient to make it inaccessible to children or persons unfamiliar with its proper use.

Do not re-use empty container.

Dispose container and any unused product by closing and wrapping container for trash collection.

**METAL ITEMS** — Glutarex™ Disinfecting and Sterilizing Solution will not prevent or remove oxidation caused by poor or improper plating, cleaning, or care of instruments. Caution should be used to avoid the immersion of and contact between dissimilar metals.

3M makes no warranty, expressed or implied, concerning the use of this product other than as indicated on the label. Buyer assumes all risk of loss or handling of this material when such use or handling is contrary to label instructions.

**Glutarex<sup>TM</sup> 3M**  
**Disinfecting And  
Sterilizing Solution**

**Activator Solution**  
For use with Glutarex  
Disinfecting and Sterilizing  
Solution No. 0500

**DANGER** See MSDS for details

**Net Contents 5.4 fl oz**

**DIRECTIONS:** Add the contents of this container to one gallon of solution. **MIX WELL.** Do not dilute. Glutarex<sup>TM</sup> Disinfecting and Sterilizing Solution is now green in color and ready for use.

**DANGER:** AVOID EYE or SKIN contact. Avoid severe eye and skin irritation. Wear rubber gloves and protective glasses when mixing and for all subsequent handling. In case of contact, flush **IMMEDIATELY** and **THOROUGHLY** with water. For eye contact, get medical attention.

RECEIVED  
Medical Products Division 3M  
MAY 19 1984

*11/2/84*

CONTAINER LABEL FOR ACTIVATOR SOLUTION  
GLUTAREX Disinfecting and Sterilizing Solution