

BEST AVAILABLE COPY

GlutarexTM 3M

Disinfecting And Sterilizing Solution

Activator Solution

For use with Glutarex Disinfecting and Sterilizing Solution No. 0500.

DANGER — see back panel

Net Contents 5.4 fl oz (159 ml)

DIRECTIONS: Add the contents of this container to one gallon of solution. **MIX WELL.** Do not dilute. GlutarexTM Disinfecting and Sterilizing Solution is now green in color and ready for use.

DANGER — AVOID EYE or SKIN contact. Causes severe eye and skin irritation. Wear rubber gloves and protective glasses when mixing and for all subsequent handling. In case of contact, flush **IMMEDIATELY** and **THOROUGHLY** with water. For eye contact, get medical attention.

Made in U.S.A. for

Medical Products Division/3M
3M Center • St. Paul, MN 55144

34-7017 2931-0

ACCEPTED
MAY 22 1985
U.S. Dept. of Agriculture
Fungicide and Rodenticide Act
as amended, for the pesticide
registered under
EPA Reg. No. 7182-4

CONTAINER LABEL FOR ACTIVATOR SOLUTION
GLUTAREX DISINFECTING AND STERILIZING SOLUTION (CORROSION INHIBITED)

RECOMMENDATIONS FOR USE

Glutarex can be reused for disinfection up to the earlier of 28 days or 140 cycles after activation.

When used as directed for DISINFECTION, Glutarex™ Disinfecting and Sterilizing Solution will kill pathogenic bacteria (including *P. aeruginosa*, *S. aureus* and *S. choleraesuis*) and pathogenic fungi (*T. mentagrophytes*).

*This product has also been shown to irreversibly inactivate lipophilic viruses (*Adenovirus Type 5*, *Influenza A, (England)* and *Herpes simplex Type 1*) and hydrophilic virus (*Poliovirus Type 1*) on environmental surfaces.

Note: If tuberculosis contamination is suspected, use unused activated Glutarex stock solution.

When used as directed for STERILIZATION, Glutarex™ Disinfecting and Sterilizing Solution will kill the desiccated resistant spores of *C. sporogenes* & *B. subtilis*. Use the solution only once when used for sterilization.

PRECLEANING SOAK TREATMENT

This solution does NOT coagulate blood. Items from septic cases should be placed directly into fresh solution until final cleaning and terminal disinfection or sterilization of contaminated objects.

We make no warranty, expressed or implied, concerning the use of this product other than as indicated on the label. Buyer assumes all risk of use or handling of this material, when such use or handling is contrary to label instructions.

No. 0500

Made in U.S.A. by
Medical Products Division 3M
 3M Company, St. Paul, MN 55144



ACCEPTED
 MAY 22 1985
 This product is listed on the Federal Insecticide, Fungicide, and Rodenticide Act, as amended, for the pesticide registered under EPA Reg. No. 7182-41

Wear Protective Glasses and Rubber Gloves When Mixing and for all Subsequent Handling of the Product.

DANGER—Causes severe EYE irritation and may cause permanent eye damage. AVOID contact with SKIN and clothing. AVOID contamination of FOOD. MAY BE HARMFUL IF SWALLOWED. Use in ventilated area.

If EYE contact occurs, flush IMMEDIATELY and THOROUGHLY with water. Get medical attention. If SKIN contact occurs, wash thoroughly with water. Wash clothing.

STORAGE AND DISPOSAL—Store this container and any of its unused contents in a cool safe area reserved for such disinfecting or sterilizing products. This should be away from drugs, medicines or food supplies. Do not contaminate water, food or feed by storage or disposal.

This product must be kept in secured storage sufficient to make it inaccessible to children or persons unfamiliar with its proper use.

Do not re-use empty container.

Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray mixture, or rinsate is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or ENVIRONMENTAL Control Agency, or the Hazardous Waste representative at the nearest EPA Regional Office for guidance.

CONTAINER DISPOSAL INSTRUCTIONS—Triple rinse (or equivalent). Then offer for recycling or reconditioning, or puncture and dispose of in a sanitary landfill, or incineration, or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

METAL ITEMS—Glutarex™ Disinfecting and Sterilizing Solution will not prevent or remove oxidation caused by poor or improper plating, cleaning or care of instruments. Caution should be used to avoid the immersion of and contact between dissimilar metals.

**COMPANY
NUMBER**

859/-

**PRODUCT
SERIAL NO.**

/

LABEL

APPROVAL

DATE

5-17-85