

PHENEEN SOLUTION is for cold disinfection of surgical instruments. It is an effective, rapid germicide and fungicide. Pheneen Solution contains sodium nitrite which protects stainless steel instruments from rust.

When used as directed **PHENEEN SOLUTION** is effective against *Staphylococcus aureus*, *Salmonella choleraesuis* and *Pseudomonas aeruginosa*. **PHENEEN SOLUTION** is also effective against pathogenic fungi, *Trichophyton interdigitale*.

CAUTION

Harmful if swallowed. Causes eye irritation. May cause skin irritation. In case of contact with skin or eyes, flush with plenty of water. Get medical attention if irritation persists.

Rinse empty container thoroughly with water and discard it.

ONE GALLON

WARI
Keep Out c
See side pa
cautions.

ction of surgical instruments.
fungicide. Pheneen Solution
inless steel instruments from

UTION is effective against
eraesuis and Pseudomonas
effective against pathogenic

n. May cause skin irritation.
h with plenty of water. Get

ter and discard it.

ONE GALLON



GERMICIDE FUNGICIDE DEODORANT

*For cold disinfection and storage
of surgical instruments.*

WARNING:

Keep Out of Reach of Children.

See side panel for additional
cautions.

ACTIVE INGREDIENTS
n Alkyl (60) C₁₄ 3
ammonium chlo
n Alkyl (68) C₁₂ 32
chlorides

INERT INGREDIENTS

EPA REG. NO. 7211-10
EPA EST. 7211-MN-1

Manufactured by

THE ULMER PHARMACAL CO.

Division Physicians and Hospital
Minneapolis, Minnesota