

Directions for Use

GENERAL CLASSIFICATION

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Instrument Disinfection


DO NOT DILUTE: SOLUTION IS READY FOR USE.

1. Clean instruments thoroughly and rinse well to remove detergents.
2. Remove excess water by blotting on towel when possible.
3. Immerse instruments completely in PHENEEN SOLUTION: allow to remain in solution for at least 15 minutes. They may be stored without damage indefinitely in the solution if desired.

Alternate Storage Instructions

4. Remove instruments from solution; allow excess to drain but do not wipe instruments.
5. Store in covered container or sterile wrap.

With continued use, the germicidal activity will be reduced. It is suggested the solution be discarded and replaced every week.

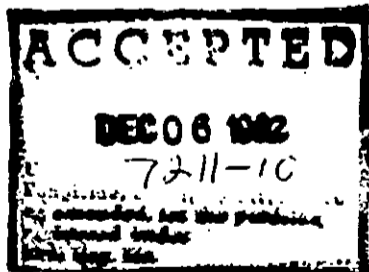

Precautionary Statements

Hazards to Human and Domestic animals

WARNING

Keep out of reach of children. May cause severe eye irritation or eye damage. Causes skin irritation. Do not get in eyes, on skin, or on clothing. Harmful if swallowed. Avoid contamination of food. Not for household use.

STORAGE AND DISPOSAL

THIS PRODUCT MUST BE KEPT UNDER LOCKED STORAGE SUFFICIENT TO MAKE IT INACCESSIBLE TO CHILDREN OR PERSON UNFAMILIAR WITH ITS PROPER USE.

—DO NOT CONTAMINATE WATER, FOOD, OR FEED BY STORAGE OR DISPOSAL

—OPEN DUMPING IS PROHIBITED

—DO NOT REUSE EMPTY CONTAINER

PESTICIDE DISPOSAL

PESTICIDE, SPRAY MIXTURE OR RINSATE THAT CANNOT BE USED OR CHEMICALLY REPROCESSED SHOULD BE DISPOSED OF IN A LANDFILL APPROVED FOR PESTICIDES OR BURIED IN A SAFE PLACE AWAY FROM WATER SUPPLIES.

CONTAINER DISPOSAL

TRIPLE RINSE (OR EQUIVALENT) AND DISPOSE IN AN INCINERATOR OR LANDFILL APPROVED FOR PESTICIDE CONTAINERS, OR BURY IN A SAFE PLACE.

GENERAL

CONSULT FEDERAL, STATE OR LOCAL DISPOSAL AUTHORITIES FOR APPROVED ALTERNATIVE PROCEDURES SUCH AS LIMITED OPEN BURNING.

FOR SALE TO, USE AND STORAGE BY
MEDICAL PERSONNEL AT INSTITUTIONS ONLY



CATALOG NO. 1660-16

PHENEEN SOLUTION is for cold disinfection of surgical instruments. It is an effective, rapid germicide and fungicide. Pheneen Solution contains sodium nitrite which protects stainless steel instruments from rust, prevents corrosion and will not damage sharps.

When used as directed PHENEEN SOLUTION is effective against *Staphylococcus aureus*, *Salmonella choleraesuis* and *Pseudomonas aeruginosa* in 10 minutes, and against pathogenic fungi, *Trichophyton interdigitale* in 15 minutes.

NET CONTENTS ONE GALLON

Active Ingredients
n-Alkyl (60%C₁₄, 30%C₁₆, 5%C₁₂, 5%C₁₈) dimethyl benzyl ammonium chlorides 0.125%
n-Alkyl (68%C₁₂, 32%C₁₄) dimethyl ethylbenzyl ammonium chlorides 0.125%
99.75%
Inert Ingredients 100.0%

KEEP OUT OF REACH OF CHILDREN
WARNING

Statement of Practical Treatment

In case of contact, immediately flush eye or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse.
If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution or if these are not available, drink large quantities of water. Avoid alcohol. Call a physician immediately.

SEE LEFT PANEL FOR
ADDITIONAL PRECAUTIONARY STATEMENTS

EPA Registration No. 7211-10

EPA Establishment No. 7211-MN-1

