

FEDERAL INSECTICIDE
AND RODENTICIDE ACT
MIC POISON REGISTER-
NO. 1211-11

igned for
' Offices,
portance.
lp in this

gallon of
lmonella
Pheneen
effective

o not get
ontamina-

of water
and wash

g whites,
ntities of

ONE GALLON



CATALOG NO. 1661-16

Guaranteed Minimum Phenol
Coefficients (A.O.A.C. 200C)
S aureus - 9.0
S typhosa - 6.0
EPA Reg. No. 7211-11
EPA Est. 7211-MN 1

WARNING:
Keep out of Reach of Children.
See side panel for first aid statement
and other precautions.

ACTIVE INGREDIENTS:
n-Alkyl (60%C14, 30%C16, 5%C12, 5%C18)
dimethyl benzyl ammonium chlorides... 0.5%
n-Alkyl (68%C12, 32%C14) dimethyl
ethylbenzyl ammonium chlorides 0.5%
INERT INGREDIENTS 99.0%
100.0%

Manufactured by

THE ULMER PHARMACAL COMPANY
Division Physicians and Hospitals Supply Co.,
Minneapolis, Minnesota 55403

Instructions for Disinfecting and Deodorizing

LABORATORY GLASSWARE, URINALS, ENAMELWARE, THERMOMETERS, RUBBER & PLASTIC ITEMS

Add 7.5 oz. Pheneen Solution NRI per gallon of water. Immerse above items in this solution for a minimum of 15 minutes. Remove items from solution and air dry without rinsing or wiping. Thermometers should be rinsed with clean water before use.

Surfaces to be disinfected should be physically clean and rinsed free of soap or detergent before disinfection.

WALLS, FLOORS AND FURNITURE.*

Add 7.5 oz. Pheneen Solution NRI per gallon of water. Apply solution with cloth or mop. Mop up excess solution and allow to dry without rinsing.

*Wood and vinyl furniture surfaces

APR 15 1975

UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND RODENTICIDE ACT
FOR ECONOMIC POISON REGISTER-
ED UNDER NO. 7211-12

For Hospital and Institutional Use

PHENEEN SANITIZER is a cleaner, disinfectant, and sanitizer designed for institutional use, such as HOSPITALS, NURSING HOMES, DOCTORS' OFFICES, AND SCHOOLS, where good housekeeping will be of prime importance. Hard surface sanitization with PHENEEN SANITIZER can help in this control.

DANGER:

Keep Out of Reach of Children. Corrosive. Causes eye damage and skin irritation. Do not get in eyes, on skin or on clothing. Protect eyes and skin when handling. Harmful if swallowed. Avoid contamination of food.

FIRST AID:

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse.

If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution or if these are not available, drink large quantities of water. Call a physician immediately.

NOTE TO PHYSICIAN:

Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

PHENEEN SANITIZER in use dilutions is non-corrosive on inanimate hard surfaces.

Do not reuse empty drum. Return to drum reconditioner or destroy by perforating or crushing and burying in a safe place.

ONE GALLON



CATALOG NO. 1685-16

Guaranteed Minimum Phenol
Coefficients (A.O.A.C.—20°C)
S. aureus—42 EPA Reg. No. 7211-12
S. typhosa—27 EPA Est. 7211-MN-1

DANGER: Keep Out of Reach of Children. See left panel for first aid and additional precautions.

Active Ingredients:

Alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chlorides	2.25%
Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chlorides	2.25%
Sodium carbonate	3.00%
Inert Ingredients:	92.50%
	100.00%

THE ULMER PHARMACAL COMPANY

Division Physicians and Hospitals Supply Co.
Minneapolis, Minnesota 55403

INSTR
AND D
WALL
Add 2
(provi
PHEN
solutio
withou
give e
This p
aerugi
stanti
Test.
If use
tact it
(1/2 oz
May l
places
Treat
water
For re
cleani
FUNG
This p
1-64 d
PHEN
floor c