

Printed in U.S.A.

Form
52C-12-82



STERILIZING GAS MIXTURE
FOR INDUSTRIAL AND MEDICAL USE ONLY IN
COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY
TRAINED TECHNICIANS.

ACTIVE INGREDIENT ETHYLENE OXIDE 10%
INERT INGREDIENT CARBON DIOXIDE 90%

WARNING

Gas and liquid under pressure. Gas reduces oxygen
available for breathing. Store and use with adequate
ventilation. See other precautions on the back of this
tag and small cylinder valve tag.

EPA EST. 7256—CA—1,—2; GA—1; IL—1; MA—1; NJ—1; TX—1

THIS CYLINDER IS
THE PROPERTY OF
MATHESON.

EPA REG. NO. 7256-7
THIS CYLINDER IS
EQUIPPED WITH
VALVE OUTLET
CGA 350


**DO NOT
REFILL**

ACCEPTED
with COMMENTS
in EPA Letter Dated:

JUN 13 1983

Under the Federal Insecticide,
Fungicide and Rodenticide Act
as amended, the pesticide
registered under EPA Reg. No.

7256-7

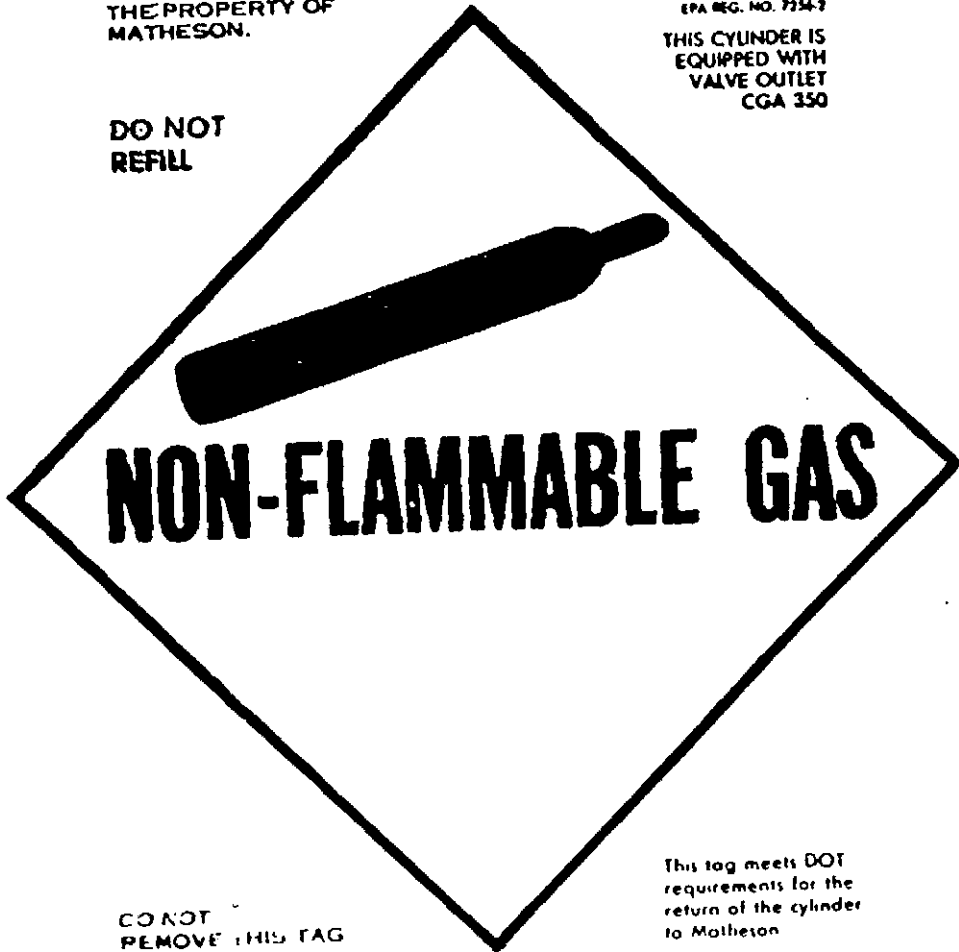

**DO NOT
REMOVE THIS TAG**

This tag meets DOT
requirements for the
return of the cylinder
to Matheson



WARNING


Gas and liquid under pressure. Gas reduces oxygen available for breathing. Store and use with adequate ventilation. See other precautions on the back of this tag and small cylinder valve tag.

EPA REG NO 7256-CA-1,-2; GA-1; IL-1; MA-1; NJ-1; TX-1

THIS CYLINDER IS THE PROPERTY OF MATHESON.

EPA REG NO 7256-2
THIS CYLINDER IS EQUIPPED WITH VALVE OUTLET CGA 350

DO NOT REFILL

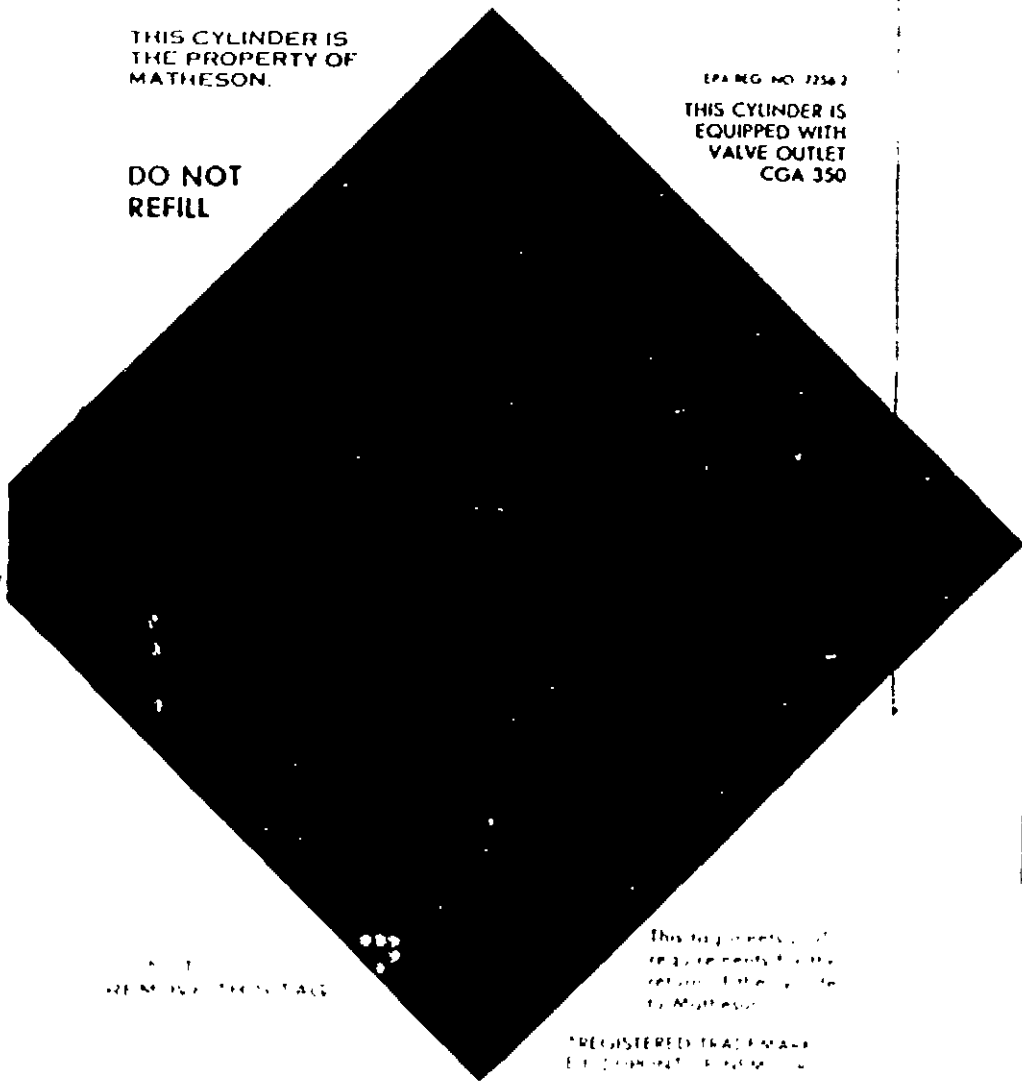

ACCEPTED
JUN 15
2056 4

SEE SMALL CYLINDER TAG

This tag meets all requirements for the return of this cylinder to Matheson.

REGISTERED TRADE NAME
EQUIMENTAL SYSTEMS

STERILIZING GAS MIXTURE
FOR INDUSTRIAL AND MEDICAL USE ONLY IN
COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY
TRAINED TECHNICIANS.

**WARNING: GAS AND LIQUID
UNDER PRESSURE**

EPA. REG. NO. 7256-2

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated place. Keep cylinder temperature from exceeding 125°F. Close valve when not in use. Never refill cylinder. Handle cylinder with care. Discharge in normal vertical position. After discharge close cylinder valve to prevent suck-back of contaminants. Ethylene oxide is subject to polymerization, use within 2 months after receipt.

**DO NOT REUSE EMPTY CYLINDERS
RETURN TO ORIGINATING SHIPPING POINT.
READ MATHESONS STERILIZING GAS MIXTURES
BROCHURE BEFORE USING.**

ACCEPTED
with COMMENTS
in EPA Letter dated:


JUN 13 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
is amended, for the pesticide
registered under EPA Reg. No.

SHIP TO

7256-4

RETURN TO MATHESON

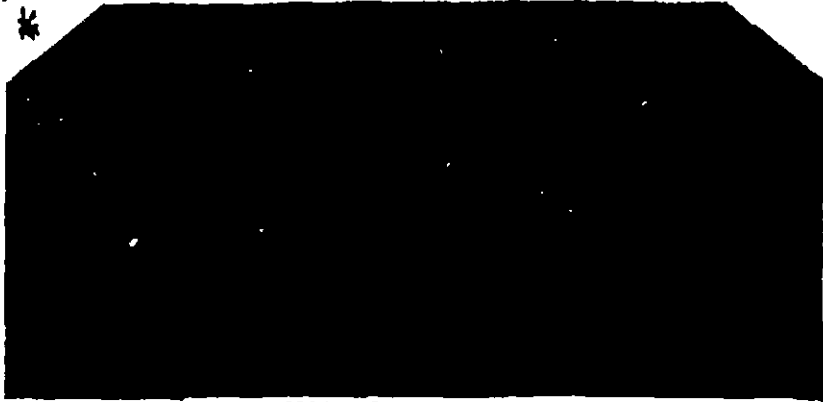

**WARNING: GAS AND LIQUID
UNDER PRESSURE**

EPA. REG. NO. 7256-2

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated place. Keep cylinder temperature from exceeding 125°F. Close valve when not in use. Never refill cylinder. Handle cylinder with care. Discharge in normal vertical position. After discharge close cylinder valve to prevent suck-back of contaminants. Ethylene oxide is subject to polymerization, use within 2 months after receipt.

**DO NOT REUSE EMPTY CYLINDERS
RETURN TO ORIGINATING SHIPPING POINT.
READ MATHESONS STERILIZING GAS MIXTURES
BROCHURE BEFORE USING.**

**EXCEPT WHEN IN USE KEEP THIS CAP ON THE CYLINDER.
DO NOT REMOVE THIS TAG.**
RETURN INSTRUCTIONS: Federal Law prohibits the improper discarding of this cylinder and cylinder contents. Obliterate Ship to Address.
CYLINDER MUST BE RETURNED TO MATHESON.

ACCEPTED
with COMMENTS
in F.I.A. Letter Dated:

JUN 13 1983

Under the Federal Insecticide,
Fungicide and Rodenticide Act
as amended by the Pesticide
reg. amend under EPA Reg. No.

7256-2

SHIP TO

RETURN TO MATHESON

* ALL BASE PLATES TO PRINT GREEN
ALL FILM PLATES TO PRINT BLACK



Form No.
52V-1-82

Matheson
A SEARLE COMPANY

East Rutherford, N.J. 07073 / Morrow, Georgia 30260
Gloucester, Massachusetts 01930 / Joliet, Illinois 60434
La Porte, Texas 77571 / Cucamonga, California 91730
Healdsburg, California 94560 / Dorsey, Maryland 21727
Bridgeport, New Jersey 08014 / Gonzales, Louisiana 70737
Dayton, Ohio 45424 / Twinsburg, Ohio 44087

**STERILIZING
GAS MIXTURE**

FOR INDUSTRIAL AND MEDICAL USE ONLY
IN COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY TRAINED
TECHNICIANS.

ACTIVE INGREDIENT 10%
ETHYLENE OXIDE

INERT INGREDIENT 90%
CARBON DIOXIDE
EPA REG. NO. 7256-4

Gross Wt. _____ lbs.

Tare Wt. _____ lbs.

Net Wt. _____ lbs.

Filling Date _____

ACCEPTED
with COMMENTS
In EPA Letter Dated:

JUN 13 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, this pesticide
registered under EPA Reg. No.

7256-4



**WARNING
GAS AND LIQUID
UNDER PRESSURE
WARNING**

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated
place. Keep cylinder temperature from
exceeding 125°F. Close valve when not in
use. Never refill cylinder. Handle cylinder
with care. Discharge in normal vertical
position. After discharge close cylinder valve
to prevent suck-back of contaminants.
Ethylene Oxide is subject to polymerization,
use within 2 months after receipt.

Avoid breathing vapors. Avoid contact
with skin & eyes. In case of contact, flush
thoroughly with water for at least 15 minutes.
For eyes, get medical attention. Avoid
contact with oxidizing materials.

Do not reuse empty cylinder. Return cylin-
der to originating shipping point.

READ MATHESONS STERILIZING
GAS MIXTURE BROCHURE
BEFORE USING

Matheson Division
Searle Medical Products USA, Inc.
2727 West Good Hope Road
Milwaukee, WI 53209

13 JUN 1983

Attention: Dr. Edward Flaherty

Gentlemen:

Subject: 10% Ethylene Oxide/90% Carbon Dioxide
EPA Registration No. 7256-4
Your Submission Dated February 16, 1983

The amendment referred to above, submitted in connection with registration under FIFRA, is acceptable provided that you:

1. Make the labeling changes listed below before you release the product for shipment bearing the amended labeling:

Under "Read Matheson's Sterilizing Gas Mixture Brochure Before Using" add the following:

GENERAL CLASSIFICATION

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

2. Submit five (5) copies of your final printed labeling before you release the product for shipment. Refer to the A-79 Enclosure for a further description of final printed labeling.

Your release for shipment of the product bearing the amended labeling constitutes acceptance of these conditions.

A stamped copy of the labeling is enclosed for your records.

Sincerely yours,

John U. Lee
Product Manager (31)
Disinfectants Branch
Registration Division (TS-767C)

Enclosure

TS-767C:LEE:DCR-18309:WANG-0487C:hmv:Raven:479-2013:5/19/83

CONCURRENCES							
SYMBOL	7256-4						
SURNAME	Lee						
DATE	5/19/83						

Printed in U.S.A.

Form
52C-12-82



STERILIZING GAS MIXTURE
FOR INDUSTRIAL AND MEDICAL USE ONLY IN
COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY
TRAINED TECHNICIANS.

ACTIVE INGREDIENT ETHYLENE OXIDE 10%
INERT INGREDIENT CARBON DIOXIDE 90%

WARNING

Gas and liquid under pressure. Gas reduces oxygen available for breathing. Store and use with adequate ventilation. See other precautions on the back of this tag and small cylinder valve tag.

EPA EST. 7256—CA—1,—2; GA—1; IL—1; MA—1; NJ—1; TX—1

THIS CYLINDER IS
THE PROPERTY OF
MATHESON.

EPA REG. NO. 7256-2

THIS CYLINDER IS
EQUIPPED WITH
VALVE OUTLET
CGA 350


DO NOT
REFILL

ACCEPTED
with COMMENTS
in EPA Letter Dated:

JUN 13 1983

Under the Fungicide, Insecticide,
Fungicide and Rodenticide Act
as amended, the pesticide
registered under EPA Reg. No.

7256-4

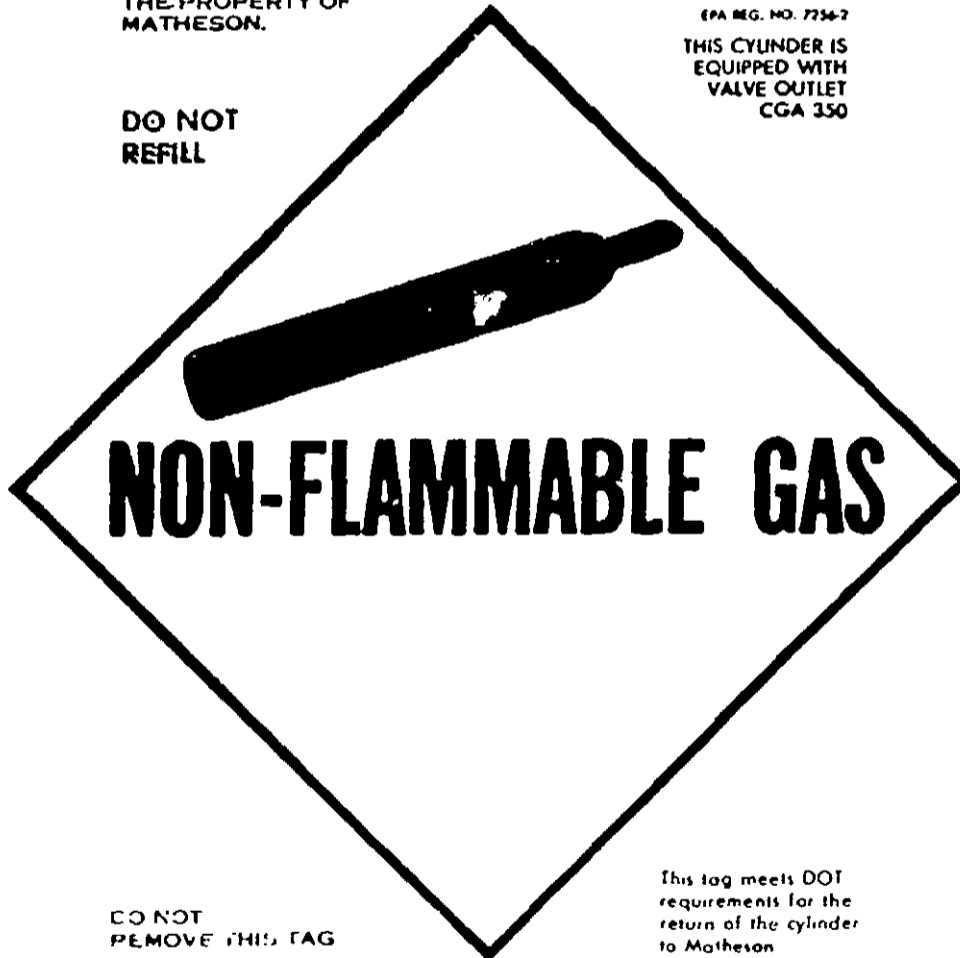

DO NOT
REMOVE THIS TAG

This tag meets DOT
requirements for the
return of the cylinder
to Matheson



WARNING

Gas and liquid under pressure. Gas reduces oxygen available for breathing. Store and use with adequate ventilation. See other precautions on the back of this tag and small cylinder valve tag.


EPA EST. 7256—CA—1,—2; GA—1; IL—1; MA—1; NJ—1; TX—1

THIS CYLINDER IS THE PROPERTY OF MATHESON.

EPA REG. NO. 7256-2

THIS CYLINDER IS EQUIPPED WITH VALVE OUTLET CGA 350

DO NOT REFILL

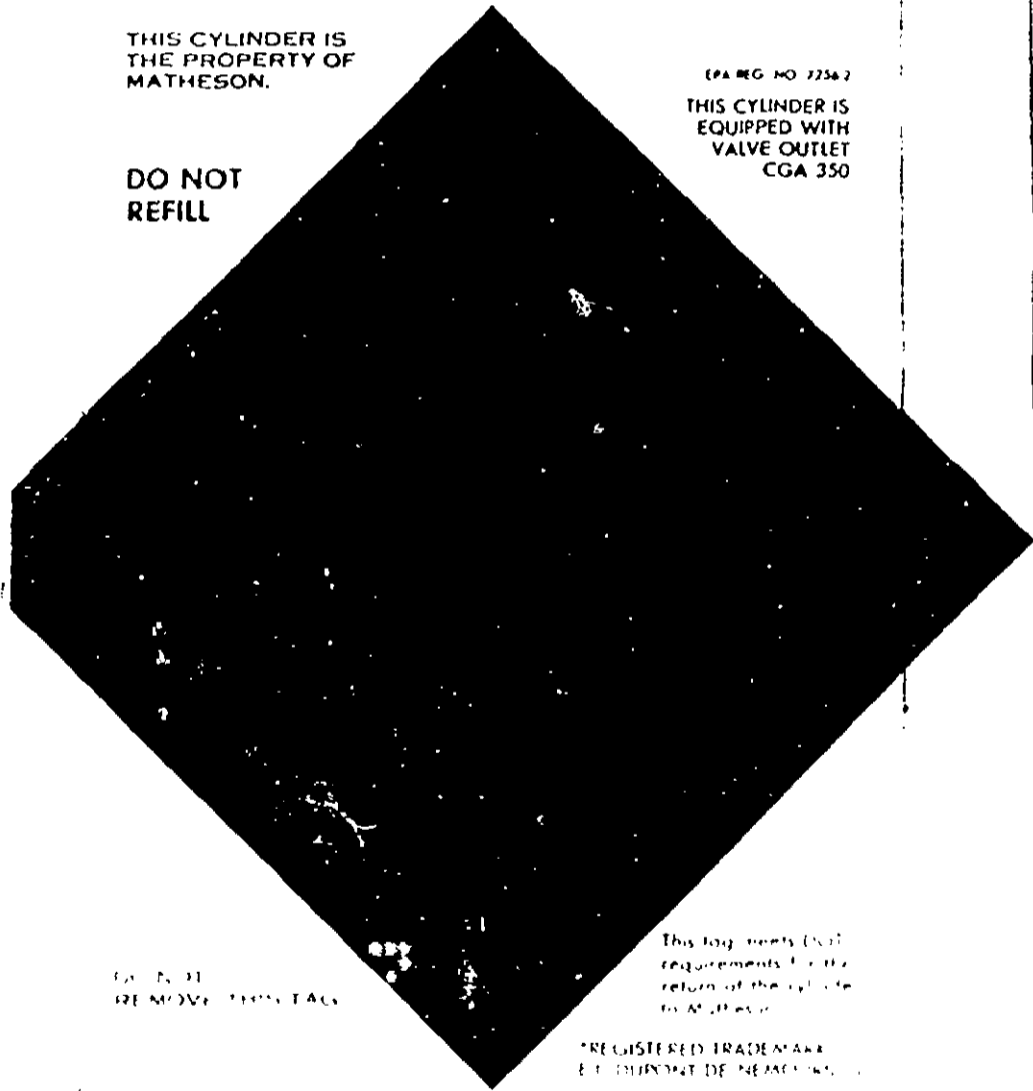

DO NOT REMOVE THIS TAG

This tag meets local requirements for the return of the cylinder to Matheson.

REGISTERED TRADEMARK
ET. DUMONT DE NEAUMEY

RESERVED

ASSIGNED

0111

10 24 1971

STERILIZING GAS MIXTURE
FOR INDUSTRIAL AND MEDICAL USE ONLY IN
COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY
TRAINED TECHNICIANS.

**WARNING: GAS AND LIQUID
UNDER PRESSURE**

EPA REG. NO. 7256-2

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated place. Keep cylinder temperature from exceeding 125°F. Close valve when not in use. Never refill cylinder. Handle cylinder with care. Discharge in normal vertical position. After discharge close cylinder valve to prevent suck-back of contaminants. Ethylene oxide is subject to polymerization, use within 2 months after receipt.

DO NOT REUSE EMPTY CYLINDERS
RETURN TO ORIGINATING SHIPPING POINT.

**READ MATHESONS STERILIZING GAS MIXTURES
BROCHURE BEFORE USING.**

ACCEPTED
with COMMENTS
in EPA Letter Book

JUN 13 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.

SHIP TO

RETURN TO MATHESON



**WARNING: GAS AND LIQUID
UNDER PRESSURE**

EPA REG NO 7256-2

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated place. Keep cylinder temperature from exceeding 125°F. Close valve when not in use. Never refill cylinder. Handle cylinder with care. Discharge in normal vertical position. After discharge close cylinder valve to prevent suck-back of contaminants. Ethylene oxide is subject to polymerization, use within 2 months after receipt.

**DO NOT REUSE EMPTY CYLINDERS
RETURN TO ORIGINATING SHIPPING POINT.**

**READ MATHESONS STERILIZING GAS MIXTURES
BROCHURE BEFORE USING.**

**EXCEPT WHEN IN USE KEEP THIS CAP ON THE CYLINDER.
DO NOT REMOVE THIS TAG.**
**RETURN INSTRUCTIONS: Federal Law prohibits the improper discarding of
this cylinder and cylinder contents. Obliterate Ship to Address.
CYLINDER MUST BE RETURNED TO MATHESON.**

ACCEPTED
W. MATHESON
JUN 13 1951
1234

SHIP TO

RETURN TO MATHESON

* ALL BASE PLATES TO PRINT GREEN
ALL FILM PLATES TO PRINT BLACK



Form No.
52V-1-B2

Matheson
A SEARLE COMPANY

East Rutherford, N.J. 07073 / Morrow Georgia 30260
Gloucester, Massachusetts 01930 / Joliet Illinois 60434
La Porte, Texas 77571 / Cuckamonga California 91730
Newark, California 94560 / Dorsey, Maryland 21227
Bridgeport, New Jersey 08014 / Gonzales, Louisiana 70737
Canton, Ohio 45224 / Twinsburg, Ohio 44087

**STERILIZING
GAS MIXTURE**

FOR INDUSTRIAL AND MEDICAL USE ONLY
IN COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY TRAINED
TECHNICIANS.

ACTIVE INGREDIENT 10%
ETHYLENE OXIDE

INERT INGREDIENT 90%
CARBON DIOXIDE

EPA REG. NO. 7256-4

Gross Wt. _____ lbs.

Tare Wt. _____ lbs.

Net Wt. _____ lbs.

Filling Date _____

ACCEPTED
with COMMENTS
in EPA Label Book

JUN 13 1963

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
Registration No. 7256-4
EPA Reg. No.

7256-4



**WARNING
GAS AND LIQUID
UNDER PRESSURE
WARNING**

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated
place. Keep cylinder temperature from
exceeding 125°F. Close valve when not in
use. Never refill cylinder. Handle cylinder
with care. Discharge in normal vertical
position. After discharge close cylinder valve
to prevent suck-back of contaminants.
Ethylene Oxide is subject to polymerization,
use within 2 months after receipt.

Avoid breathing vapors. Avoid contact
with skin & eyes. In case of contact, flush
thoroughly with water for at least 15 minutes.
For eyes, get medical attention. Avoid
contact with oxidizing materials.

Do not reuse empty cylinder. Return cylin-
der to originating shipping point.

**READ MATHESONS STERILIZING
GAS MIXTURE BROCHURE
BEFORE USING**

Matheson Division
Searle Medical Products USA, Inc.
2727 West Good Hope Road
Milwaukee, WI 53209

13 JUN 1983

Attention: Dr. Eduardo Planerty

Gentlemen:

Subject: 10% Ethylene Oxide/90% Carbon Dioxide
EPA Registration No. 7256-4
Your Submission Dated February 16, 1983

The amendment referred to above, submitted in connection with registration under FIFRA, is acceptable provided that you:

1. Make the labeling changes listed below before you release the product for shipment bearing the amended labeling:

Under "Read Matheson's Sterilizing Gas Mixture Brochure Before Using" add the following:

GENERAL CLASSIFICATION


It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

2. Submit five (5) copies of your final printed labeling before you release the product for shipment. Refer to the A-79 Enclosure for a further description of final printed labeling.

Your release for shipment of the product bearing the amended labeling constitutes acceptance of these conditions.

A stamped copy of the labeling is enclosed for your records.

Sincerely yours,

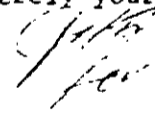

John H. Lee
Product Manager (31)
Disinfectants Branch
Registration Division (28-767C)

Enclosure

28-767C:LBE:DCR-16309:WANG-0487C:HEW:Raven:479-2013:5/19/83

DATE	INITIALS	REMARKS	DATE	INITIALS	REMARKS	DATE	INITIALS	REMARKS	DATE	INITIALS	REMARKS
5/19/83											

EPA Form 1483

OFFICIAL FILE COPY