

Printed in U.S.A.

Form
53C-12-82



STERILIZING GAS MIXTURE
FOR INDUSTRIAL AND MEDICAL USE ONLY IN
COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY
TRAINED TECHNICIANS.

ACTIVE INGREDIENT ETHYLENE OXIDE 12%
INERT INGREDIENT *FREON - 12 88%

WARNING

Gas and liquid under pressure. Gas reduces oxygen available for breathing. Store and use with adequate ventilation. See other precautions on the back of this tag and small cylinder valve tag.

EPA EST. 7256—CA—1,—2; GA—1; IL—1; MA—1; NJ—1; TX—1

THIS CYLINDER IS
THE PROPERTY OF
MATHESON.

EPA REG. NO. 7256-2
THIS CYLINDER IS
EQUIPPED WITH
VALVE OUTLET
CGA 350

DO NOT
REFILL

ACCEPTED
with COMMENTS
in EPA Letter Dated:

JUN 13 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.

7256-2
7256-5

NON-FLAMMABLE GAS

BEST DOCUMENT AVAILABLE

DO NOT
REMOVE THIS TAG

This tag meets DOT
requirements for the
return of the cylinder
to Matheson

*REGISTERED TRADEMARK
E.I. DUPONT DE NEMOURS CO



WARNING:

Gas and liquid under pressure. Gas reduces oxygen available for breathing. Store and use with adequate ventilation. See other precautions on the back of this tag and small cylinder valve tag.


EPA EST. 7256—CA—1,—2; GA—1; IL—1; MA—1; NJ—1; TX—1

THIS CYLINDER IS THE PROPERTY OF MATHESON.

EPA REG. NO. 7256-7

THIS CYLINDER IS EQUIPPED WITH VALVE OUTLET CGA 350

DO NOT REFILL

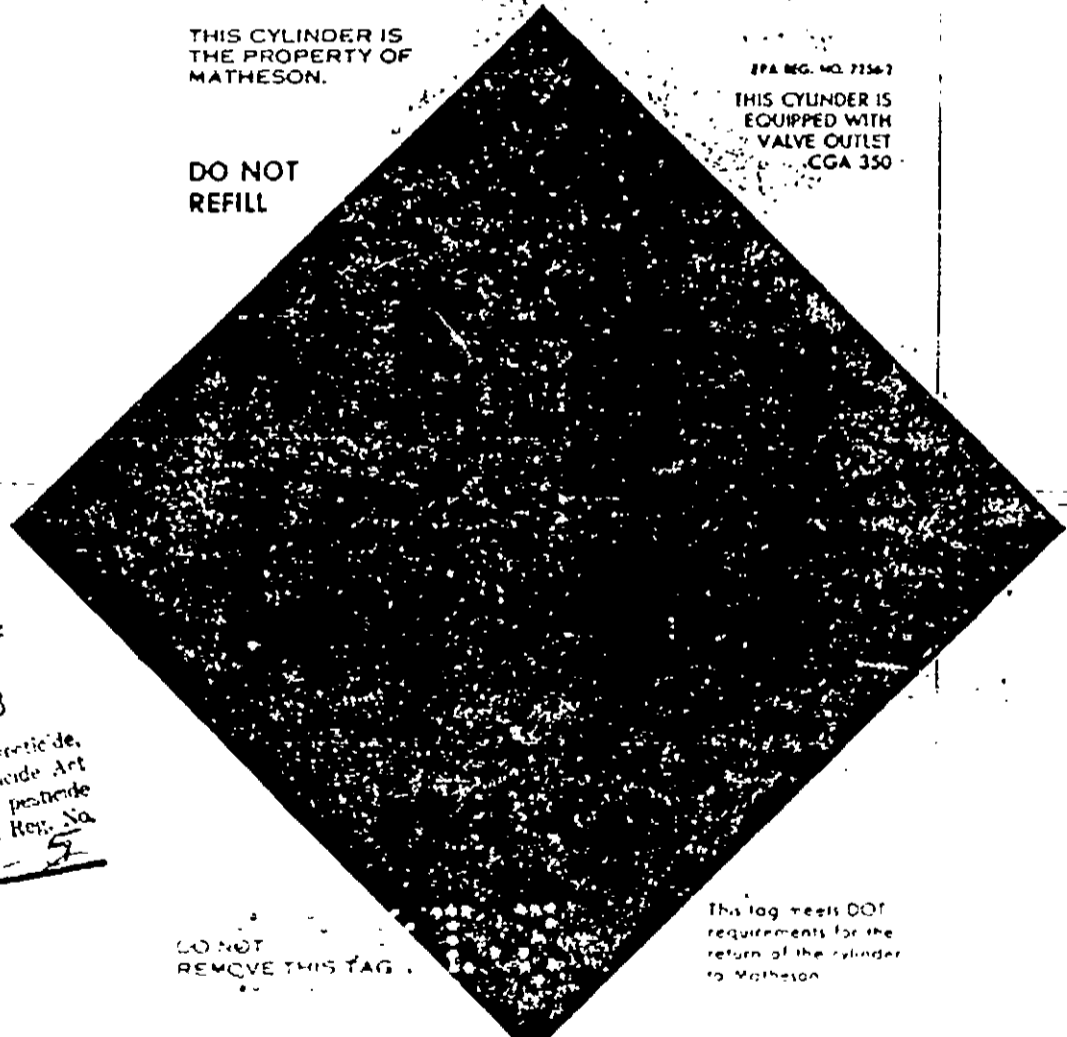

ACCEPTED with COMMENTS in EPA Letter Dated

JUN 13 1983

Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide registered under EPA Reg. No. 7256-7

DO NOT REMOVE THIS TAG

This tag meets DOT requirements for the return of the cylinder to Matheson

7256-5 3

STERILIZING GAS MIXTURE
FOR INDUSTRIAL AND MEDICAL USE ONLY IN
COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY
TRAINED TECHNICIANS.

**WARNING: GAS AND LIQUID
UNDER PRESSURE**

EPA REG. NO. 7256-2

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated place. Keep cylinder temperature from exceeding 125°F. Close valve when not in use. Never refill cylinder. Handle cylinder with care. Discharge in normal vertical position. After discharge close cylinder valve to prevent suck-back of contaminants. Ethylene oxide is subject to polymerization, use within 2 months after receipt.

**DO NOT REUSE EMPTY CYLINDERS
RETURN TO ORIGINATING SHIPPING POINT.
READ MATHESONS STERILIZING GAS MIXTURES
BROCHURE BEFORE USING.**


ACCEPTED
with COMMENTS
In EPA Letter Dated:

JUN 13 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.
[Signature]

SHIP TO

RETURN TO MATHESON

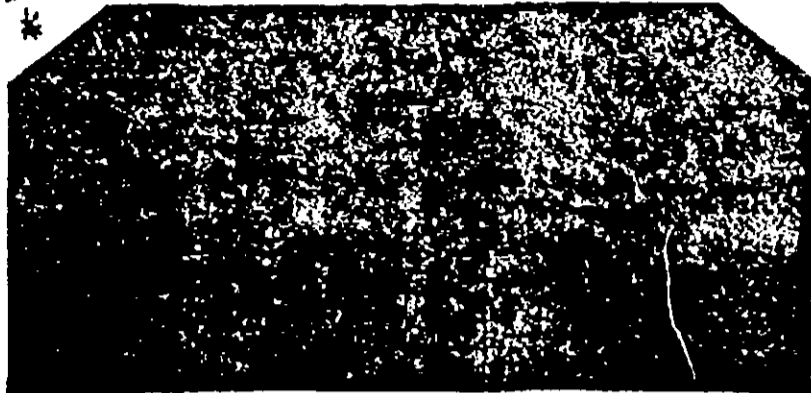

**WARNING: GAS AND LIQUID
UNDER PRESSURE**

EPA REG. NO. 7256-2

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated place. Keep cylinder temperature from exceeding 125°F. Close valve when not in use. Never refill cylinder. Handle cylinder with care. Discharge in normal vertical position. After discharge close cylinder valve to prevent suck-back of contaminants. Ethylene oxide is subject to polymerization, use within 2 months after receipt.

**DO NOT REUSE EMPTY CYLINDERS
RETURN TO ORIGINATING SHIPPING POINT.**

**READ MATHESONS STERILIZING GAS MIXTURES
BROCHURE BEFORE USING.**

**EXCEPT WHEN IN USE KEEP THIS CAP ON THE CYLINDER.
DO NOT REMOVE THIS TAG.**

**RETURN INSTRUCTIONS: Federal Law prohibits the improper discarding of
this cylinder and cylinder contents. Obliterate Ship to Address.
CYLINDER MUST BE RETURNED TO MATHESON.**

ACCEPTED
with COMMENTS
In EPA Letter Dated:

JUN 13 1983

Under the Federal Insecticide,
Fungicide, and Plant Growth Regulator Act
as amended, or the pesticide
registered under EPA Reg. No.
7456-2

SHIP TO

RETURN TO MATHESON

* ALL BASE PLATES TO PRINT GREEN
ALL FILM PLATES TO PRINT BLACK

7256-5 5



Form No.
53V-1-82

Matheson
A SEARLE COMPANY

East Rutherford, N.J. 07073 / Morrow Georgia 30260
Gloucester, Massachusetts 01930 / Joliet Illinois 60434
La Porte, Texas 77571 / Cucamonga, California 91730
Newark, California 94560 / Dorsey Maryland 21227
Bridgeport, New Jersey 08014 / Gonzales, Louisiana 70737
Dayton Ohio 45424 / Twinsburg, Ohio 44087

**STERILIZING
GAS MIXTURE**

FOR INDUSTRIAL AND MEDICAL USE ONLY
IN COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY TRAINED
TECHNICIANS.

ACTIVE INGREDIENT 12%
ETHYLENE OXIDE

INERT INGREDIENT 88%

*FREON - 12

EPA REG. NO. 7256-5

Gross Wt. _____ lbs.

Total Wt. _____ lbs.

Net Wt. _____ lbs.

Filling Date _____

REGISTERED TRADEMARK
E.I. DUPONT DE NEMOURS CO.

ACCEPTED
with COMMENTS
by EPA Letter Dated:

JUN 13 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.

[Handwritten signature]



**WARNING
GAS AND LIQUID
UNDER PRESSURE
WARNING
STORAGE AND DISPOSAL**

Use and store in cool, dry, well ventilated
place. Keep cylinder temperature from
exceeding 125°F. Close valve when not in
use. Never refill cylinder. Handle cylinder
with care. Discharge in normal vertical
position. After discharge close cylinder valve
to prevent suck-back of contaminants.
Ethylene Oxide is subject to polymerization,
use within 2 months after receipt.

Avoid breathing vapors. Avoid contact
with skin & eyes. In case of contact, flush
thoroughly with water for at least 15 minutes.
For eyes, get medical attention. Avoid
contact with oxidizing materials.

Do not reuse empty cylinder. Return cylin-
der to originating shipping point.

**READ MATHESONS STERILIZING
GAS MIXTURE BROCHURE
BEFORE USING**

BEST DOCUMENT AVAILABLE

Matheson Division
Searle Medical Products USA, Inc.
2727 West Good Hope Road
Milwaukee, WI 53209

13 JUN 1983

Attention: Dr. Edward Flaherty

Gentlemen:

Subject: 12% Ethylene Oxide 88% Freon - 12
EPA Registration No. 7256-5
Your Submission Dated February 16, 1983

The amendment referred to above, submitted in connection with registration under FIFRA, is acceptable provided that you:

1. Make the labeling changes listed below before you release the product for shipment bearing the amended labeling:

Under "Read Matheson Sterilizing Gas Mixture Brochure Before Using" add the following:

GENERAL CLASSIFICATION

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

2. Submit five (5) copies of your final printed labeling before you release the product for shipment. Refer to the A-79 Enclosure for a further description of final printed labeling.

Your release for shipment of the product bearing the amended labeling constitutes acceptance of these conditions.

A stamped copy of the labeling is enclosed for your records.

Sincerely yours,

John H. Lee
Product Manager (31)
Disinfectants Branch
Registration Division (TS-767C)

Enclosure

TS-767C:LEE:DCR-18309:WANG-0487C:bmw:Raven:479-2013:5/19/83

CONCURRENCES							
SYMBOL	TS-767						
SIGNATURE	LEE						
DATE	5/19/83						

Printed in U.S.A.

Form
53C-12-82



STERILIZING GAS MIXTURE
FOR INDUSTRIAL AND MEDICAL USE ONLY IN
COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY
TRAINED TECHNICIANS.

ACTIVE INGREDIENT ETHYLENE OXIDE 12%
INERT INGREDIENT *FREON - 12 88%

WARNING

Gas and liquid under pressure. Gas reduces oxygen
available for breathing. Store and use with adequate
ventilation. See other precautions on the back of this
tag and small cylinder valve tag.

EPA EST. 7256—CA—1,—2; GA—1; IL—1; MA—1; NJ—1; TX—1

THIS CYLINDER IS
THE PROPERTY OF
MATHESON.

EPA REG NO. 7256-2
THIS CYLINDER IS
EQUIPPED WITH
VALVE OUTLET
CGA 350

DO NOT
REFILL

ACCEPTED
with COMMENTS
In EPA Letter Dated:

JUN 13 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for use pursuant to
registered under EPA Reg. No.

7256-2
7256-5

NON-FLAMMABLE GAS

DO NOT
REMOVE THIS TAG

This tag meets DOT
requirements for the
return of the cylinder
to Matheson

*REGISTERED TRADEMARK
E. I. DUPONT DE NEMOURS CO



WARNING


Gas and liquid under pressure. Gas reduces oxygen available for breathing. Store and use with adequate ventilation. See other precautions on the back of this tag and small cylinder valve tag.

EPA EST 7256-CA-1,-2; GA-1; IL-1; MA-1; NJ-1; TX-1

THIS CYLINDER IS
THE PROPERTY OF
MATHESON.

EPA REG. NO 7256-2
THIS CYLINDER IS
EQUIPPED WITH
VALVE OUTLET
CGA 350

DO NOT
REFILL

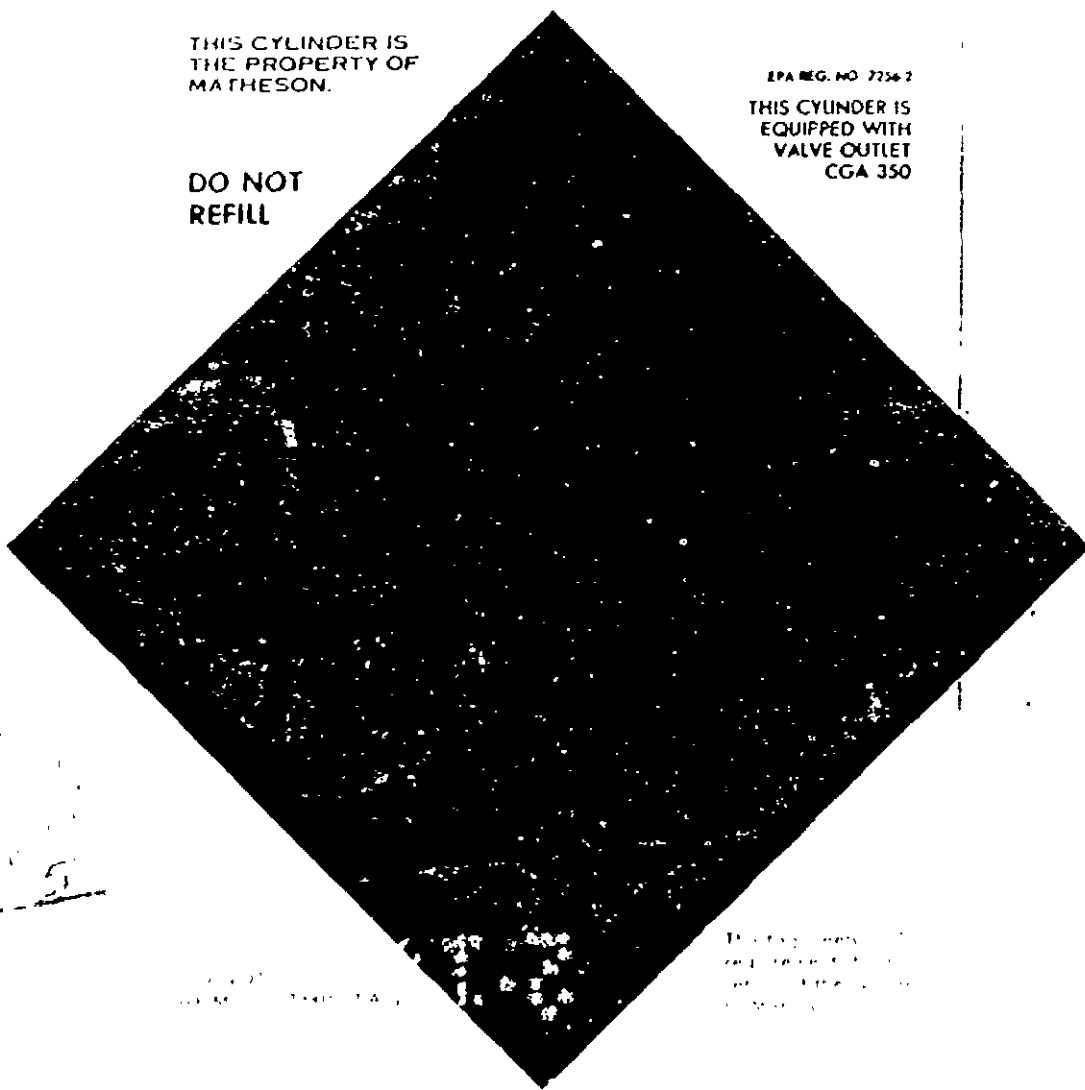

ACCEPTED
MATHESON
Cylinder No. 12345
Date 12/15/88
By [Signature]

12/15/88

12/15/88

STERILIZING GAS MIXTURE
FOR INDUSTRIAL AND MEDICAL USE ONLY IN
COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY
TRAINED TECHNICIANS.

**WARNING: GAS AND LIQUID
UNDER PRESSURE**

EPA REG. NO. 7256-2

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated place. Keep cylinder temperature from exceeding 125°F. Close valve when not in use. Never refill cylinder. Handle cylinder with care. Discharge in normal vertical position. After discharge close cylinder valve to prevent suck-back of contaminants. Ethylene oxide is subject to polymerization, use within 2 months after receipt.

DO NOT REUSE EMPTY CYLINDERS
RETURN TO ORIGINATING SHIPPING POINT.

**READ MATHESONS STERILIZING GAS MIXTURES
BROCHURE BEFORE USING.**

ACCEPTED
WITH COMMENTS
In EPA Letter 10-608

JUN 13 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No. 7256-2

SHIP TO

RETURN TO MATHESON



WARNING: GAS AND LIQUID UNDER PRESSURE

EPA REG. NO 7256-2

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated place. Keep cylinder temperature from exceeding 125°F. Close valve when not in use. Never refill cylinder. Handle cylinder with care. Discharge in normal vertical position. After discharge close cylinder valve to prevent suck-back of contaminants. Ethylene oxide is subject to polymerization, use within 2 months after receipt.

**DO NOT REUSE EMPTY CYLINDERS
RETURN TO ORIGINATING SHIPPING POINT.**

**READ MATHESONS STERILIZING GAS MIXTURES
BROCHURE BEFORE USING.**

**EXCEPT WHEN IN USE KEEP THIS CAP ON THE CYLINDER.
DO NOT REMOVE THIS TAG.**

RETURN INSTRUCTIONS: Federal Law prohibits the improper discarding of this cylinder and cylinder contents. Obliterate Ship to Address. CYLINDER MUST BE RETURNED TO MATHESON.

ACCEPTED
MATHESON
JUN 13 1983
EPA REG. NO. 7256-2
5

SHIP TO

RETURN TO MATHESON

* ALL BASE PLATES TO PRINT GREEN
ALL FILM PLATES TO PRINT BLACK



Form No.
53V-1-82

Matheson
A SEALE COMPANY

East Rutherford, N.J. 07073 / Morrow Georgia 30260
Gloucester, Massachusetts 01930 / Joliet Illinois 60434
La Porte, Texas 77571 / Cucamonga, California 91730
Newark, California 94560 / Dorsey Maryland 21227
Bridgeport, New Jersey 08014 / Gonzales Louisiana 70737
Dayton Ohio 45424 / Twinsburg, Ohio 44087

**STERILIZING
GAS MIXTURE**

FOR INDUSTRIAL AND MEDICAL USE ONLY
IN COMMERCIAL GAS STERILIZER BY MEDICAL
PROFESSIONALS OR APPROPRIATELY TRAINED
TECHNICIANS.

ACTIVE INGREDIENT 12%
ETHYLENE OXIDE

INERT INGREDIENT 88%
*FREON - 12

EPA REG. NO. 7256-5

Gross Wt. _____ lbs.

Tare Wt. _____ lbs.

Net Wt. _____ lbs.

Filling Date _____

REGISTERED TRADEMARK
E.I. DUPONT DE NEMOURS CO.

ACCREDITED
BY EPA
IN THE LABORATORY

JUN 13 1983

Handwritten notes and signatures, including a large '5' at the bottom.



**WARNING
GAS AND LIQUID
UNDER PRESSURE
WARNING**

STORAGE AND DISPOSAL

Use and store in cool, dry, well ventilated place. Keep cylinder temperature from exceeding 125°F. Close valve when not in use. Never refill cylinder. Handle cylinder with care. Discharge in normal vertical position. After discharge close cylinder valve to prevent suck-back of contaminants. Ethylene Oxide is subject to polymerization, use within 2 months after receipt.

Avoid breathing vapors. Avoid contact with skin & eyes. In case of contact, flush thoroughly with water for at least 15 minutes. For eyes, get medical attention. Avoid contact with oxidizing materials.

Do not reuse empty cylinder. Return cylinder to originating shipping point.

**READ MATHESONS STERILIZING
GAS MIXTURE BROCHURE
BEFORE USING**

BEST DOCUMENT AVAILABLE

Matheson Division
Searle Medical Products USA, Inc.
2727 West Good Hope Road
Milwaukee, WI 53209

18/11

Attention: Dr. Lowato Flaherty

Gentlemen:

Subject: 12% ethylene oxide 88% Freon - 12
EPA registration No. 7256-9
Your Submission Dated February 16, 1983

The amendment referred to above, submitted in connection with registration under FIFRA, is acceptable provided that you:

- 1. Make the labeling changes listed below before you release the product for shipment bearing the amended labeling:

Under "Read Mathesons Sterilizing Gas Mixture Brochure Before Using" and the following:

GENERAL CLASSIFICATION

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

- 2. Submit five (5) copies of your final printed labeling before you release the product for shipment. Refer to the A-79 Enclosure for a further description of final printed labeling.

Your release for shipment of the product bearing the amended labeling constitutes acceptance of these conditions.

A stamped copy of the labeling is enclosed for your records.

Sincerely yours,

John H. Lee
Product Manager (31)
Disinfectants Branch
Registration Division (TS-767C)

Enclosure

TS-767C:LEE:DCR-18309:WANG-0400:mlw:Raven:4/9-2013:5/19/83

DOCUMENTS							
SYMBOL	11						
FILENAME							
DATE							

EPA Form 1320-1-481

OFFICIAL FILE COPY