

Instrument & Equipment Disinfectant

For Disinfecting
Operating Room and
Laboratory Equipment,
Inhalation Therapy
Equipment, Anesthesia
Equipment.

USE DIRECTIONS
Ready to Use—
No Mixing Required

ACTIVE INGREDIENTS:


n-Alkyl (60% C ₁₄ , 30% C ₁₆ , 5% C ₁₂ , 5% C ₁₈) dimethyl benzyl ammonium chlorides	0.0625%
n-Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chlorides	0.0625%
INERT INGREDIENTS	99.8750%
	100.0000%

CONTAINS SODIUM NITRITE

CAUTION: Keep Out of Reach of Children
Avoid contact with eyes. In case of eye contact, flush with water.
Rinse container thoroughly and discard.



Distributed by **WATER SOLUBLE PRODUCTS**
Division of M.D. Industries, Inc. Deerfield, Illinois 60015, U.S.A.

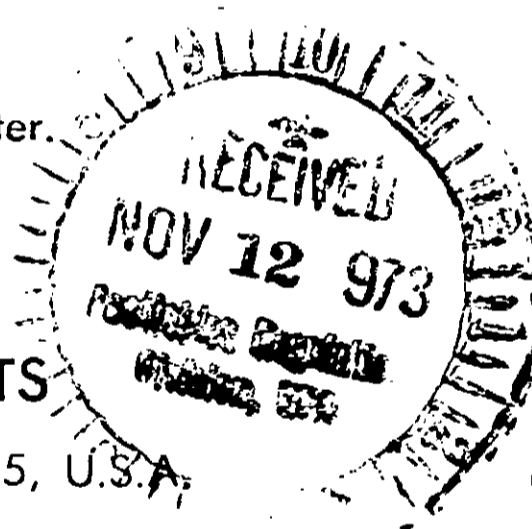

For Disinfecting Instruments: Pre-clean instruments, barber tools, dental equipment, etc. and immerse in "K.I.L. 4" for at least ten (10) minutes.

"K.I.L. 4" at its ready to use level is effective against Pseudomonas Aeruginosa PRD 10. This has been substantiated by the AOAC Use Dilution Confirmation Test.

For Disinfecting Larger Equipment: Wash equipment such as sinks, tables, counters and other hard surfaces thoroughly with "K.I.L. 4": Rinse and repeat if heavily soiled.

Fungicidal Properties: "K.I.L. 4" at its ready to use level is fungicidal against pathogenic fungi such as Trichophyton interdigitale.

**HARMLESS TO METAL, RUBBER, PLASTICS
IN USE SOLUTION.**

CONTENTS: 1 GALLON

CAT. NO. 30-00

