

GERMICIDE

AQUEOUS

For Cold Disinfection of Surgical-Dental Instruments

ACTIVE INGREDIENTS:

Alkyl (C14, 50%; C12, 40%; C16, 10%) Dimethyl Benzyl Ammonium Chloride	0.133%
Inert Ingredients	99.867%

Contains Sodium Nitrite

CAUTION

KEEP OUT OF REACH OF CHILDREN

NOT FOR INTERNAL USE

Harmful if swallowed. Avoid getting in eyes and contamination of food. In case of contact, flush with plenty of water. If irritation persists, get medical attention.

THE SOLUTION WILL NOT RUST OR CORRODE SURGICAL STEEL OR OTHER METALS. MAY BE USED FOR DISINFECTION OF METAL, PLASTIC, RUBBER OR GLASS INSTRUMENTS AND APPARATUS INCLUDING CATHETERS, GLOVES, HYPODERMIC EQUIPMENT AND TUBING.

THE SOLUTION SHOULD BE USED FULL STRENGTH. FOR COMPLETE DESTRUCTION OF ALL SPORE-BEARING BACTERIA, AUTO-CLAVING IS RECOMMENDED, AFTER WHICH INSTRUMENTS OR NEEDLES MAY BE STORED IN THIS SOLUTION UNTIL USED.

Instruments must be thoroughly cleaned of contaminating materials such as all adhering blood and serous exudates prior to immersion in disinfecting solution. Immersion time must be at least 10 minutes or longer.

Rinse empty container thoroughly with water and discard it.

NET CONTENTS 1 GALLON

LIBBY LABORATORIES INC.

2214 Sixth St.

Berkeley, Ca. 94710

EPA Est.No.07601CA01

EPA Reg.No.7601-10

