



**UNITED  
OSTOMY  
GERMICIDE  
CONCENTRATED**

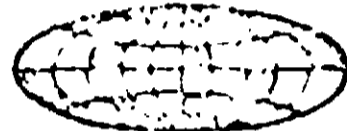

4140

**DANGER  
KEEP OUT OF REACH  
OF  
CHILDREN**

**SEE ANTIDOTE STATEMENT  
&  
OTHER PRECAUTIONS  
ON BACK PANEL**

 **UNITED SURGICAL**  
DIVISION OF Howmedica, Inc.  
LARGO, FLORIDA 33540, U.S.A.

**UNITED OSTOMY GERMICIDE  
CONCENTRATED  
4140**

A solution for the cold disinfecting of ostomy appliances, various tubes and parts used in the management of colostomy, ileostomy and ileal conduit cases. Works equally well on metal, rubber or plastic parts.

**DIRECTIONS**

Clean articles well before immersion. Mix 1 cap (10cc) of Germicide to 1 quart of cold water. Immerse parts in solution for 10 minutes. Parts can then be rinsed in water and handled according to instructions included with appliances.

**ACTIVE INGREDIENTS**

n-Alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chlorides ..... 5%  
n-Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chlorides ..... 5%

**INERT INGREDIENTS** ..... 90%

United Ostomy Germicide is a Concentrate and should be diluted before using.

**DANGER:** Do not get in eyes or on skin. May cause damage to eyes or skin irritation. In case of skin contact, wash thoroughly with water. Get medical attention if skin irritation persists. For eyes, immediately get medical attention. Avoid contamination of food. Do not take internally. Harmful if swallowed.

**FIRST AID:** Do not induce vomiting. Drink large amount of fluids and call a physician immediately.

E.P.A. Reg. No. 7683-2

MADE IN U.S.A.

Contents 1 Pint

Lot No. ....

Rinse empty container thoroughly with water and discard.

**ACCEPTED**  
  
FEB 25 1974  
  
UNDER THE FEDERAL INSECTICIDE  
FUNGICIDE AND RODENTICIDE ACT  
FOR ECONOMIC POISON REGISTERED  
EPA REG. NO. 7683-2 SUBJECT  
TO ATTACHED COMMENTS.

7683-2

Cameraman's Note  
Poor Copy