

DIAL-A-THERM®



ACTIVE INGREDIENTS:

Polyethoxypolypropoxy-
polyethoxyethanol-
iodine complex..... 9.10%
Nonylphenoxypoly-
ethoxyethanol-iodine
complex..... 8.74%

INERT INGREDIENTS:

(Water).....82.16%
(Titratable Iodine 1.6%)

DIAL-A-THERM Germicide acts rapidly killing types I, II, III of the polio virus, Hemolytic Staphylococcus, Streptococcus, Pseudomonas, Aeruginosa and other pathogenic bacteria within ~~five~~ minutes. It is considered non-selective in its action.

IMPORTANT: With proper dilution, Germicide is a dark amber color. When color fades appreciably, replace with fresh solution. Refer to DIAL-A-THERM collateral literature for additional use directions.

many

ACCEPTED
NOV 10 1972
UNDER THE FEDERAL INSECTICIDE
FUNGICIDE AND RODENTICIDE ACT
FOR ECONOMIC POISON REGISTER-
ED UNDER NO. 7809-4 SUBJECT
TO ATTACHED COMMENTS.

7809-4

OCT 27 1972

DIAL-A-THERM®



DIRECTIONS FOR 1:40 DILUTION:
DIAL-A-THERM SR. (holds eleven thermometers.) Measure three (3) capful into plastic container and fill with hard or soft water up to one inch from top.

DIAL-A-THERM JR. (holds five thermometers.) Measure one half (1/2) capful into plastic container and fill with hard or soft water up to one inch from top.

IMPORTANT: With proper dilution, Germicide is a dark amber color. When color fades appreciably, replace, or change weekly.


CAUTION: Avoid contact with skin and eyes. In case of contact, wash with plenty of water. If eye irritation persists, get medical attention. Avoid contamination of food.

CAUTION: KEEP OUT OF THE REACH OF CHILDREN

When empty, rinse container with water and discard it.

CONTENTS: ONE PINT

Mfg. by
MED-RAM, Inc.,
Osakis, Minnesota 56360
EPA Reg. No. 7809-4

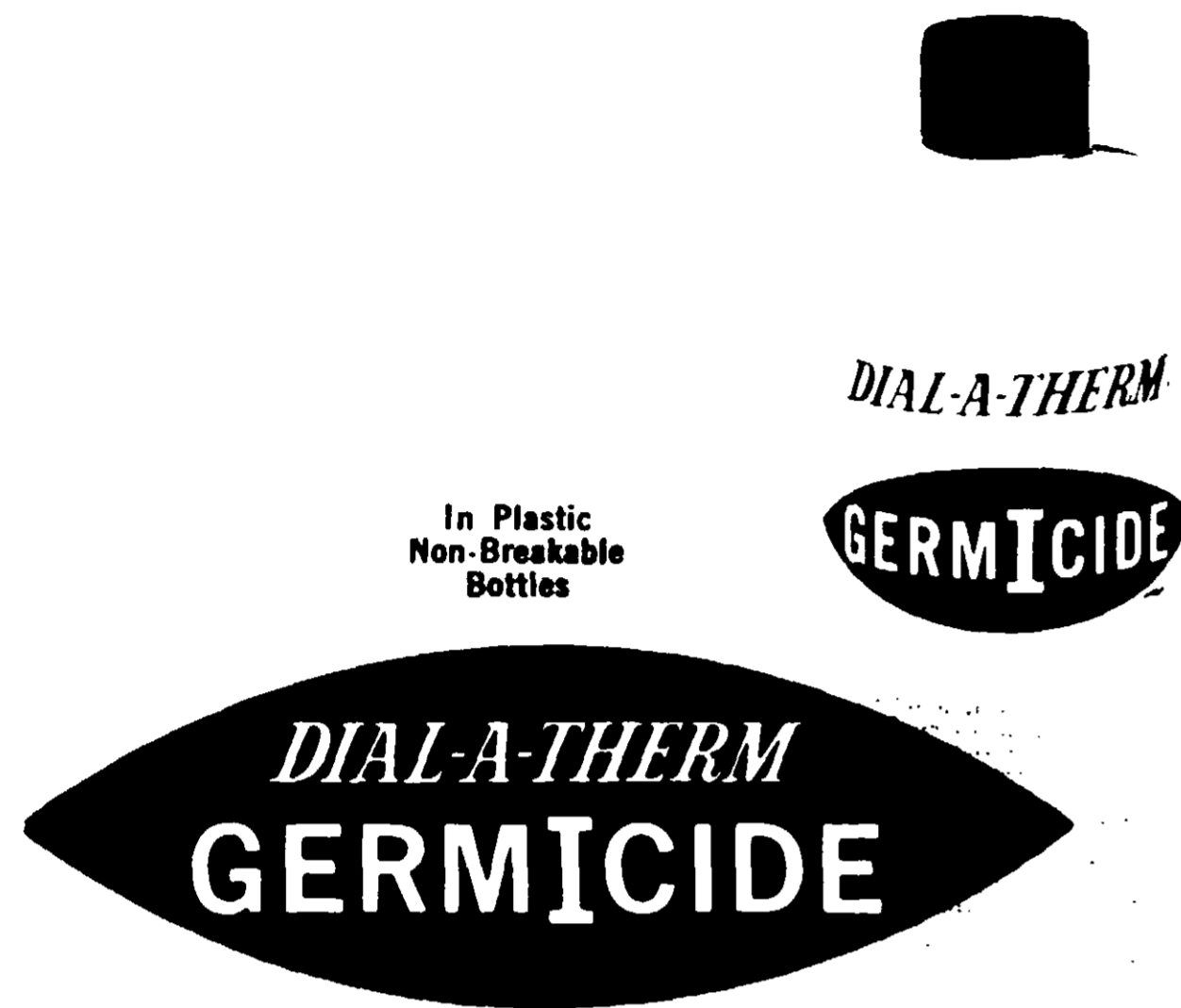

ONE OF THE MOST
CONVENIENT AND
ADEQUATE METHODS
OF THERMOMETER
DISINFECTION IN THE
MEDICAL PROFESSION

ACTIVE INGREDIENTS:
 Polyethoxypolypropoxypolyethoxyethanol-iodine complex 9.10%
 Nonylphenoxypolethoxethanol-iodine complex. . . 8.74%

INERT INGREDIENTS: (Water) 82.16%
 (Titratable Iodine 1.6%)

DIAL-A-THERM Germicide acts rapidly killing types I, II, III of the polio virus, Hemolytic Staphylococcus, Streptococcus, Pseudomonas aeruginosa and many other pathogenic bacteria within ten minutes. It is considered non-selective in its action.

IMPORTANT: With proper dilution, Germicide is a dark amber color. When color fades appreciably, replace with fresh solution. Refer to DIAL-A-THERM collateral literature for additional use directions.

CAUTION: Avoid contact with skin and eyes. In case of contact, wash with plenty of water. If eye irritation persists, get medical attention. Avoid contamination of food.

CAUTION: KEEP OUT OF REACH OF CHILDREN. When empty, rinse container with water and discard it.

Reg. by MED-RAM, Inc., Osakis, Minnesota 56360

ACCEPTED

NOV 10 1972

UNDER THE FEDERAL INSECTICIDE, FUNGICIDE AND RODENTICIDE ACT FOR ECONOMIC POISON REGISTERED UNDER NO. 10094-1 SUBJECT TO ATTACHED COMMENTS





DIAL-A-THERM

1972

EXTRAORDINARY DISINFECTANT PROPERTIES

DIAL-A-THERM GERMICIDE

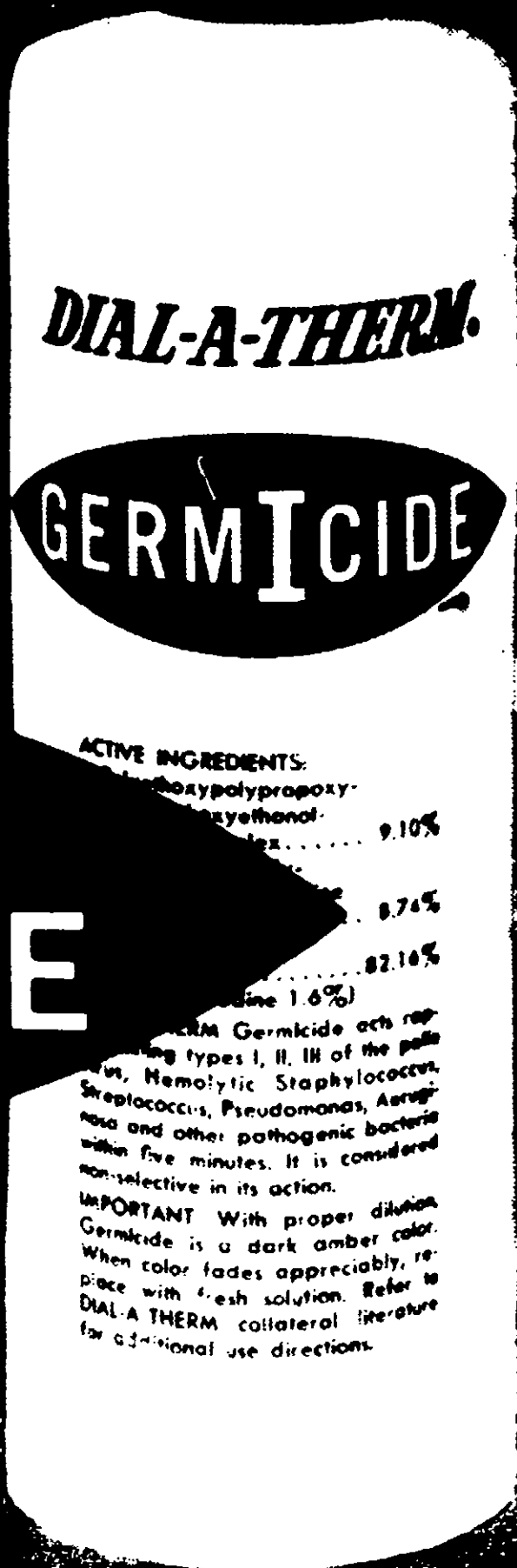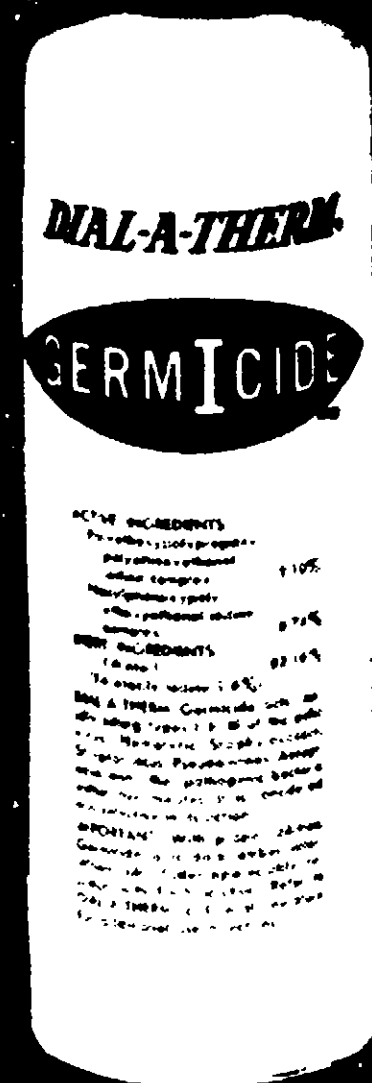

ACTIVE INGREDIENTS:
 Polyhexamethylene biguanide hydrochloride 9.10%
 Benzalkonium chloride 0.74%
 Chlorine 0.216% (available 1.6%)

INERT INGREDIENTS: 90.054%

Directions:
 DIAL-A-THERM Germicide acts rapidly killing most I, II, III of the pathogenic bacteria, Streptococcus, Pseudomonas, Aeromonas and many other pathogenic bacteria.

IMPORTANT: With proper dilution Germicide is a dark amber color. When color fades appreciably, replace with fresh solution. Refer to DIAL-A-THERM collateral literature for additional use directions.

ACTIVE INGREDIENTS:
 Polyhexamethylene biguanide hydrochloride 9.10%
 Benzalkonium chloride 0.74%
 Chlorine 0.216% (available 1.6%)

INERT INGREDIENTS: 90.054%

Directions:
 DIAL-A-THERM Germicide acts rapidly killing most I, II, III of the pathogenic bacteria, Streptococcus, Pseudomonas, Aeromonas and many other pathogenic bacteria.


IMPORTANT: With proper dilution Germicide is a dark amber color. When color fades appreciably, replace with fresh solution. Refer to DIAL-A-THERM collateral literature for additional use directions.




\$8⁹⁵

MODEL S
DIAL-A-THERM
JUNIOR

Never fill all six openings. Remove for use clockwise. In rotation.

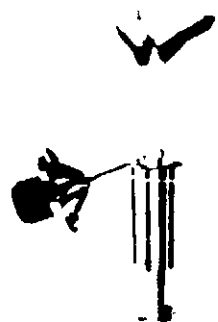

Replace counter-clockwise, after wiping free of organic material.

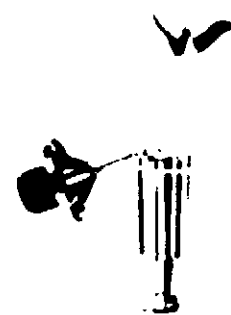

Repl
for c
sub




SANI SAFE THERMOMETERS

MARKINGS GUARANTEED PERMANENT
CANNOT BE REMOVED BY ANY GERMICIDE

1. No rough etched markings for bacteria accumulation.
2. Sure grip Tri Top for safe shaking down.
3. Meets or exceeds Mich. Seal specifications for accuracy.
4. Markings guaranteed permanent.

4 for
\$3⁰⁰

IN THE DISPENSARY FOR MU

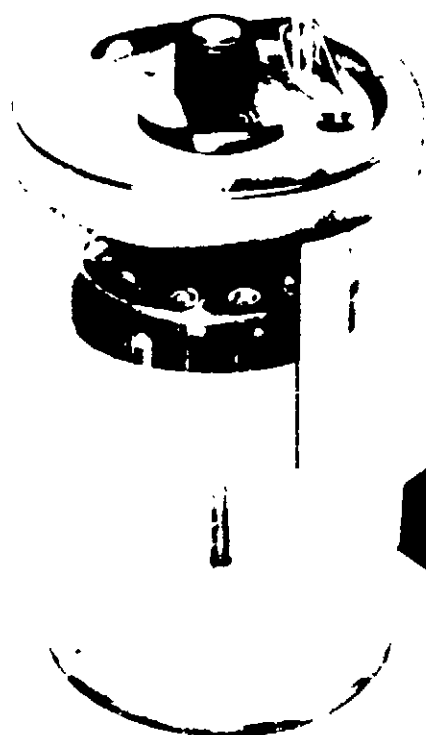

\$19⁵⁰

MODEL L
DIAL-A-THERM
SENIOR

Dial a thermometer.



Remove and use.



Drop
men
turn
off
the

DIAL-A-THERM care reduces thermometer breakage

