

DIAL-A-THERM.



ACTIVE INGREDIENTS:

Polyethoxypolypropoxy-
polyethoxyethanol-
iodine complex.....9.10%

Nonylphenoxypoly-
ethoxyethanol-iodine
complex.....8.74%

INERT INGREDIENTS:

(Water).....82.16% (

(Titratable Iodine 1.6%)

DIAL-A-THERM GERMICIDE inactivates
types I,II,III of the polio
virus, Hemolytic Staphylococcus
Aureus, Staphylococcus Epidermidis
Streptococcus Fecalis, Streptococcus
Pyrogenes, Pseudomonas aeruginosa and
many other gram positive and gram negative
bacteria within 10 minutes.

IMPORTANT: With proper dilution, Germicide
is a dark amber color. When color fades
appreciably, replace with fresh solution.
Refer to DIAL-A-Therm collateral literature
for additional use directions.

"Keep Out of Reach of Children"

"Caution"

"See other precautions on back panel."

ACCEPTED

APR 2 1997

Under the Federal Insecticide, Fungicide, and
Rodenticide Act, as amended, for the
pesticide registered under
EPA Reg. No. 4809-4

JAN 27 1996

2 92

DIAL-A-THERM.



DIRECTIONS FOR USE:

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

DIRECTIONS FOR 1:40 DILUTION:

DIAL-A-THERM SR. (holds eleven thermometers). Measure three (3) capsful into plastic container and fill with 400 ppm hard or soft water up to one inch from the top.

DIAL-A-THERM JR. (holds five thermometers.) Measure one half capful into plastic container and fill with 400 ppm hard or soft water to one inch from the top.

IMPORTANT: with proper dilution, GERMICIDE is a dark amber color. When color fades appreciably, replace or change weekly.

"Precautionary Statements"

"Hazards to Human and Domestic Animals"


CAUTION: Avoid contact with skin and eyes. In case of contact, wash with plenty of water. If eye irritation persists, get medical attention. Avoid contamination of food.

"Disposal"

When Empty, rinse container with water and discard it.

CONTENTS: ONE PINT

Mfg. by
MED-RAM, INC.
PO BOX 550
Osakis, Mn. 56360
EPA Reg. No. 7809-4

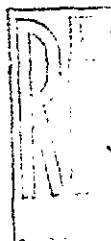

JAN 27 1996

