

U.S.D.A. REG. NO. 8354-1

CAUTION

Not for sterilization of instruments contaminated with spore-bearing pathogens, viral hepatitis, tuberculosis or rusty instruments. Scrub instruments with soap and water until free of organic matter and rinse thoroughly. Contaminated instruments and hypodermic needles should always be sterilized by autoclaving. For technical use of hospitals, medical, dental, and veterinary professions.

STANDARDIZED



REAGENTS

**ALCOHOLIC
RUST PREVENTING**

**READY FOR USE
NON-CORROSIVE**

**STERI-SOL
GERMICIDAL SOLUTION**

ALCOHOLIC SOLUTION 1:750

ACTIVE INGREDIENTS:

ISOPROPYL ALCOHOL	50.000 %
HIGHER MOLECULAR ALKYL DIMETHYL BENZYL AMMONIUM CHLORIDES (C ₁₄ -58%, C ₁₆ -20%, C ₁₂ -14%)	1.133 %
INERT INGREDIENTS: (WATER)	49.867 %

Not to be taken internally. Avoid getting the concentrate in eyes. In case of contact, flush with plenty of water. If irritation persists, get medical attention. Avoid contamination of foods.

CAUTION: KEEP OUT OF REACH OF CHILDREN

MEDICAL CHEMICALS CO. INC.
Sampa/Monroe, Calif.

DIRECTIONS

Instruments should be thoroughly cleaned of contaminating materials and immersed in solution for at least 15 minutes or longer.

Hollow instruments should be slightly tilted in the solution to displace the trapped air pockets. Upon removal from the solution and before storing instruments, wipe dry with sterile towel.

Do not use with soap or other cleaning agents.

1 Gallon Net

ACCEPTED
OCT 11 1967
UNDER THE FEDERAL
FUNGICIDE AND
FOR ECONOMIC
ED UNDER NO. 8354-1
TO ATTENTION